

# MyotonPRO Digital Palpation Device USER MANUAL

Device model: 1B, device firmware v.1.8.1  
Myoton Desktop Software v.5.0.0.261

**For Research Use Only**

Revision 32  
10<sup>th</sup> July 2024



## Copyright

Copyright © 2010, Myoton AS  
Address: Lootsa Str.8A,  
Tallinn 11415, Estonia  
e-mail: [contact@myoton.com](mailto:contact@myoton.com)  
web: [www.myoton.com](http://www.myoton.com)

## Trademarks

Windows and MS Excel are registered trademarks of Microsoft Corporation.

All the other trademarks and product names in this document are registered trademarks of Myoton AS.

## WARRANTY

The materials and technological appliances of the MyotonPRO (hereinafter referred to as “the Device”) have a two years warranty period if not agreed otherwise starting from the selling date of the item.

During the warranty period, Myoton AS undertakes to repair and/or replace the defective parts of the Device.

No other warranties will apply to the Device.

Myoton AS will apply the warranties mentioned above only for the purposes described below.

Myoton AS will take responsibility for the safety and reliability of service of the Device only if:

- the Device is assembled or dismantled, changed or repaired by people authorised by Myoton AS;

- the Device is used in accordance with the guidelines provided in the product user manual.

## WARRANTY RESTRICTIONS

The warranties described above will not apply to defects that have been caused to the Device by:

- improper storage by the buyer; or
- mishandling; or
- use of the Device outside the environment described in the specification of the product.

In addition to the restrictions described above, the warranty will not be extended to the cable, battery, power adapter or skin marker.

**Warranty will NOT apply** if the Device housing has been opened or any change has been made, or damage done to the Device.

## MAINTENANCE

**The Manufacturer guarantees the device DOES NOT need regular maintenance or re-calibration.**

Repair, battery change, calibration and testing can be performed ONLY by the Manufacturer. Depend on Device usage the battery may need replacement with new one in every 5 to 10 years of service.

## Global Medical Devices Nomenclature (GMDN)

The five-digit GMDN code is defined in “EN ISO 15225, Annex A, Code 04 (Electro-mechanical medical devices)”.

## European Legislation on Medical Devices

The MyotonPRO Digital Palpation Device is FOR RESEARCH USE ONLY. The Device conforms essential safety requirements of MDD 93/42/EEC + 2007/47/EC.

The Device is designed and manufactured in accordance with EN 60601-1-2: 2007; EN 55011: 2007 + A2: 2007; EN 61000 Part 3 and Part 4.

## Storage of the product and opening of the package

When unpacking, please check that the package has not been damaged during the shipping. In case of any damage to the package, please contact the Manufacturer.

Open the package and compare its contents with the list of product items in the documentation.

One package contains one set of the Device. If the content of the package does not conform to the list of documentation, contact the Manufacturer immediately.

## Manufacturer:

Myoton AS,  
Address: Lootsa Str.8A,  
Tallinn 11415, Estonia  
e-mail: [contact@myoton.com](mailto:contact@myoton.com)  
website: [www.myoton.com](http://www.myoton.com)

# Contents

Contents .....	3
Terminology.....	5
System requirements .....	7
1. General information .....	8
1.1 Icon glossary .....	8
1.2 Contents of the package.....	10
1.3 Accessories .....	11
1.4 Device description .....	12
1.4.1 Measurable parameters .....	12
1.4.2 Device overview.....	13
1.4.3 Areas of application.....	14
1.4.4 Method of measurement .....	14
1.4.5 Limitations .....	15
2. Important information.....	16
2.1 Safety and precautionary measures.....	16
2.1.1 Data protection measures .....	16
2.2 Requirements for the measurement process.....	17
2.2.1 The range of measurement axis.....	17
2.2.2 Environmental requirements .....	18
2.2.3 Subject preparation .....	18
2.2.4 Device requirements .....	19
2.2.5 Evaluation of the measurement variability .....	20
2.3 Measurement interference detection system.....	21
2.3.1 Evaluation of a Tissue oscillation signal.....	21
2.3.2 Optimal measurement impulse time .....	22
2.3.3 Probe jump-off .....	22
2.3.4 Non-inherent frequencies .....	23
3. Application of MyotonPRO .....	24
3.1 Quick instruction.....	24
3.2 Preparations for use .....	25
3.2.1 Charging the battery.....	25
3.2.2 Connecting the Device to the computer .....	27
3.3 Test mode .....	29
3.3.1 Activation of Test mode in the Device.....	29
3.3.2 Activation of the Test mode in Software.....	33
3.4 Preparation of the Measurement Set.....	34
3.4.1 Input of Subject’s data.....	34
3.4.2 Composing Patterns .....	38
3.4.3 Uploading the Measurement Set .....	44
3.5 Device measurement mode.....	47
3.5.1 Measurement with the Device .....	47
3.5.2 Display of the Acceleration signal .....	51
3.5.3 Input of new Subjects .....	52
3.5.4 Selection of Objects in a different order .....	53
3.5.5 Change of the impulse time .....	54
3.5.6 View of previous measurement results.....	55

3.6	Visualisation of measurement results in the Software .....	57
3.6.1	Download of measurement results .....	57
3.6.2	Composing the <i>Symmetry</i> report .....	59
3.6.3	Composing the <i>Multiscan Analysis</i> report.....	61
3.6.4	Composing the <i>Comparison</i> report .....	63
3.6.5	Composing the graphs.....	65
3.6.6	Display of the acceleration signal .....	67
3.6.7	Muscle function <i>Estimation</i> graph.....	69
3.7	Export of the measurement results.....	72
3.7.1	Automatic export to Microsoft Excel.....	72
3.7.2	Manual export .....	73
3.8	Data management .....	76
3.8.1	Data backup of the Device.....	76
3.8.2	Deletion of backup from the Device.....	77
3.8.3	Software database management .....	78
3.8.4	Import of database .....	79
3.8.5	Deletion of measurement results from the Software .....	80
4.	Menu instructions.....	82
4.1	Device menu .....	82
4.1.1	The main screen .....	82
4.1.2	Device Shut Down.....	84
4.1.3	Device system info.....	85
4.1.4	Device settings.....	86
4.1.5	Device probe replacement .....	87
4.2	Software menu .....	88
4.2.1	The main window .....	88
4.2.2	<i>Upload Preparation</i> window.....	89
4.2.3	<i>Results</i> window.....	91
4.2.4	<i>Manual export to CSV</i> .....	94
4.2.5	<i>Device settings</i> window .....	95
4.2.6	<i>Test measuring</i> window.....	96
4.2.7	<i>Locations</i> window .....	97
4.2.8	<i>Preferences</i> window .....	98
4.2.9	<i>Deleted items</i> window .....	100
	Appendixes .....	101
	Appendix 1 – Definitions of parameters.....	101
	F – Oscillation Frequency [Hz] .....	101
	S – Dynamic Stiffness [N/m] .....	101
	D – Logarithmic Decrement.....	101
	R – Mechanical Stress Relaxation Time [ms].....	101
	C – Ratio of Relaxation and Deformation time.....	101
	Appendix 2 – The formulas and Legend of the Acceleration Signal .....	102
	Appendix 3 – Technical Specification .....	104
	Appendix 4 – List of Error Messages .....	107
	Appendix 5 – Environmental Conditions .....	109
	Important information regarding Electro Magnetic Compatibility (EMC) .....	110
	Mechanical and electrical safety .....	110
	Correct Disposal Electrical & Electronic Equipment.....	110

# Terminology

## Following terminology used in the User Manual, Software and Device:

- **Device** – the MyotonPRO Digital Palpation Device, also called in the literature as Myotonometer, Myometer, Myoton technology;
- **Software** – the Myoton Desktop Software;
- **Computer** – a computer with Myoton Desktop Software installed for Microsoft Windows;
- **Subject** – a person being measured;
- **Operator** – a person performing measurements;
- **Pattern** – a Measurement protocol consisting: Pattern name, Measurement mode, Object names, and Measurement profiles;
- **Measurement** – a Measurement series that consist of single measurements, conducted under the measurement mode;
- **Measurement mode** – the mode in which the Measurement is conducted, Triplescan or Multiscan;
- **Triplescan** – a measurement mode consists of three single measurements. The device displays the median of the Frequency parameter. Other parameters: S, D, R, and C, displayed on the device screen, correspond to that particular single measurement.
- **Multiscan** – a measurement mode which consists of more than three single measurements (by default five single measurements). The Device displays the average value.;
- **CV** – Coefficient of Variation of the Measurement, to describe relative dispersion of data points;
- **Tissue** – the soft biological tissue;
- **Tissue layer** – a set of different types of tissues located under the Probe and being involved in the Measurement (e.g. skin, adipose fat, fascia, skeletal muscles, tendons);
- **Object** – the Latin name of a tissue being measured (a skeletal muscle, tendon, skin, etc.);
- **Muscle** – a superficial skeletal muscle;
- **Tendon** – a superficial skeletal tendon;
- **Accel.Graph** – the raw acceleration signal of an Object's natural oscillation;

- **Co-oscillation** – an oscillation consisting of an Object, pre-compressed subcutaneous tissues, the Probe and the Device measurement system;
- **Measurement set** – a list of Subjects and Patterns selected for upload to the Device;
- **Measurement result** – a result containing the set of measured parameters (chapter 1.4.1);
- **Upload** – a transfer of the Measurement set from the Software to the Device;
- **Download** – a transfer of the Measurement results from the Device to the Software;
- **Probe** – the flat end cylindrical polycarbonate rod (standard diameter 3mm);
- **Probe disk** – the probe disk attachment (chapter 1.3);
- **Measurement position** – is the Probe position within the range of 3mm, indicated with the green light, and where the Device automatically conducts the Measurement;
- **Delay** – the time delay preceding the first measurement impulse of a Measurement (default Delay 0.7 seconds);
- **Tap time**<sup>1</sup> – the measurement impulse time duration (default impulse 15ms);
- **Tap Interval** – the interval between the measurement impulses in a Measurement Series (default value 0.8 seconds);
- **Measurement profile** – a list of aspects of how an Object is measured (see below);

**The Measurement profile consists of:**

1. **Tissue** – the type of Object being measured:
  - *Muscle*
  - *Tendon*
  - *Ligament*
  - *Skin, Other*
2. **Object** – the Latin name of an Object being measured;
3. **Position** – a Subject position when measured:
  - *Lying*
  - *Sitting*
  - *Standing*
4. **Side** – the body side being measured:
  - *Left*
  - *Right*

---

<sup>1</sup> The standard Tap time of 15 milliseconds is optimal for most of the measurements. By changing the Tap time, the Impulse Force will be changed as **Impulse Force = Force \* impulse duration**. The impulse force is not separately adjustable.

- *Centre (Axial)*
5. **Location** – anatomical location of the measurement point on Object:
- *Central* – on the middle part, e.g. on the muscle belly
  - *Superior* – on the upper part
  - *Inferior* – at the lower part
  - *Medial* – on the side of the centre line of the body
  - *Lateral* – on the side of the outward line of the body
6. **State** – the state of the Object being measured:
- *Relaxed (complete relaxation, full rest)*
  - *Contracted (low, medium, sub-maximum, maximum)*

## System requirements

The Device and Software is a stand-alone system.

### Running on Windows

The Software is for Windows Operating System starting from Windows XP or later versions (32 or 64-bit Operating System).

### Running on Mac

The Software can be operational on a Mac computer equipped with Windows OS as the mandatory operation system. Mac and Windows OS can be simultaneously used via Parallels Desktop Software. Read for more at <https://www.parallels.com>.

### Disk space

Installation of the Software requires at least 25Mb of free disk space. Software database files (file extension .sdf) will require additional disk space. The size of database files depends on the amount of collected data which may be up to the 10th of Mb.

### Display size

The minimum required display size is 12-inch.

### Analytics

For the automatic export of Measurement results for analytical purposes, it is recommended to install Microsoft Excel as a part of the MS Office package.



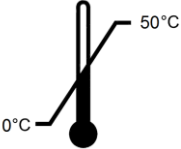
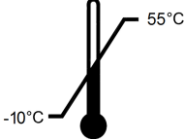
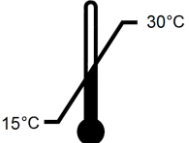



The Software contains a feature of an automatic data export to MS Excel (chapter 3.7.1.). MS Excel provides comprehensive tools for data processing, statistics and visualisation of the results.

### Communication







The Device connects to the computer via USB cable (chapter 3.2.2).

# 1. General information

## 1.1 Icon glossary

Symbol	Description
	Symbol for CE Mark. This symbol certifies that MyotonPRO palpation device is designed, manufactured and in full compliance with the following standards: EN 60601-1-2: 2007 – Medical Electrical Equipment Part 1-2 EN 55011:2007+A2:2007 – Industrial, scientific and medical (ISM) radio frequency equipment EN 61000 Part 3 and Part 4 – Electromagnetic compatibility (EMC)
	Serial number. Indicates the manufacturer's serial number so that a specific device can be identified.
	Operating temperature limits.
	Transportation temperature limits.
	Suggested operating temperature limits.
	According to the IEC 60601-1 MyotonPRO is a type B device which has a protection against electric shock. The leakage current of the device remains within the valid norms.
	<b>Attention</b> , Follow instructions for use!
	Do not disassemble or modify this Device.





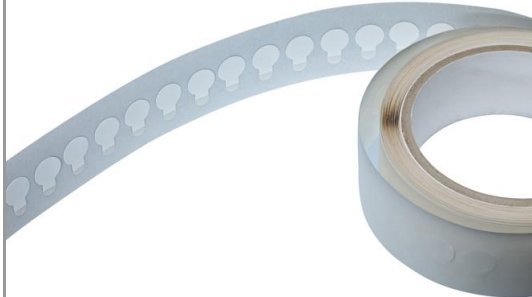
	<p>Non-sterile. Indicates a device that has not been subjected to a sterilization process.</p>
	<p>Do not use if the package is damaged or opened. Please contact the local representative or manufacturer.</p>
	<p>Keep dry.</p>
	<p>Fragile - handle with care.</p>
	<p>Do not discard with household waste. Waste handling according to local rules.</p>
	<p>Transport package is made of a recyclable material.</p>

## 1.2 Contents of the package

The package consist:	Photo
<p><b>Digital Palpation Device</b> – for the non-invasive measurement of biomechanical and viscoelastic properties of soft biological tissues.</p> <p>See Appendix 3 – Technical Specification</p>	
<p><b>1. USB cable, A plug to Type-C plug</b> – for charging the battery and the Device and Software communication.</p>	
<p><b>2. AC Power Adapter</b> with USB DC socket – for charging the battery from the mains power supply.</p> <p>See Appendix 3 – Technical Specification</p>	
<p><b>3. Medical Skin Marker</b> – for marking measurement points on the skin.</p> <p>ALL SKIN, with prep resistant ViXL ink, 1447XLB</p>	

## 1.3 Accessories

Following accessories can be separately ordered. Please contact Myoton AS for quotation.

Description	Image
<p><b>1. Probes disks</b> – increases the contact area and tissue's ability to respond to the measurement impulse and/or reduces the depth of measurement. (chapter 2.3.1 and 2.3.2).</p> <p>1. the probe with the disk attached;            2. disk D=10mm;            3. disk D=12mm;            4. the standard probe D=3mm.</p>	
<p><b>2. The L-shape skin probe (medium arm)</b> – The probe enables isolated measurements from deeper underlying adipose layers or muscles tissues.</p>	
<p><b>3. Double sided skin tape</b> – attaches the skin probe to the skin and prevents the probe from sliding on the skin surface (chapter 2.3.3).</p> <p>The stickers are available in roles. One role consists of 500 stickers. Sticker diameter is 12mm, thickness 0,1mm.</p>	



**For thin tissues, increase the probe contact area by using the Probe disk D=10mm. The Probe disk reduces the measurement depth or helps thin facial Muscles better to respond to the measurement impulse (chapter 1.3 p.1).**

## 1.4 Device description

The Device is for the non-invasive measurement of biomechanical and viscoelastic properties of superficial soft biological tissues. Skeletal muscles or tendons can be measured at different regions, at full rest or in contraction.

The Device measurement system enables measurements in any direction to the vector of the gravity and in weightlessness (zero gravity).

Mathematically, the measurement result describes the property of the tissue layers involved in the measurement as a weighted average.

### **The tissue types that can be measured:**

- single superficial skeletal muscles or different regions,
- tendons and ligaments,
- fascia and underlying fibrous layers,
- skin (experimental use),
- other superficial soft biological tissues.

### **The Device features:**

- non-invasive and painless,
- reliable, reproducible, repeatable,
- fully automatic measurement sequence,
- measures at any angle to the vector of the gravity,
- functional in different gravity fields (from Earth's gravity to zero gravity),
- the depth of measurement 20-30mm, depending on dampening of underlying tissue structures,
- measures underlying tissues in 3D (three dimensionally);
- hand-held, cost-effective, quick and easy to use.

### 1.4.1 Measurable parameters

**The Device measures (computes simultaneously) the following five parameters<sup>2</sup>:**

#### **• State of tension:**

1. F - Natural oscillation frequency [Hz] characterising TONE<sup>3</sup> or state of tension;

#### **• Biomechanical properties:**

2. S - Dynamic STIFFNESS [N/m];
3. D - Logarithmic Decrement, characterising ELASTICITY or dissipation of natural oscillation;

#### **• Viscoelastic properties:**

4. R - Mechanical Stress Relaxation time [ms];
5. C - ratio of Relaxation time to Deformation time, characterising CREEP (Deborah number).



---

<sup>2</sup> Definitions of parameters – see Appendix 1.

<sup>3</sup> Muscle Resting Tone must be measured at full rest. Muscle measurement in contraction characterises the state of functional tension.

**Classification of tone:** Passive Tone, Resting Tone, Postural Tone.

## 1.4.2 Device overview

Front of the Device	Rear of the Device
	
<ol style="list-style-type: none"> <li>1. USB socket,</li> <li>2. Touch-screen,</li> <li>3. Measurement menu,</li> <li>4. Device menu,</li> <li>5. <i>ON/NEXT</i> button,</li> <li>6. Probe light guide,</li> <li>7. Probe,</li> <li>8. Measurement position marker,</li> <li>9. Battery charging LED,</li> <li>10. Standby lights,</li> <li>11. Help menu,</li> <li>12. Device Operator name,</li> <li>13. Date &amp; time,</li> <li>14. Battery level indicator,</li> </ol>	<ol style="list-style-type: none"> <li>15. Wrist belt fixation,</li> <li>16. Product label,</li> <li>17. RESET button - see Appendix 5,</li> <li>18. Speaker.</li> </ol>

## 1.4.3 Areas of application

MyotonPRO Digital Palpation Device is for research use only!

The Device can be used for basic science research, clinical research and other types of research.

### Potential application areas:

#### 1. Clinical medicine:

- monitoring the changes in tissues caused by health disorders,
- disease progression monitoring;
- obtaining supplementary information for health or disease diagnosis,
- intervention efficacy evaluation,
- physiotherapy or rehabilitation outcome monitoring;

#### 2. Sports medicine:

- monitoring of key Muscles training efficacy,
- detection of fatigue and Muscle's overtraining,
- determination of the optimal training load for Muscles,
- prevention of Muscle and tendon injuries;

#### 3. Occupational health care:

- prevention of trauma caused by Muscle overload due to uncomfortable positions assumed when working over a prolonged period,
- monitoring the changes to Muscle condition due to uncomfortable working positions assumed over a prolonged period,
- obtaining supplementary information when diagnosing Muscle diseases;

#### 4. Veterinary medicine:

- prevention of Muscle and tendon injuries,
- evaluation of the applied interventions or treatments.

## 1.4.4 Method of measurement

The Device measurement method is non-invasive, safe and painless, easily applicable anywhere on the field, cost-effective and practical in clinical ward settings.

The method consists of recording the damped natural oscillation of soft biological tissue in the form of an acceleration signal and the subsequent simultaneous computation of the parameters, including State of Tension, Biomechanical and Viscoelastic properties.

### To measure:

1. place the Probe perpendicular to the skin surface over an Object in interest;
2. lower the Device towards the skin until green light illuminates;
3. hold the Device gently and steadily in the Measurement position;

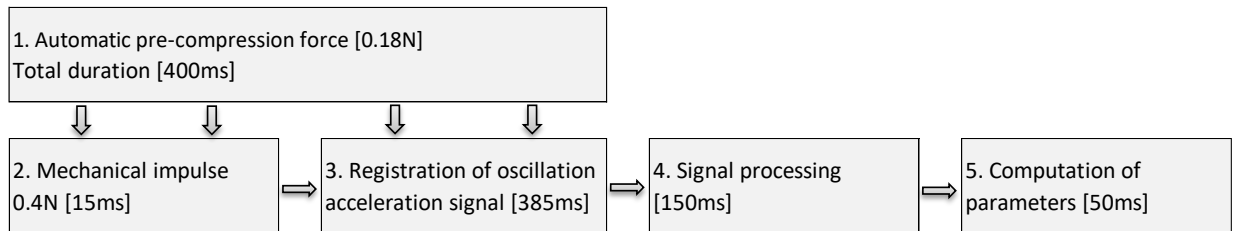
4. the Device performs the pre-defined Measurement automatically.

The Measurement series consisting of single measurements is executed with an interval of 0.8 seconds.

**The Measurement consists of following automatic stages:**

1. Pre-compression of subcutaneous adipose tissues throughout the measurement;
2. Mechanical impulse followed with quick release;
3. Registration of tissue response to the mechanical impulse in the form of acceleration signal of damped natural oscillation;
4. Processing of the raw acceleration signal;
5. Simultaneous computation of parameters.

**A diagram of the measuring process:**



The duration of the measurement impulse is 15ms and involves minimal force (pre-load 0.18N + impulse 0.42N = **0,60N**), hence, it does not cause residual mechanical deformations nor neurological reactions of the tissues being evaluated.

## 1.4.5 Limitations

**The Device cannot measure:**

- Isolated tissues or tissue layers;
- Muscle groups;
- Thin muscles or tissues (close to the skin surface, supported by bone surfaces), but below < 2mm of its thickness;
- Small muscles between other muscles other tissue layers;
- Un-palpable deep Muscles or tissues located below other tissue layers;
- Muscles covered by subcutaneous fat over >20mm of its thickness;
- Tissues that are not classified as soft biological tissue.

**The following can compromise meaningful measurement results:**

- the Muscle or tendon is measured at its maximum or minimum length;
- visual or gentle palpatory test of a Muscle location is obstructed with a thick subcutaneous fat layer;
- the Subject’s body position is not convenient for full relaxation;
- continuous muscle tremor due to a neurological condition;
- an Operator’s device holding hand is not supported and trembles.

## 2. Important information

### 2.1 Device safety and precautionary measures

- Do not attempt to disassemble the Device under any circumstances! Unauthorised disassembly of the Device will void the warranty and cause damage to the Device!
- Do not place heavy items on it, Do not drop or apply strong shock.
- Do not place the Device near strong permanent magnets or strong electromagnetic fields.
- Do not submerge<sup>4</sup>, rinse, or spray excessive amounts of any liquids to clean the device.

The device can be sterilized only in a UV Sterilizer Cabinet, without heat and moisture. Cold sterilization, using approved liquid sterilants and disinfectants, is allowed only for the probe.

#### **Cleaning the Device by Alcohol Wipes:**

The device housing can be cleaned on the outside using semi-dry medical cotton wipes containing either isopropyl or isopropanol alcohol. Retrieve an alcohol wipe and unfold it. Gently and thoroughly wipe the surface of the device with the alcohol wipe. Ensure all surfaces of the device are wiped, paying attention to crevices, corners, and any areas that are touched frequently. Extend the probe by pressing the "Probe" button, pull out the probe, clean it thoroughly with a separate alcohol wipe, and position it back into the device. Press Done.

#### **Do not store the Device:**

- under direct sunlight, dusty or salty environments,
- on inclined surfaces, or subject to vibration and or shock,
- in freezing or high temperatures or high humidity.

#### **Precautions:**

- Disconnect the Device from the adapter and mains power supply before use.
- Do not use the Device if damaged and contact the manufacturer.
- The repair or battery change, re-calibration or testing can be performed **ONLY** by the manufacturer.
- To ensure the best use of the Device, it is recommended to take training from trainers authorised by the manufacturer.

#### 2.1.1 Data protection measures

Use confidentiality and data protection measures, such as:

- Ensure that any confidential personal data contained in the Software and the Device will not be disclosed or accessed by unauthorised third parties;
- storage of the Device in a secured place, not accessible to unauthorised personnel;
- download the results and delete the entire measurement history before handing the Device over to third parties (chapter 3.8.2).

---

<sup>4</sup> Rinsing or submerging the device in any liquid will cause permanent damage to its electronics and inevitable malfunction, making repair impossible and voiding the warranty.




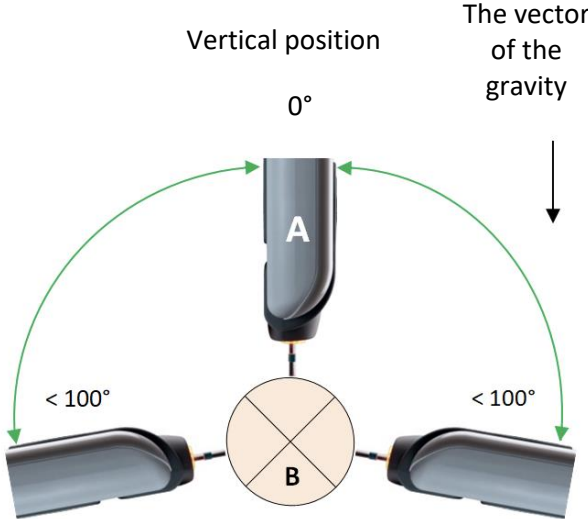
## 2.2 Requirements for the measurement process

This chapter focuses on the information and procedures of reliable measurement results.



Please read the following instructions carefully!

### 2.2.1 The range of measurement axis

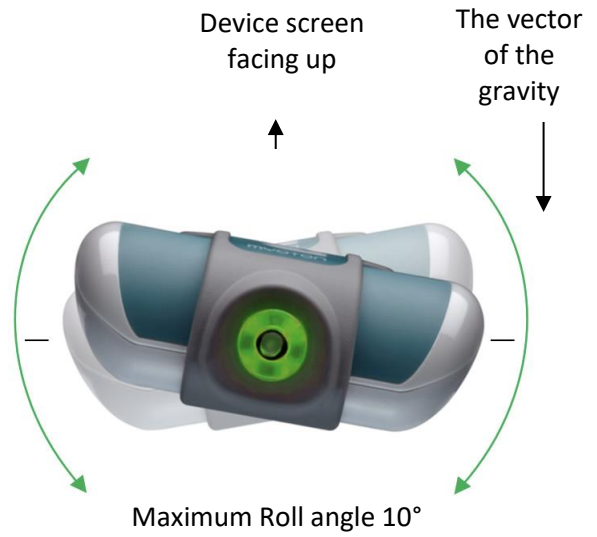
Description	Figure
<p>1. When measuring, keep the Probe always perpendicular to the skin surface above the Object being measured.</p>	
<p>2. Avoid<sup>5</sup> the angle of measurement exceeding 100° from the vertical position.</p> <p>A – Device side view B – an Object cross-section view</p>	

<sup>5</sup> Avoid measurements opposite to the vector of gravity, i.e. the Probe facing up. The force of the gravity suppresses the measurement impulse and subsequent oscillation of an Object being measured.

3. When measuring horizontally, keep the Device screen facing upwards  $\pm 10^\circ$ .

If the Device is rolled more than  $10^\circ$ , a warning notice **Rotate!** will be displayed.

Roll the Device round the probe axis until the notice disappears.



## 2.2.2 Environmental requirements

The measurement environment should be favourable for conducting measurements with minimal external disturbances. Avoid disturbing factors such as bright light, loud noises, vibration, too hot or cold ambient temperature or participation of unauthorised persons.

The most suitable temperature for measurements is room temperature, e.g.  $22^\circ\text{C}$  ( $72^\circ\text{F}$ ).



---

**For eg. hot temperature decreases the stiffness of tissues, but cold temperature increases the stiffness.**

---

## 2.2.3 Subject preparation

Explain to new Subjects the measuring procedure is non-invasive, painless, and convenient. The explanation aims to ensure the person feels good and relaxed.

Instruct the Subject to arrive in good time for at least five minutes before the session. If necessary, instruct Subjects to avoid intensive physical activities or consume medications for at least three hours before the measurement session. Some medicines may affect the measurements results.

The long-term monitoring measurements are advisable to conduct approximately at the same time of the day and environments so that the Subject's condition would be as similar as possible throughout the measurement program.

**1. Ensure following:**

- the Subject's position allows to be fully relaxed and is easily reproducible in subsequent measurement sessions. Use an examination or massage table if possible;
- allow the Subject to rest for at least 5 minutes prior to the measurements (use this time for preparatory activities, e.g. marking the measurement points);
- clothes or items are not in contact with the Tissue being measured, as it may obstruct free oscillation and influence results;
- the Muscle is not measured at its maximum or minimum length<sup>6</sup>;
- the Object is supported, either naturally<sup>7</sup> by the skeleton or by artificial<sup>8</sup> means;
- repetitive or comparative measurements are always taken at the same measurement point and the constant Muscle length;
- when positioning the Subject, ensure the repetitive or comparative measurements are always taken at the same angle;
- during the measurements, the Subject focuses on full rest.

**2. On establishing and marking the measurement point:**

- establish the location and contours<sup>9</sup> of the Object and mark the measurement point by the **Medical Skin Marker** (chapter 1.2);
- in case of a muscle measurements, mark the measurement point above the Muscle belly (above the biggest cross-section);
- ensure that the measurement points are marked symmetrically when measuring the same Object on either side of the body;
- check immediately before measuring, by both the palpatory test and questioning the Subject, whether the Object being measured is fully relaxed.



---

**Whenever necessary, take a photo of the Subject's body position to reproduce the same body position for subsequent measurement sessions.**

---

## 2.2.4 Device requirements

**When measuring with the Device ensure:**

- 1. the characteristics of the mechanical impulses of repetitive or comparative measurements are always identical **The Probe**<sup>10</sup> – standard probe diameter 3mm.
- **Tap time** – default value 15ms,
- **Tap interval** – default value 0.8 sec,

---

<sup>6</sup> Unless if it is the aim of the study.

<sup>7</sup> Natural support – an Object being measured is supported by the skeletal system.

<sup>8</sup> Artificial support – an Object being measured is supported by an artificial supporting means. For more detailed information, don't hesitate to get in touch with the manufacturer.

<sup>9</sup> For establishing the location and contours of the Muscle, use simultaneously the manual palpation and resistance to the limb movement to cause medium contraction of the Muscle in interest (e.g. ref. Kendall, et al. 2005 "Muscles Testing and Functions with Posture and Pain").

<sup>10</sup> In the case of experimental measurements, the standard Probe can be replaced with custom made probes or with a bigger diameter or/and length. For more detailed information please contact the manufacturer.

2. nothing but the Probe is in physical contact with the skin surface located directly above the Object being measured;
3. the Probe is always perpendicular to the skin surface;
4. the Device is held as stable as possible within the range of the measurement position indicated by the:
  - green light,
  - probe marker,
  - **“Hold position!”** instruction on the Device screen;
5. the measurements are performed at the same measuring point;
6. during the measurement impulse, the Probe does not slide on the skin surface;
7. the Device holding hand, is well supported to ensure stability of the Device and comfortable measuring positions throughout the session;
8. the Device is held gently without gripping it tightly<sup>11</sup>;
9. the focus on the Probe while conducting the measurements.



---

**Re-measure, if any of the above listed aspects where not maintained.**

---

## 2.2.5 Evaluation of the measurement variability

The Device offers the Coefficient of Variation for all five Parameters (hereinafter – “CV”). The CV describes the relative dispersion (variability) of the Measurement series.

The CV of the measurement depends on the Subject, Object, Operator, Device and the measurement environment.

The lower the CV, the higher the stability of the Measurement.

---

**On the Device screen evaluate the results CV. If the CV is above 3% tap “Re-measure”, otherwise tap “Next” or “Save”.**

**The CV above 3% is show in red.**

**If the CV remains above 3%, try to determine the source of higher variability and make corrective actions.**

---



However, higher CV > 3% may also be caused by the Subject’s neurological condition and tremor. In this case, the variability reflects the current health condition and should be accepted.

---

<sup>11</sup> A tight grip of the Device may cause the hand tremor and decrease the accuracy of the results.

## 2.3 Measurement interference detection system

The Device contains an automatic interference detection<sup>12</sup> system. Measurement raw signals are verified for:


1. required datapoints that are mandatory for parameters computing;
2. optimal impulse time (Tap time);
3. probe jump-off (loss of contact with the skin surface).
4. non-inherent frequencies;

On detection of the above-listed interferences, the following notices are displayed on the device screen<sup>13</sup>:

1. *Measurement data invalid!*
2. *Optimizing impulse!*
3. *Probe jump-off detected!*
4. *Excessive noise detected!*

### 2.3.1 Evaluation of a Tissue oscillation signal

For computing all five Parameters, the oscillation acceleration signal must contain at least two positive peaks  $a_1$  and  $a_3$  (Appendix 2).

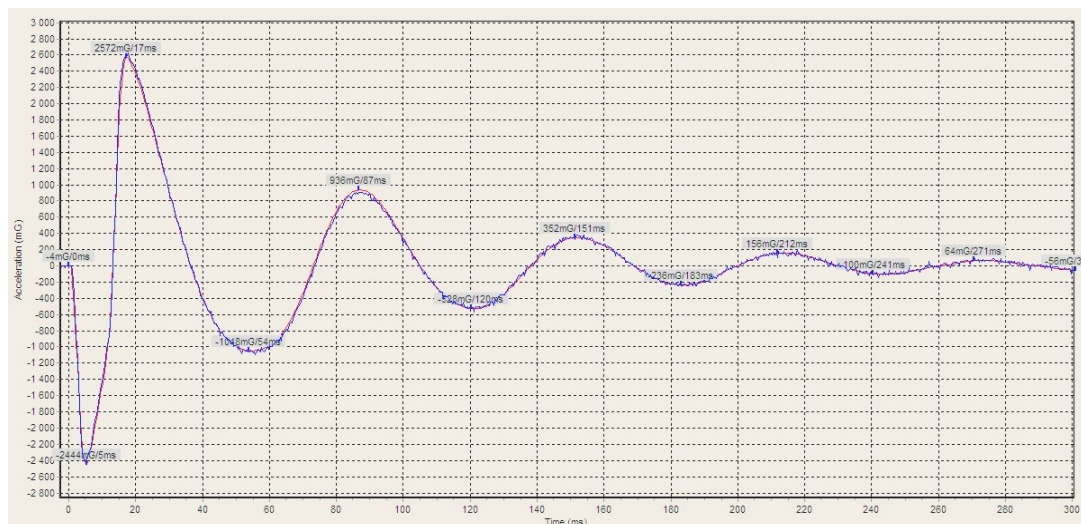

**Figure 1:** A typical soft tissue oscillation acceleration signal. The blue line - raw acceleration signal.

The acceleration signal with at least three positive peaks is typical for a tissue natural oscillation.

<sup>13</sup> See also chapter 3.5.1 p.7.

## 2.3.2 Optimal measurement impulse time


The measurement's standard impulse time (Tap time) is 15 milliseconds in duration.

The Device contains Optimal impulse time identification system. In case of non-optimal impulse Device provides "Optimizing impulse"<sup>14</sup> notice while optimising the impulse automatically.

## 2.3.3 Probe jump-off

In the event of the Probe jump-off, the Probe disconnects from the skin surface. The short-term disconnection may occur when a tissue oscillates more forcefully than the Device measurement system can physically follow.

The phenomenon may occur while measuring contracted tendons or muscles (see Figure 2).

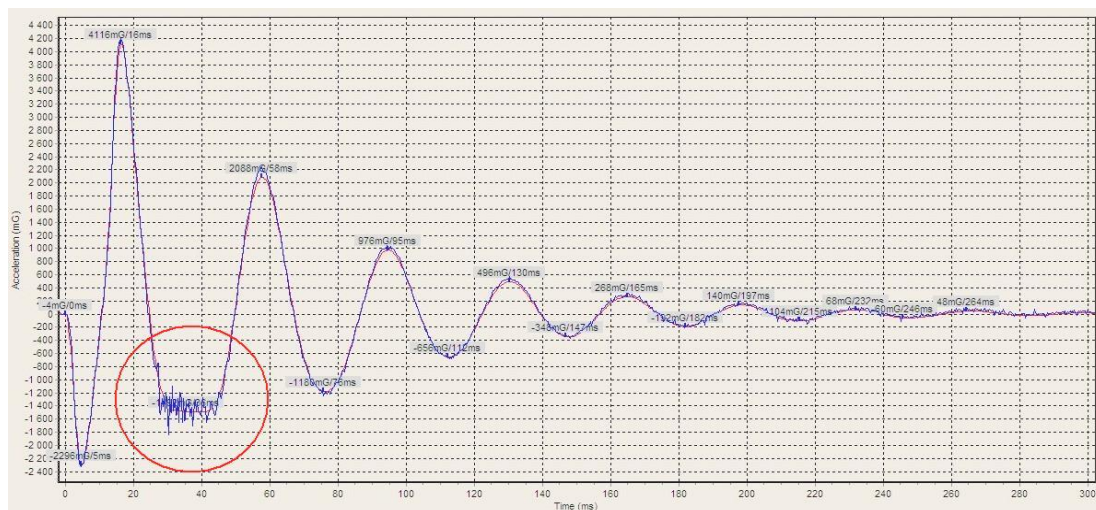

**Figure 2:** A typical Probe jump-off.



**Due to the computation algorithms, parameters F, S, R and C are not affected by the Probe jump-off phenomenon.**

**To avoid Probe jump-off decrease the impulse time duration by adjusting the Tap time, e.g. from 15ms to 10ms.**


<sup>14</sup> "Optimizing impulse" system is available from November 2022, MyotonPRO device, Model 1B, device firmware v. 1.6.2.

## 2.3.4 Non-inherent frequencies

The acceleration signal may contain non-inherent frequencies, which may influence the results and increase the measurement variation.

**Non-inherent frequencies or Excessive noise may occur e.g.:**

- a) during the oscillation the Probe is in contact with the Device probe opening (see Figure 3),
- b) A damaged<sup>15</sup> Device, measurement system or loose components.

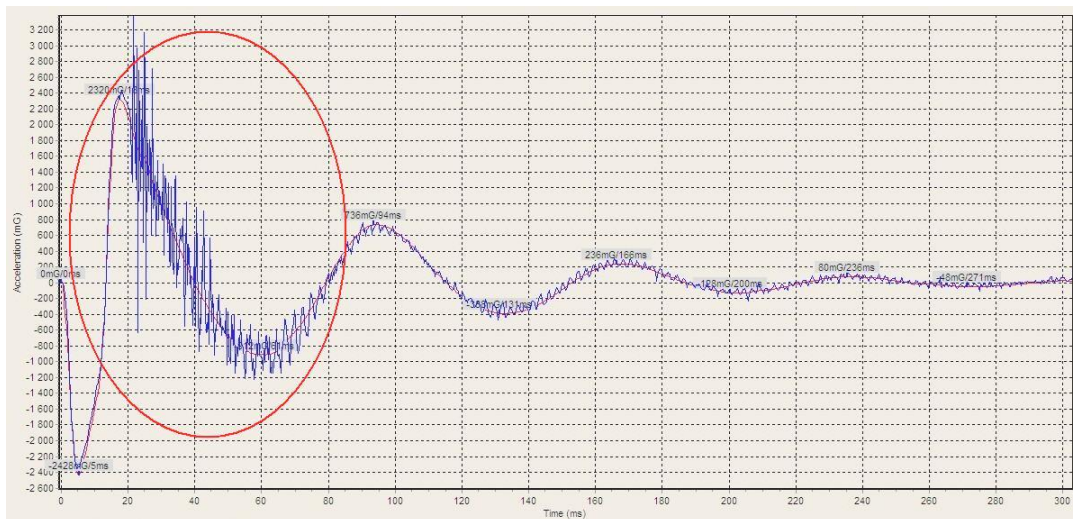

**Figure 3:** A raw acceleration signal with excessive noise. The figure illustrates a measurement signal while the Probe was in contact with the probe opening of the Device.

A signal of good quality is shown in the Figure 1.



**Possible damage to the Device measurement system can be identified using the test mode by generating a raw acceleration signal - blue line (see also chapter 3.3.2).**

**In the event of damage to the Device, don't hesitate to contact the manufacturer.**

<sup>15</sup> The Device may be damaged as a consequence of it having been dropped.

## 3. Application of MyotonPRO

### 3.1 Quick instruction

The measurement process consists of the following steps:

1. Use the Myoton Desktop Software to prepare and upload the Subjects and Patterns.
2. Mark the measurement point with the skin marker on the skin surface directly above the Object (e.g. muscle or tendon) being measured.
3. Press the ON/NEXT button or tap the "Measurements" label on the screen.
4. Press the ON/NEXT button or select the Subject and the Pattern on the screen.
5. Press the ON/NEXT button or tap the "Perform new measurement" label, or view previous measurements.
6. Place the Probe perpendicular to the pre-marked measurement point above the Object being measured.
7. Move the Device towards the skin until the green light indicates the measurement position.
8. Hold the Device gently and keep it steady within the measurement position until the Device automatically performs the Measurement.
9. Press the ON/NEXT button to continue with measurements until all Subjects and Patterns have been completed.
10. Download the results to the Myoton Desktop Software for further analysis.



## 3.2 Preparations for use

This chapter describes the procedure for the application of the Device.



---

**Use of the Device requires the Myoton Desktop Software for Windows.**

**The Software is not freeware. Download and use of the Software are only permitted for MyotonPRO device users.**

---

**The Software updates and the User Manual are available under the toolbar section “User Manual & About”.**

**The main functions of the Software:**

- Device settings set-up .....chapter 4.2.5
- Preparation of the Measurement set.....chapter 3.4.1 – 3.4.2
- Upload of the Measurement set .....chapter 3.4.3
- Conducting of test measurements.....chapter 3.3.2
- Download of the Measurement results from the Device.....chapter 3.6.1
- Compilation of basic reports.....chapter 3.6.2 – 3.6.7
- Export of the measurement results for external analysis.....chapter 3.7

### 3.2.1 Charging the battery

The Device is provided with an internal Lithium Polymer battery, which at the time of delivery has been charged to approx. 40%.

Charge the battery before use of the Device. The charging time for an empty battery with the current of 1A is up to 3 hours.



---




**Charge the battery before performing the measurements to at least 20% of capacity. The maximum voltage of fully a charged battery of 100% is 4.2V. The battery level is provided on the Device screen.**

---

**The battery can be charged either:**

- a) with the USB cable from a computer,  
NB! If the Device is charged through a general computer, it is required to make sure that it is compatible with the standard IEC 60950 and it is done outside the patient area.  
or
- b) from the mains power supply with the AC-DC 5V/1A USB adapter delivered by the manufacturer (see technical specification).

NB! When charging the battery from the mains power supply, the charger should be positioned for easy disconnection.

Description	Photo
<p><b>1.</b> Charging the battery from the computer, connect the Device to the computer with the USB cable.</p> <p>Ensure the computer is switched ON for the duration of charging.</p>	
<p><b>2.</b> Charging the battery from the adapter, connect the Device to the adapter with the USB.</p>	
<p><b>3.</b> The battery is fully charged when the LED below the ON/NEXT button turns GREEN.</p>	



During the process of charging or use, the Device may become warm. To check the temperature of the Device, press *Menu* and *System info*. If the temperature of the Device is over 50 °C (122°F), turn off the Device and wait until it cools down to avoid possible damage to the battery or the Device.

Charge the battery from the mains power supply **ONLY** with the adapter and USB cable delivered by the manufacturer. Remove the adapter from the mains power supply if not in use. The charging LED turns RED in case of required maximum voltage level of 4.2V has not been reached within 5 hours of charging.

In this case, repeat the charging process one more time. In case the LED remains RED after the second charging process, this indicates the battery should be replaced.

**Never fully discharge the battery. For prolonged storage, charge the battery in every 6 months.**

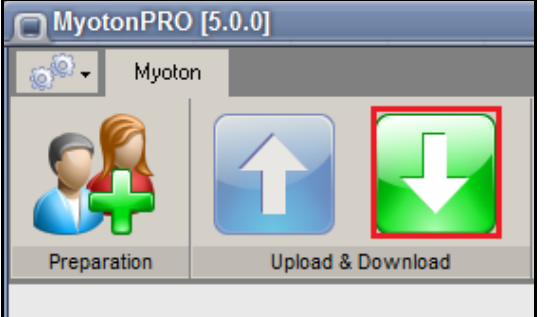

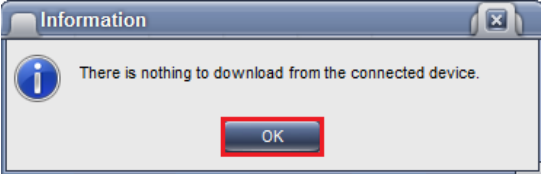
If the Device does not start, charge the battery for 15 minutes and try again. If the problem persists, please contact support.

The battery can be replaced **ONLY** by the manufacturer!

### 3.2.2 Connecting the Device to the computer

To establish the connection, connect the Device to the computer with the USB cable and open the Software.

For checking the connection:

Description	Screen image
<p>1. Switch on the Device, open the Software. The Device serial number should automatically appear on the Software status bar or click the <i>Download</i> button on the main toolbar.</p>	
<p>2. The connection is successfully established when the serial number appears on the Software system bar or the <i>Download</i> window opens and displays the serial number of the connected Device.</p> <p>While the connection is being established, a two arrow symbol appears for a short period in the top left corner of the Device screen.</p> <p>If you do not need to download previously taken measurements, close the window by clicking <i>Close</i>, otherwise click <i>Download</i>.</p>	
<p>3. In case the connection has been successfully established and there is nothing to download, click OK.</p>	

4. If the connection was not established, the Device connection window will open after clicking on the Download button. Wait for 5–10 seconds or unplug and plug the USB wire again when using the USB connection.



---

**If there is no connection with the computer, the following actions may help.**

- UNPLUG and PLUG the USB cable between the Device and computer. If necessary, try different USB sockets of the Computer if available.
  - Check that the USB cable is properly connected to the Device and computer.
  - Check that the Device is turned on and showing the main screen.
  - Close and open the Software and repeat the actions above to establish the connection.
  - Check the USB cable is not broken.
-

## 3.3 Test mode

The Device and Software allow to perform test measurements in the Test mode provided with the image of the acceleration signal of the Tissue's natural oscillation.

**Test mode is used for the following:**

- to study the measurement procedure,
- to evaluate the Co-oscillation (chapter 2.3),
- to test that the Device is functioning properly.

**There are two options to start the Test mode:**

- on the Device (chapter 3.3.1),
- in the Software (chapter 3.3.2).


**The benefits of using the Test mode in the Software are:**



- display of the test measurement protocol on the computer screen, printing and saving the test result in JPG format;
- display of the raw and filtered acceleration signal of the natural oscillation of an Object (blue and red line);
- the possibility to examine the acceleration signal in detail (zoom-in and out);
- the possibility to export the values of the raw and filtered signals;
- display of other measurement related parameters.

### 3.3.1 Activation of Test mode in the Device


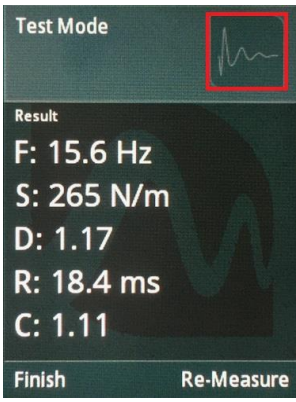
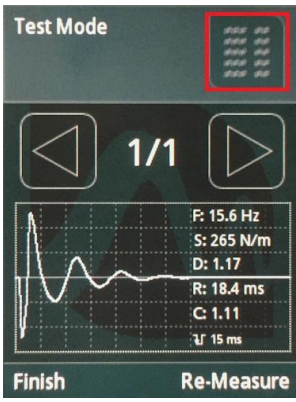
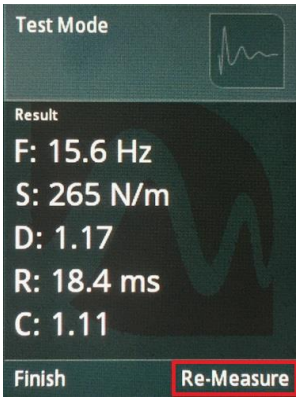




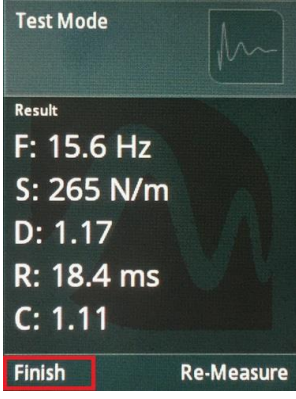
If the Test mode is started on the Device, detailed information is not displayed on the computer screen. The Test mode in the Device can be used only for general assessments.

Description	Screen image
<ol style="list-style-type: none"> <li>1. Click <i>Measurements</i> on the main screen of the Device.</li> </ol>	

<p>2. Click <i>Test</i> on the bottom left of the <i>Select subject</i> screen.</p>	
<p>3. The <i>Test mode</i> screen opens, where the instructions to the Operator are displayed: Move into position!</p> <p>Place the Probe perpendicular to the skin surface above the pre-marked measurement point on the Object. Move the Device towards the skin surface until the Measurement position<sup>16</sup> is indicated by the green light.</p>	

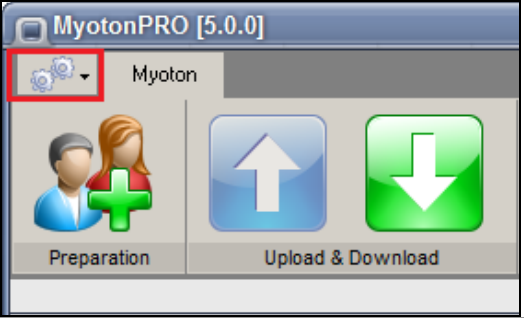
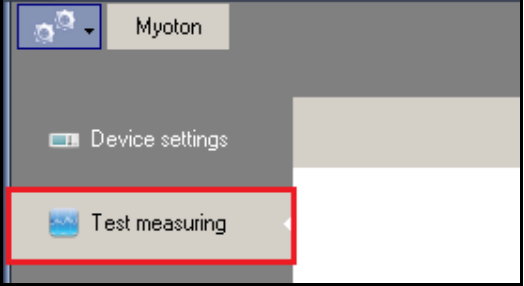
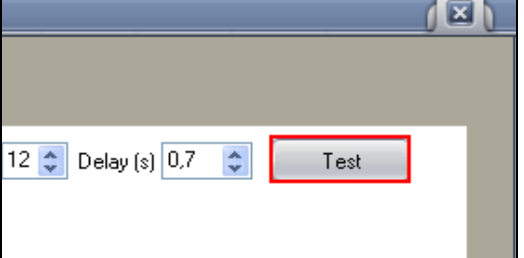
<sup>16</sup> The information regarding measurement position is provided in chapter 2.2.4, point 4.

<p><b>4.</b> Keep the Device steady within the measurement position. The Device performs the measurement automatically.</p> <p>Follow the instructions displayed on the screen:</p> <ul style="list-style-type: none"> <li>• <i>Move into position!</i> – move the Device towards the Object;</li> <li>• <i>Hold position!</i> – hold the Device within the range of 3mm of the measuring position;</li> <li>• <i>Move back!</i> – move the Device away from the skin surface;</li> </ul> <p><i>Rotate!</i> – Roll the Device around its longitudinal axis (chapter 2.2.1 p3).</p>	
<p><b>5.</b> After the measurement, the results are displayed.</p> <p>If necessary, the Tissue oscillation acceleration curve can be checked by clicking the Signal icon on the top right of the screen.</p> <p>Observation of the acceleration signal allows for the evaluation of the natural oscillation of the tissue (chapter 2.3).</p>	
<p><b>6.</b> To navigate back, click the <i>Results</i> icon on the top right of the screen.</p>	
<p><b>7.</b> To repeat the measurement, click the Re-Measure icon.</p>	

<p><b>8.</b> If necessary the Tap time can be changed by clicking on the Tap time icon.</p>	 <p>Test Mode</p> <p>Move into position!</p> <p>Single scan, 1/1 <span style="border: 1px solid red; padding: 2px;">12 ms</span></p> <p>Cancel</p>
<p><b>9.</b> Click the "up/down" arrows on the Set Tap Time window and confirm by clicking OK.</p>	 <p>Test Mode</p> <p>Set Tap Time</p> <p><span style="border: 1px solid red; padding: 2px;">▽</span> 12 ms <span style="border: 1px solid red; padding: 2px;">△</span></p> <p><span style="border: 1px solid red; padding: 2px;">OK</span> Cancel</p>
<p><b>10.</b> To exit the Test mode, click Finish.</p>	 <p>Test Mode</p> <p>Result</p> <p>F: 15.6 Hz S: 265 N/m D: 1.17 R: 18.4 ms C: 1.11</p> <p><span style="border: 1px solid red; padding: 2px;">Finish</span> Re-Measure</p>



### 3.3.2 Activation of the Test mode in Software

Description	Screen image
<p>1. Open the Software, connect the Device to the computer and ensure the Device is showing the main screen. In the Software click the <i>Settings</i> icon on the main toolbar.</p>	
<p>2. In the <i>Settings</i> menu, click the <i>Test measuring</i> button.</p> <p>Menu instructions of the <i>Test measuring</i> window are provided in chapter 4.2.6.</p>	
<p>3. In the <i>Test measuring</i> window, click the <i>Test</i> icon and perform a test measurement with the Device<sup>17</sup>. An acceleration signal is displayed on the computer screen. This can be evaluated based on chapter 2.3.</p>	

<sup>17</sup> Chapter 3.3.1, points 3 to 5.

## 3.4 Preparation of the Measurement Set

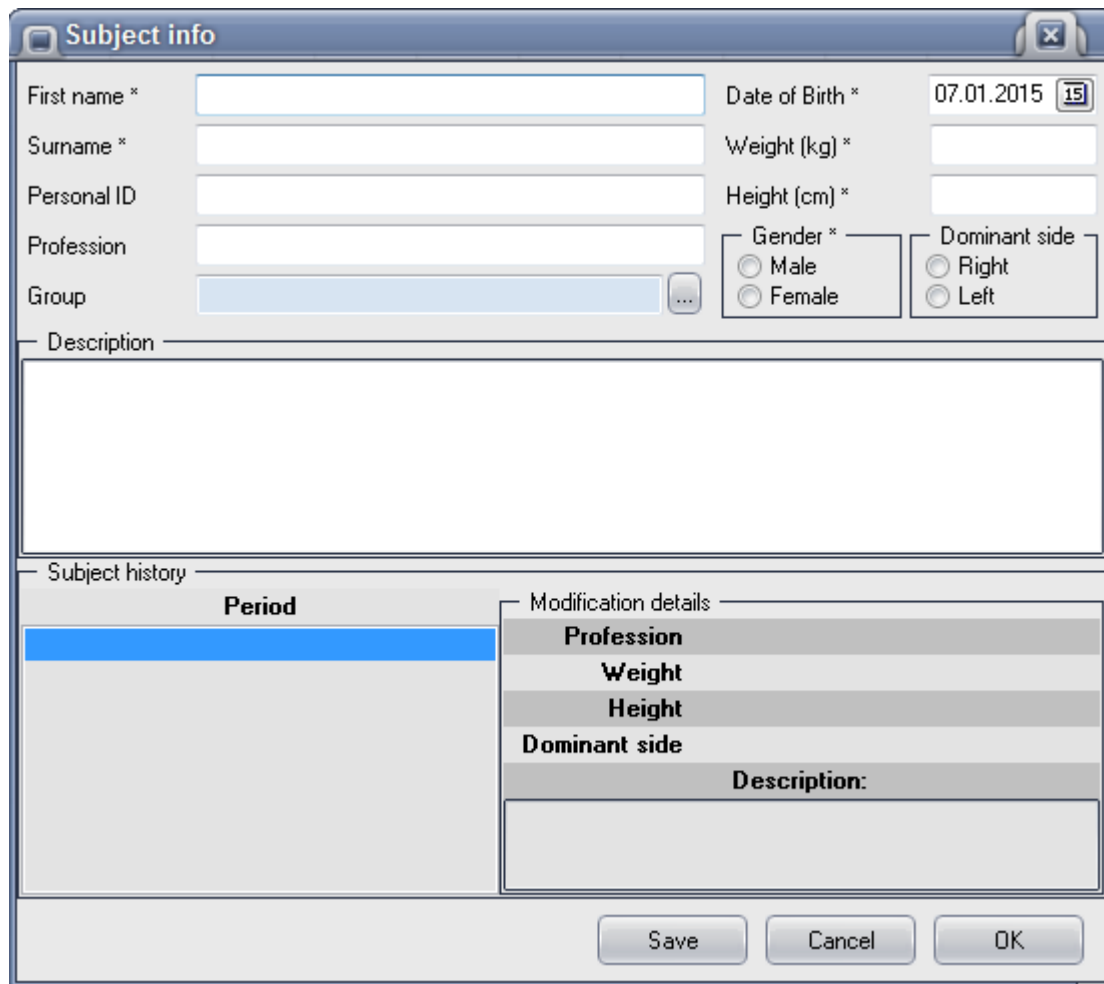
This chapter describes the preparation of the Measurement set for uploading to the Device. The Measurement set contains at least one Subject and one Pattern. It is possible to upload a maximum of 50 Subjects and 50 Patterns to the Device.

**There are two inputs needed to compose a measurement set:**

1. Subject information (chapter 3.4.1)
2. Pattern information (chapter 3.4.2)

### 3.4.1 Input of Subject's data

The Subject's data should be entered in the *Subject info* window:



**Subject info**

First name \*  Date of Birth \* 07.01.2015

Surname \*  Weight (kg) \*

Personal ID  Height (cm) \*

Profession

Group

Gender \*  Male  Female

Dominant side  Right  Left

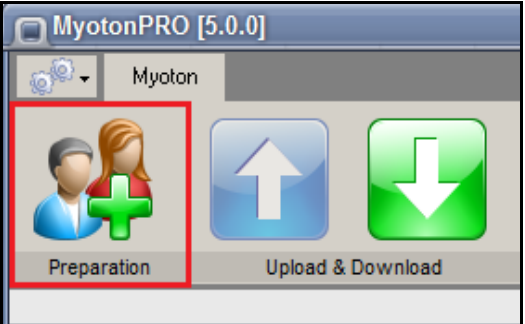
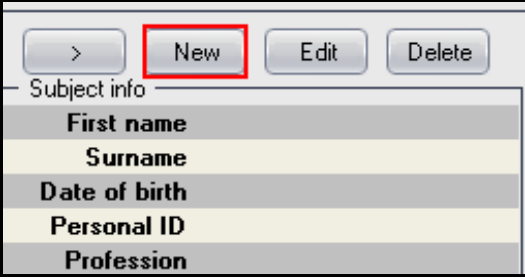
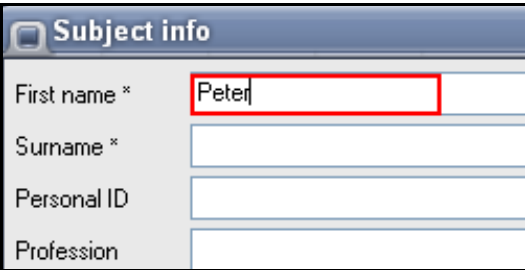
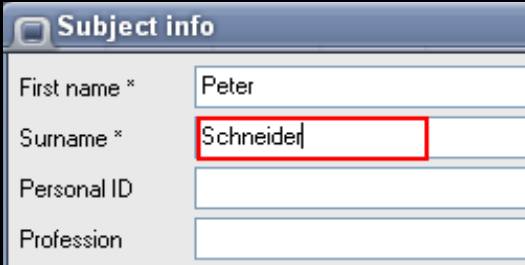
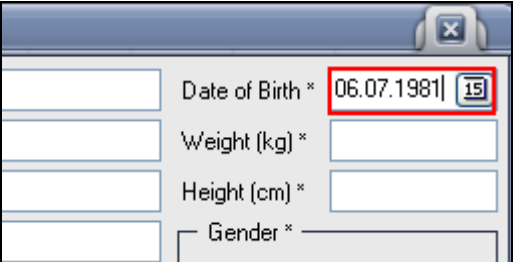
Description

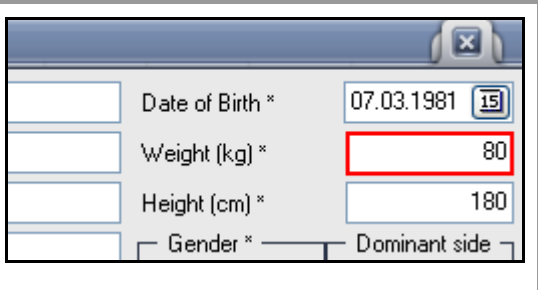
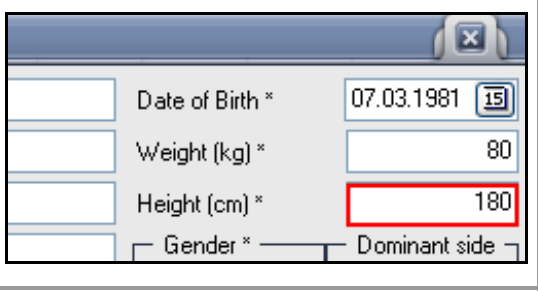
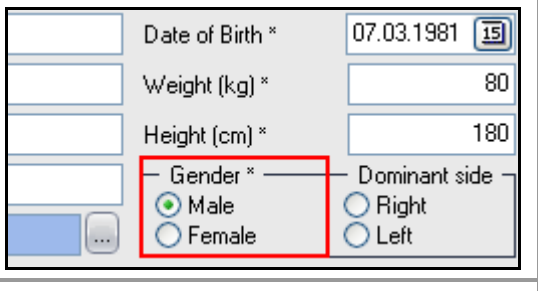
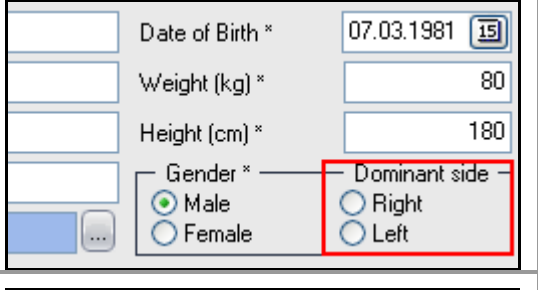
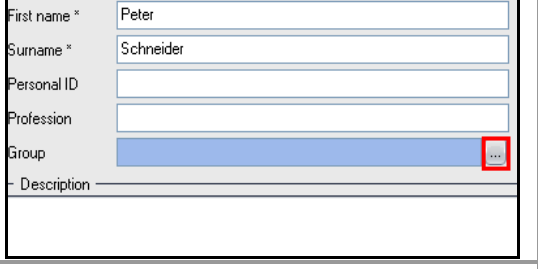
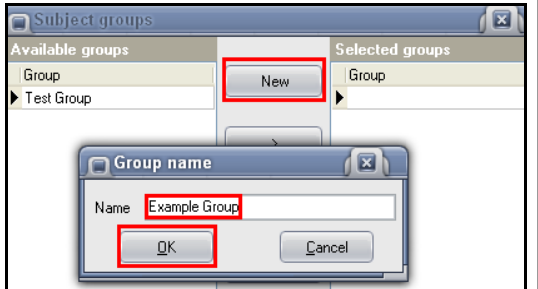
Subject history

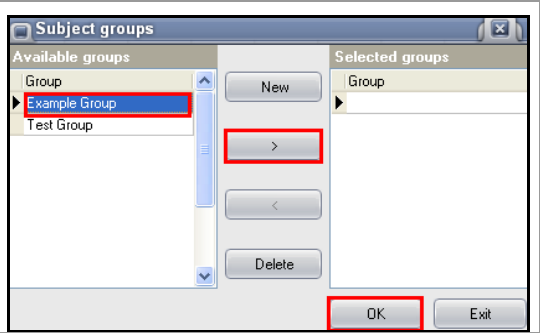
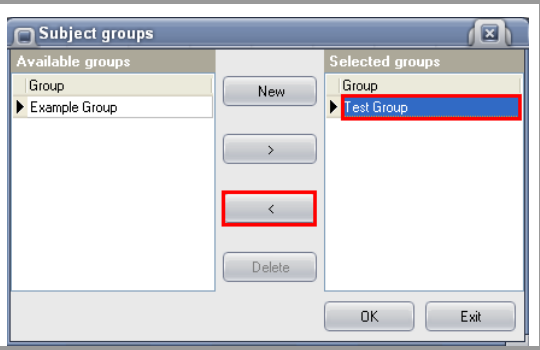
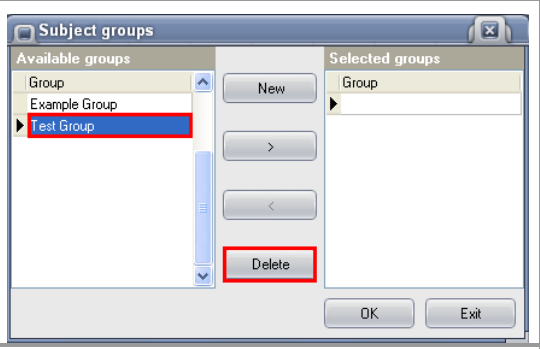
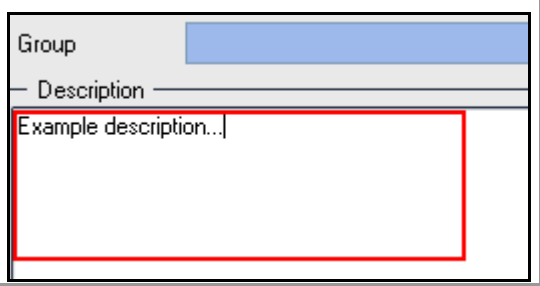
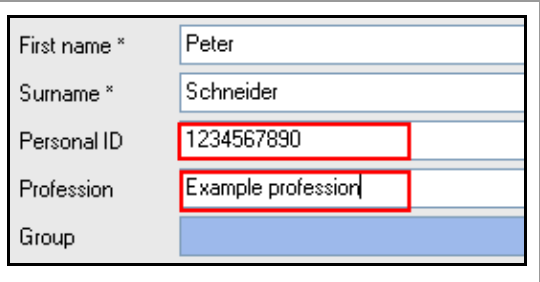
Period	Modification details
	Profession
	Weight
	Height
	Dominant side
	Description:

Save Cancel OK

**For entering the Subject's info:**

Description	Screen image
<p>1. Open the Software and click the <i>Upload Preparation</i> icon on the main toolbar.</p>	
<p>2. In the <i>Upload Preparation</i> window under the section <i>Subjects</i>, click the <i>New</i> Icon.</p>	
<p>3. In the <i>Subject info</i> window, write the Subject's first name in the field of the <i>First name</i>.</p> <p>NB! The Subject's data can be limited by the first name only. See "<b>Mandatory subject info</b>" checkbox in Preferences window - chapter 4.2.8</p>	
<p>4. Write the Subject's surname in the field of <i>Surname</i>.</p>	
<p>5. Write the Subject's date of birth (dd.mm.yyyy) in the field of <i>Date of Birth</i>.</p>	

<p>6. Write the Subject's weight in kg in the field of <i>Weight</i>.</p> <p><b>Conversion:</b>  1kg = 2.2pounds  1pound = 0.45kg</p>	
<p>7. Into the field <i>Height</i>, write the Subject's height in cm.</p> <p><b>Conversion:</b>  1cm = 0.0328feet  1foot = 30.48cm</p>	
<p>8. Assign the Subject's gender in the section <i>Gender</i>.</p>	
<p>9. If necessary it is possible to mark the Subject's <i>Dominant side</i>.</p>	
<p>10. If necessary to group the Subjects (e.g. different conditions, locations, genders), it is possible to assign a group for the Subject. Start assigning the group from the button shown on the image.</p>	
<p>11. To create a new group, click the <i>New</i> icon, enter the Group name and click <i>OK</i>.</p>	

<p><b>12.</b> To assign the Subject to the group, select or double-click on the name of the group and click the arrow button. To confirm the selection(s), click <i>OK</i>.</p>	
<p><b>13.</b> To discontinue a group, make a selection in the column of <i>Selected groups</i> and click the left arrow button.</p>	
<p><b>14.</b> To delete a group, select the group in the column of <i>Available groups</i> and click <i>Delete</i>.</p>	
<p><b>15.</b> A Subject description can be added to the <i>Description section</i>, if necessary.</p> <p>On entry of a new description, the previous one can be seen in the <i>History section</i>.</p>	
<p><b>16.</b> The Subject ID and Profession can be entered in subsequent fields.</p>	

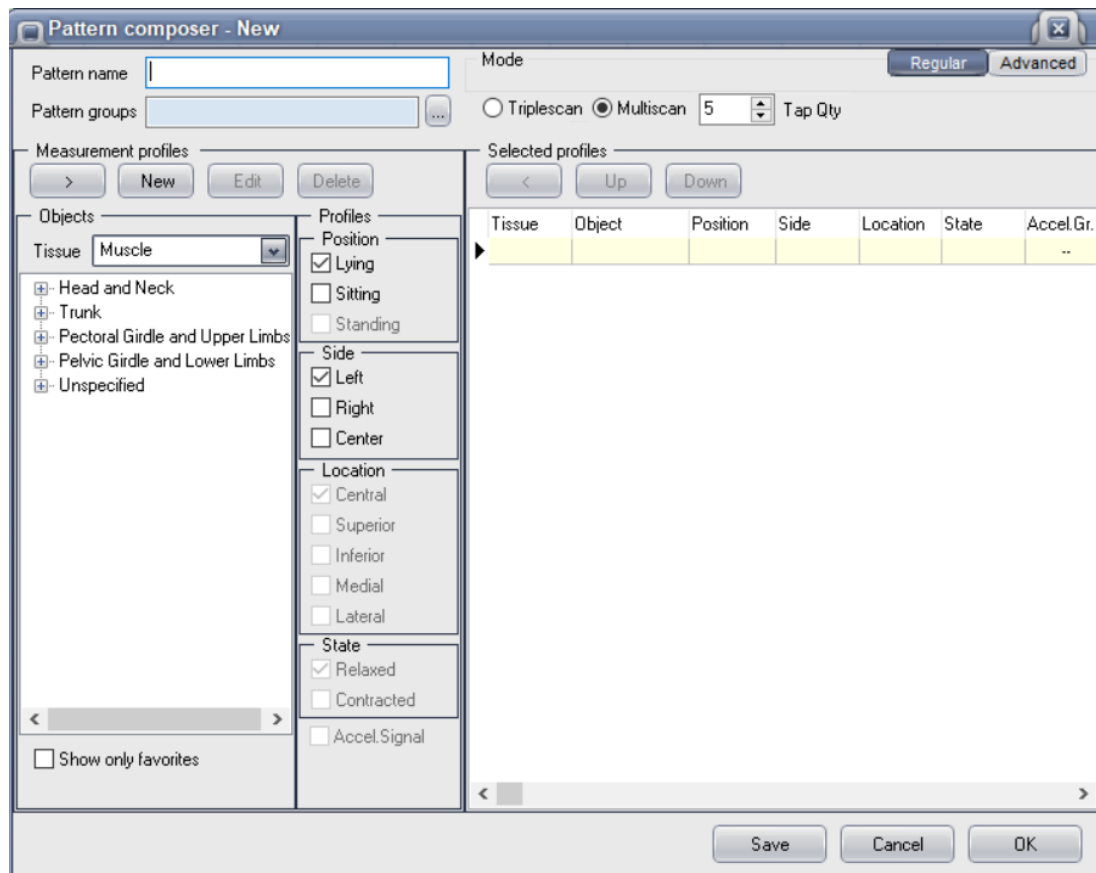
**17.** Click *Save* and *OK*, to finish entering the Subject information.

<b>Profession</b>
<b>Weight</b>
<b>Height</b>
<b>Dominant side</b>
<b>Description:</b>
<div style="display: flex; justify-content: space-around; margin-top: 10px;"> <div style="border: 2px solid red; padding: 2px 10px;">Save</div> <div style="border: 1px solid gray; padding: 2px 10px;">Cancel</div> <div style="border: 2px solid red; padding: 2px 10px;">OK</div> </div>

### 3.4.2 Composing Patterns

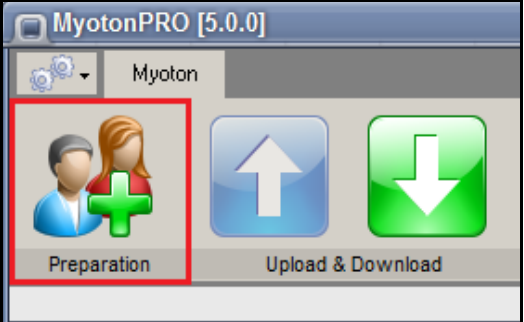
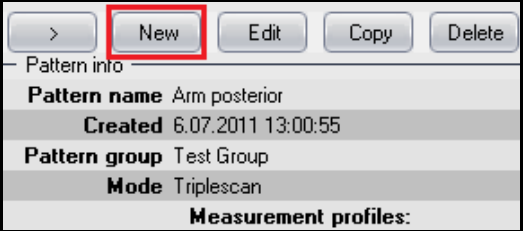
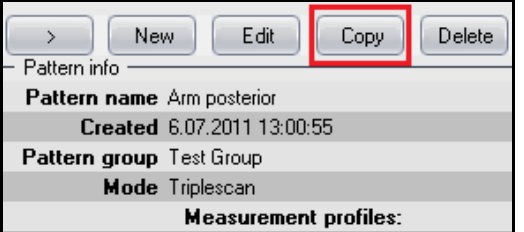

Patterns contain Objects and the profile of how they are to be measured.

Patterns are composed in the *Pattern Composer* window:



The screenshot shows the 'Pattern composer - New' window. At the top, there are fields for 'Pattern name' and 'Pattern groups'. The 'Mode' section has 'Regular' and 'Advanced' buttons, with 'Multiscan' selected and 'Tap Qty' set to 5. Below this are 'Measurement profiles' and 'Selected profiles' sections. The 'Measurement profiles' section includes a tree view of 'Objects' (Muscle, Head and Neck, Trunk, Pectoral Girdle and Upper Limbs, Pelvic Girdle and Lower Limbs, Unspecified) and a list of 'Profiles' with checkboxes for Position (Lying, Sitting, Standing), Side (Left, Right, Center), Location (Central, Superior, Inferior, Medial, Lateral), and State (Relaxed, Contracted, Accel.Signal). The 'Selected profiles' section shows a table with columns: Tissue, Object, Position, Side, Location, State, and Accel.Gr. The table is currently empty. At the bottom, there are 'Save', 'Cancel', and 'OK' buttons.

**For composing a Pattern:**

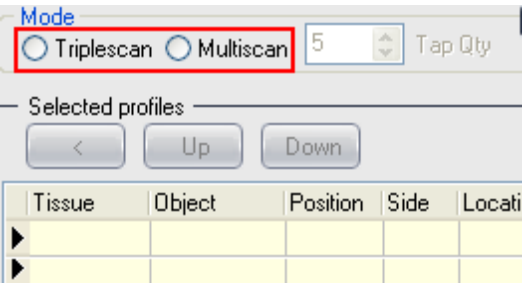
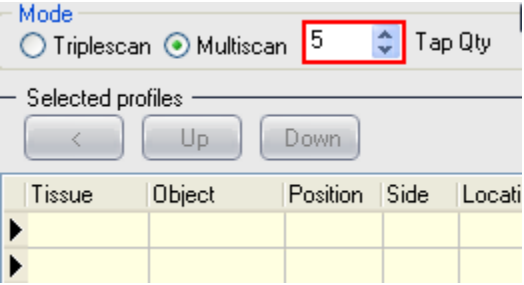
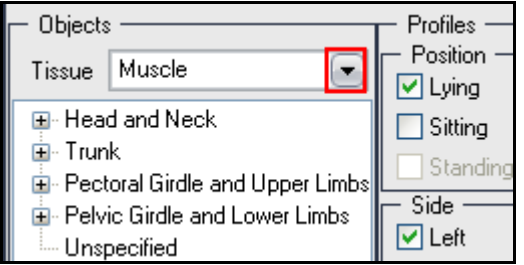
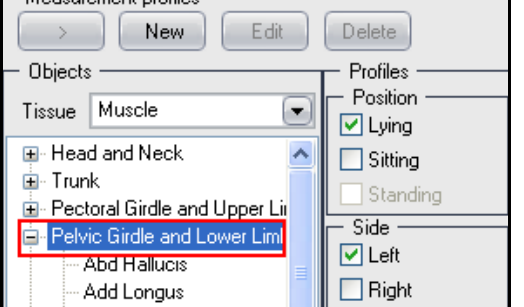
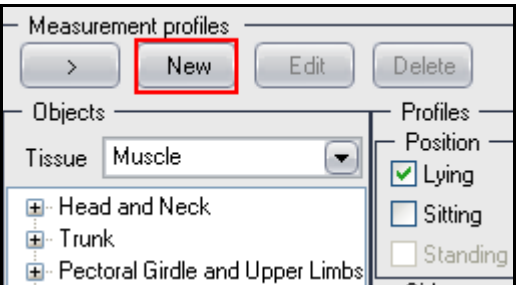
Description	Screen image
<p>1. Click the <i>Upload Preparation</i> icon in the main toolbar.</p>	
<p>2. In the Upload Preparation window, click <i>New</i> in the <i>Patterns</i> section to open the <i>Pattern composer</i> window.</p>	
<p>3. To copy the content of the existing Pattern into the new Pattern, activate a Pattern name in the list and click <i>Copy</i> in the <i>Upload Preparation</i> window. A pre-filled <i>Pattern Composer</i> window will open.</p>	
<p>4. Enter the new pattern name in the space titled <i>Pattern name</i>.</p>	



Create Pattern names containing some info about the study design or measurement program (e.g. PRE interv.....; POST interv.....; ..... t0; ..... t1; etc).

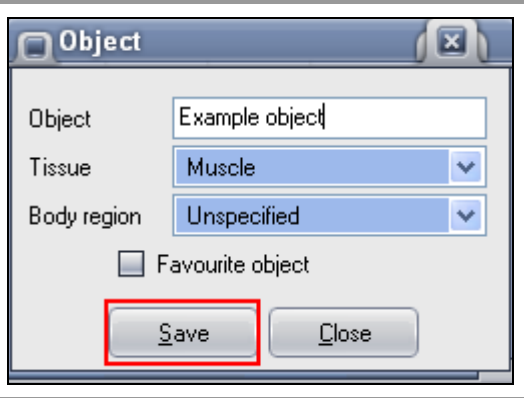
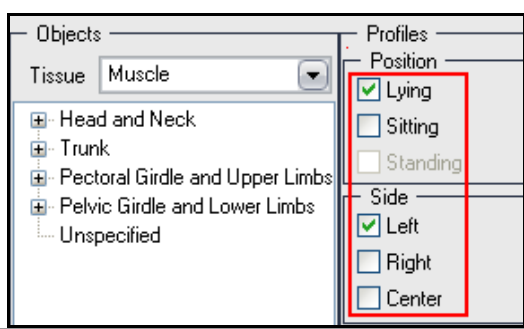
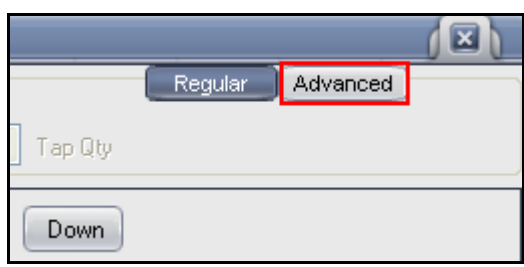
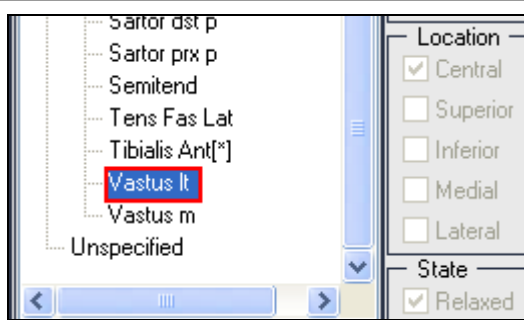
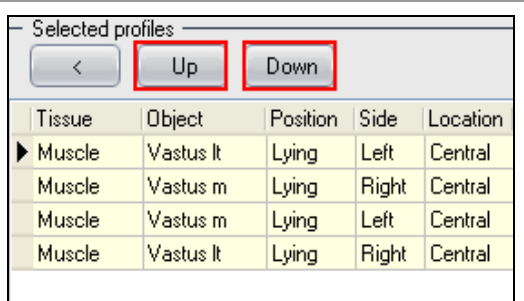
Describing Pattern names is an important source of information for identifying the measurement results later in statistical analysis.

Up to 32 characters for Pattern and Object names are displayed on the Device screen.

<p>5. Select the measurement mode<sup>18</sup> in the <i>Mode</i> section.</p> <p>NB! Default measurement mode is Multiscan.</p>	
<p>6. For the Multiscan measurement mode, select the number of single measurements in the series.</p> <p>NB! Default quantity of single measurements in the Multiscan measurement series is 5.</p>	
<p>7. Select the tissue type in interest: Muscle, Tendon, Ligament or Other.</p> <p>For every Tissue type, the body region is broken down into five classifications.</p>	
<p>8. In the <i>Objects</i> section, open the corresponding body region where the required Object is located.</p> <p>PS. The initial classification of body regions includes Muscles only.</p>	
<p>9. Click <i>New</i> to add missing Objects.</p>	

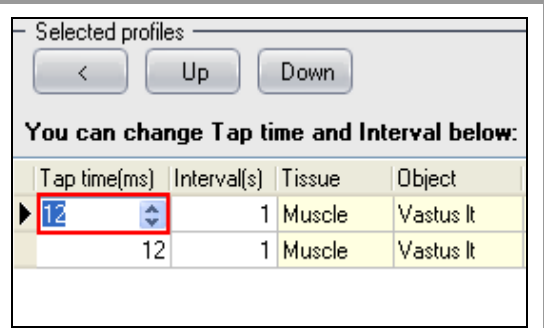
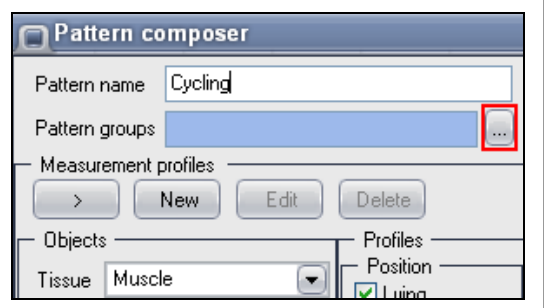
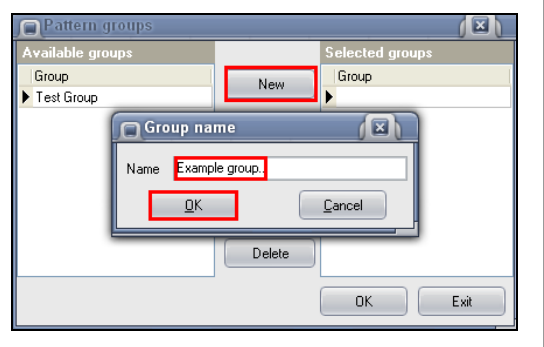
<sup>18</sup> Measurement modes (Triplescan and Multiscan) are explained in the chapter “Terminology”.



<p><b>10.</b> Enter a new Object name<sup>19</sup>, select the appropriate <i>Tissue</i> type and <i>Body region</i> and confirm by <i>Save</i>.</p>																										
<p><b>11.</b> Set up the Object's measurement Profile by checking relevant check-boxes in the <i>Profiles</i> section:</p> <p>(Body position, Body Side, Location of the measurement point and State of activation).</p>																										
<p><b>12.</b> To activate disabled options (<i>Location, State</i> and <i>Accel. Signal</i><sup>20</sup>), click the <i>Advanced</i> button on the upper right corner of the <i>Pattern composer</i> window.</p>																										
<p><b>13.</b> Select the Object(s) for the Pattern by double-clicking or highlighting and then clicking on the right arrow button above the Objects.</p> <p>A maximum of 50 Objects for one Pattern can be selected.</p>																										
<p><b>14.</b> Click <i>Up</i> or <i>Down</i> buttons to set the order in which the Objects are measured.</p> <p>NB! The aim is a minimal disturbance to the Subject.</p>	 <table border="1" data-bbox="980 1646 1500 1843"> <thead> <tr> <th>Tissue</th> <th>Object</th> <th>Position</th> <th>Side</th> <th>Location</th> </tr> </thead> <tbody> <tr> <td>Muscle</td> <td>Vastus It</td> <td>Lying</td> <td>Left</td> <td>Central</td> </tr> <tr> <td>Muscle</td> <td>Vastus m</td> <td>Lying</td> <td>Right</td> <td>Central</td> </tr> <tr> <td>Muscle</td> <td>Vastus m</td> <td>Lying</td> <td>Left</td> <td>Central</td> </tr> <tr> <td>Muscle</td> <td>Vastus It</td> <td>Lying</td> <td>Right</td> <td>Central</td> </tr> </tbody> </table>	Tissue	Object	Position	Side	Location	Muscle	Vastus It	Lying	Left	Central	Muscle	Vastus m	Lying	Right	Central	Muscle	Vastus m	Lying	Left	Central	Muscle	Vastus It	Lying	Right	Central
Tissue	Object	Position	Side	Location																						
Muscle	Vastus It	Lying	Left	Central																						
Muscle	Vastus m	Lying	Right	Central																						
Muscle	Vastus m	Lying	Left	Central																						
Muscle	Vastus It	Lying	Right	Central																						

<sup>19</sup> Up to 32 characters for Pattern and Object names are displayed on the Device screen.

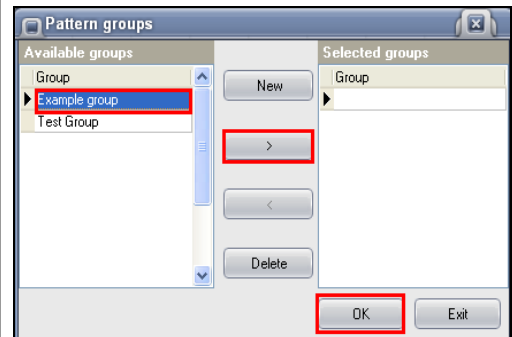
<sup>20</sup> The check-box *Accel. Signal* saves a raw acceleration signal that consists of 1280 rows of data for every measurement result. To avoid slowed data transfer and save the memory, activate the *Accel. Signal* only if necessary!

<p><b>15.</b> To change the values measurement characteristics (<i>Tap time</i> and <i>Interval</i>),<sup>21</sup> click the <i>Advanced</i> button first.</p> <p><b>NB! Do not change these values unless necessary!</b></p>	 <p>Selected profiles</p> <p>&lt; Up Down</p> <p><b>You can change Tap time and Interval below:</b></p> <table border="1"> <thead> <tr> <th>Tap time(ms)</th> <th>Interval(s)</th> <th>Tissue</th> <th>Object</th> </tr> </thead> <tbody> <tr> <td>12</td> <td>1</td> <td>Muscle</td> <td>Vastus It</td> </tr> <tr> <td>12</td> <td>1</td> <td>Muscle</td> <td>Vastus It</td> </tr> </tbody> </table>	Tap time(ms)	Interval(s)	Tissue	Object	12	1	Muscle	Vastus It	12	1	Muscle	Vastus It
Tap time(ms)	Interval(s)	Tissue	Object										
12	1	Muscle	Vastus It										
12	1	Muscle	Vastus It										
<p><b>16.</b> To assign Patterns a Pattern group<sup>22</sup>, click the button shown on the image.</p>	 <p>Pattern composer</p> <p>Pattern name: Cycling</p> <p>Pattern groups: [red box around ellipsis]</p> <p>Measurement profiles: &gt; New Edit Delete</p> <p>Objects: Tissue: Muscle</p> <p>Profiles: Position: <input checked="" type="checkbox"/> Lying</p>												
<p><b>17.</b> To create a new group, click <i>New</i>, enter the group name and click <i>OK</i>.</p>	 <p>Pattern groups</p> <p>Available groups: Group, Test Group</p> <p>Selected groups: Group</p> <p>New</p> <p>Group name dialog:</p> <p>Name: Example group</p> <p>OK Cancel</p>												

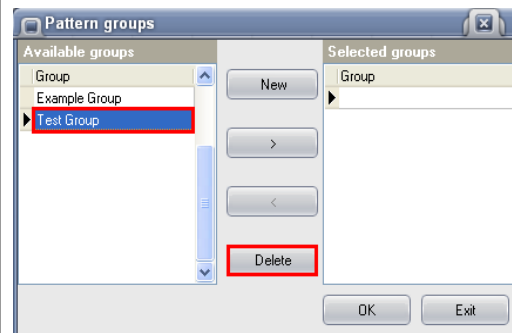
<sup>21</sup> “Tap time” and “Interval” are explained in the chapter “Terminology”.

<sup>22</sup> The patterns relevant to a particular study, health condition, field or activity can be selected for the group (e.g. Physiotherapy).

**18.** To assign the Patterns to the group, select or double-click on the name of the group and click the arrow button. To confirm the selection(s), click *OK*.



**19.** To delete a group, select the group in the column of *Available groups* and click *Delete*.



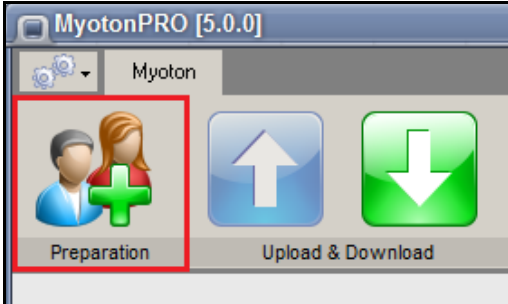
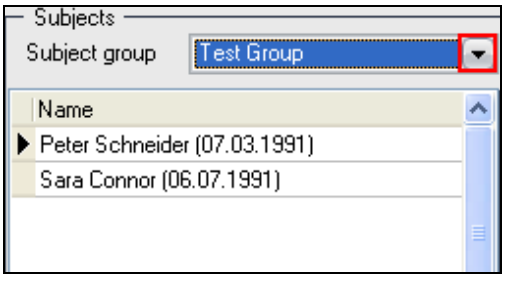
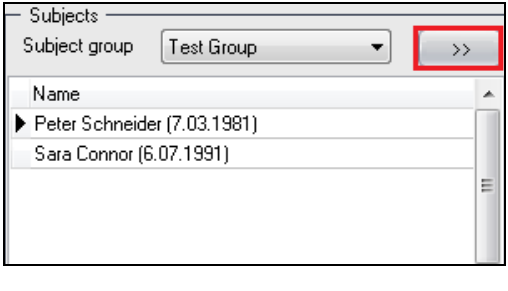
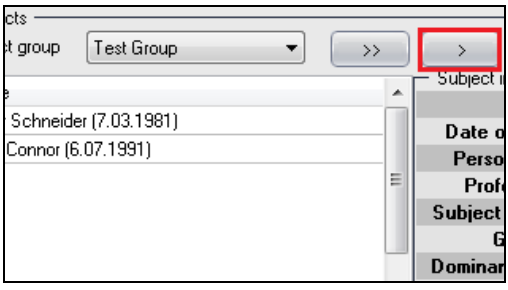
**20.** Click *Save* and *OK*, to finish composing the Pattern.



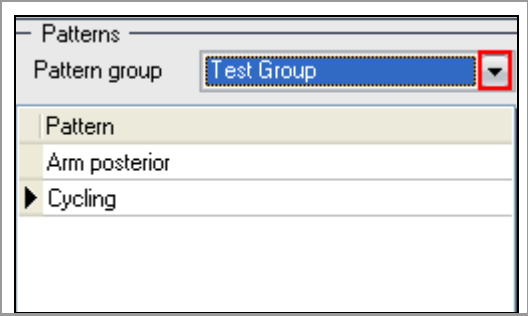
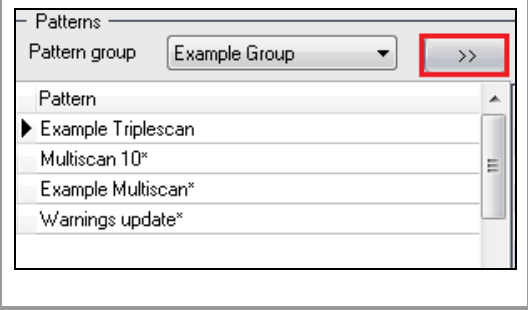
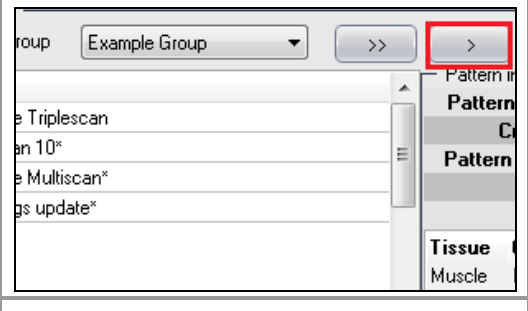

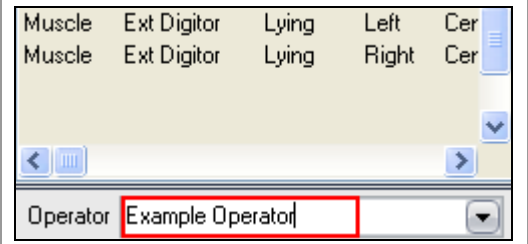
### 3.4.3 Uploading the Measurement Set

To perform measurements, the Measurement Set has to be prepared and uploaded to the Device. The Measurement Set consists of at least one Subject and one Pattern (see also *Upload Preparation* chapter 4.2.2).

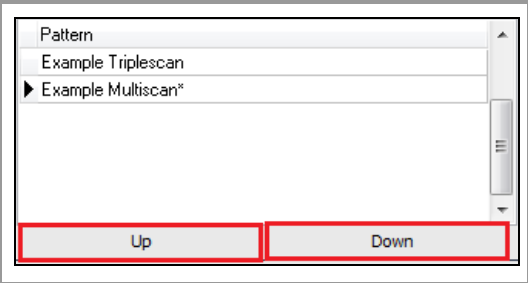
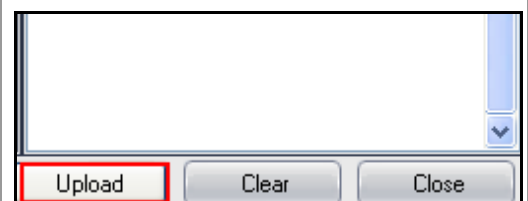
**To upload the Measurement Set:**

Description	Screen image
<p>1. Open the Software, establish the connection between the computer and the Device (chapter 3.2.2) and click the <i>Upload Preparation</i> icon in the main toolbar.</p>	
<p>2. If the measured Subject(s) are grouped<sup>23</sup> (chapter 3.4.1, point 10-14), it is possible to filter Subjects within the group by using the <i>Subject group</i>.</p>	
<p>3. Select the Subjects or the entire group for the Measurement Set by clicking on the arrow buttons above the Subject info.</p> <p>A maximum of 50 Subjects can be selected for the Measurement Set.</p>	
<p>4. Select the Subjects one by one for the Measurement Set by double-clicking the Subject name or selecting Subject name and clicking the arrow button above Subject info.</p>	

<sup>23</sup> Subjects can be added to or removed from the groups at a later stage.

<p>5. If the Patterns are grouped<sup>24</sup> (chapter 3.4.2, point 17-20), it is possible to filter Patterns within the group.</p>	
<p>6. Select the Patterns or entire group for the Measurement Set by clicking on the arrow buttons above Pattern info.</p> <p>A maximum of 50 Patterns can be selected for the Measurement Set.</p>	
<p>7. Select the Patterns one by one for the Measurement Set by double-clicking the Pattern name or selecting Pattern name and clicking the arrow button above the Pattern info.</p>	
<p>8. Enter or select the address where the measurements are performed in the box titled <i>Address</i> on the left bottom of the screen.</p>	
<p>9. Enter the person's name performing the measurements in the box titled <i>Operator</i> in the middle bottom of the screen.</p> <p>NB! The Operator name is mandatory.</p>	

<sup>24</sup> Patterns can be added to or removed from the groups at a later stage.

<p><b>10.</b> The measuring order<sup>25</sup> of the Subjects and Patterns can be changed by <i>Up</i> or <i>Down</i> buttons below the <i>Selected Subjects</i> or <i>Selected Patterns</i>.</p>	
<p><b>11.</b> Upload the Measurement Set to the Device by clicking <i>Upload</i> at the bottom right-hand side of the screen or the Upload icon in the main toolbar.</p>	



**Before uploading the Measurement Set, download the previous Measurement results from the Device first (chapter 3.6.1).**

**By the new upload, the previously uploaded Measurement Set will be deactivated.**

**Whenever necessary, the deactivated or active data can be downloaded to any database of the Software (chapter 3.8.1)**

**For upload or download, the Device must be on the main screen and connected to the computer (chapter 3.2.2).**

<sup>25</sup> It is important to consider the order in which Patterns are uploaded, to minimise disturbance to the Subject, thus reducing dispersion of the results.

If the sequence is correct, the Operator can only use the *ON/NEXT* button to seamlessly conduct all the measurements (chapter 3.5.1).


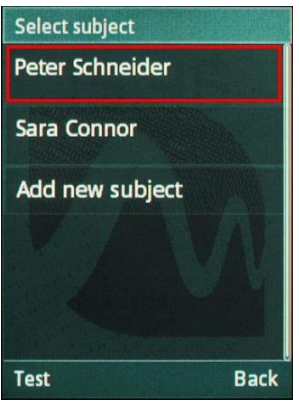
## 3.5 Device measurement mode


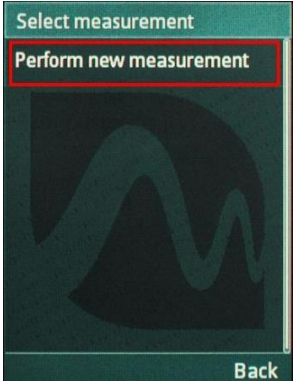

### 3.5.1 Measurement with the Device



Before performing the measurements, read carefully the chapter 2.2.



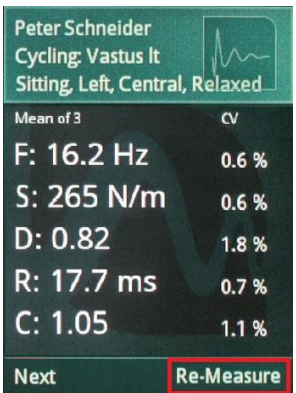
**COMMENT** The measurements can be performed by using only the *ON/NEXT* button. For this, make sure the Measurement profiles, Subjects, and Patterns have been uploaded in the correct order. Make sure that also the Objects inside the Patter are listed in the correct order!

Description of the action	Screen image
<ol style="list-style-type: none"> <li>1. Press the <i>ON/NEXT</i> button of the Device or tap the <i>Measurements</i> label on the main screen of the Device.</li> </ol>	
<ol style="list-style-type: none"> <li>2. Press the <i>ON/NEXT</i> button if measuring the first Subject in the list or tap any Subject in the list.</li> </ol>	

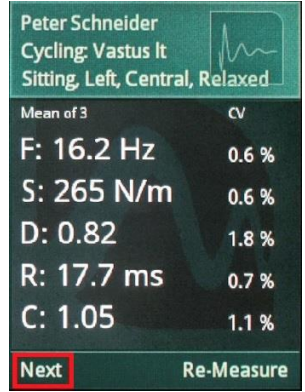
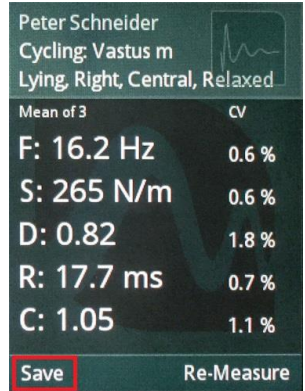


<p>3. Press the <i>ON/NEXT</i> button if using the first Pattern in the list or tap any Pattern in the list.</p>	
<p>4. Press the <i>ON/NEXT</i> button to Perform new measurements or tap on this label.</p>	
<p>5. The probe exerts, and the Measurement screen instructs: <i>Move into position!</i></p> <p>Place the Probe perpendicular to the skin surface on the pre-marked measurement point directly above the Object. Move the Device towards the skin surface until the Measurement position<sup>26</sup> is achieved and indicated by the green light.</p>	

<sup>26</sup> The information regarding measurement position is provided in chapter 2.2.4, point 4.



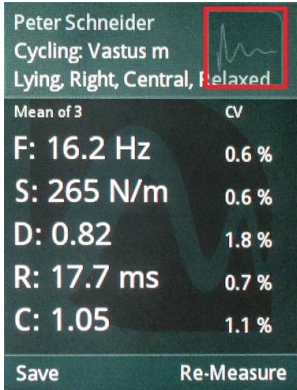
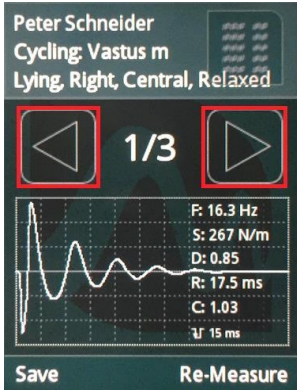
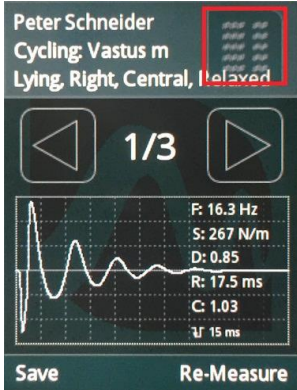
<p>6. Keep the Device steady in the range of the acceptable measurement position. There is a <i>Delay</i> of 0.7sec before the Device automatically starts the Measurement.</p> <p><b>Follow the instructions displayed on the screen:</b></p> <ul style="list-style-type: none"> <li>• <i>Move into position!</i> – move the Device towards the Object;</li> <li>• <i>Hold position!</i> – hold the Device within the range 3mm of the measuring position;</li> <li>• <i>Move back!</i> – move the Device away from the skin surface;</li> <li>• <i>Rotate!</i> – roll the Device around the Probe axis (chapter 2.2.1 p3).</li> </ul>	
<p>7. If the Device has detected an oscillation interference, a sound signal will be played, the probe light changes to red, and the Device screen displays an interference notification (chapter 2.3).</p> <p>If the interferences occur repeatedly, identified the cause and make corrective actions applied.</p>	
<p>8. After completing the Measurement, the Measurement results<sup>27</sup> are displayed with the variation coefficient (CV).</p> <p>Before pressing the <i>ON/NEXT</i> button to proceed, verify whether the CV of all the Parameters is less than 3% (chapter 2.2.5).</p> <p>If the CV is not satisfactory, tap <i>Re-Measure</i> to repeat the measurement.</p>	

<sup>27</sup> In case of Triplescan or Multiscan modes the Measurement result to be displayed is calculated by using different statistical methods (chapter “Terminology”).

<p><b>9.</b> To measure the next Measurement profile, press the <i>ON/NEXT</i> button or tap <i>Next</i>.</p> <p>NB! After pressing the <i>ON/NEXT</i> button or taping <i>Next</i>, the Measurements results will be saved and cannot be replaced. However, the Measurement can be repeated (see point 11).</p> <p>Unwanted measurement results can be deleted only in the Software after being downloaded to the Desktop Software.</p>	
<p><b>10.</b> To finish the Pattern, press the <i>ON/NEXT</i> button or tap <i>Save</i>.</p> <p>NB! After pressing the <i>ON/NEXT</i> button or taping <i>Save</i>, the Measurements results will be saved and cannot be replaced. However, the Measurement can be repeated (see point 11).</p> <p>Unwanted measurement results can be deleted only in the Software after being downloaded to the Desktop Software.</p>	
<p><b>11.</b> To continue with measurements, select Next Pattern, Next Subject or Repeat Pattern and follow above described measuring process until all the measurements have been conducted.</p>	
<p><b>12.</b> To finish the measurements, press <i>ON/NEXT</i> button or tap <i>Finish</i> on the Device screen.</p>	

### 3.5.2 Display of the Acceleration signal

After completing the Measurements, it is possible to display Acceleration signals for evaluation<sup>28</sup> purposes.

Description of the action	Screen image
<p>1. In the window for Measurement results (chapter 3.5.1, point 8), tap on the <i>Acceleration signal</i> icon on the top right of the screen.</p>	 <p>Peter Schneider Cycling: Vastus m Lying, Right, Central, Relaxed</p> <p>Mean of 3 CV</p> <p>F: 16.2 Hz 0.6 % S: 265 N/m 0.6 % D: 0.82 1.8 % R: 17.7 ms 0.7 % C: 1.05 1.1 %</p> <p>Save Re-Measure</p>
<p>2. To view all Acceleration signals of every single measurement, tap on the <i>LEFT/RIGHT</i> icons.</p>	 <p>Peter Schneider Cycling: Vastus m Lying, Right, Central, Relaxed</p> <p>1/3</p> <p>F: 16.3 Hz S: 267 N/m D: 0.85 R: 17.5 ms C: 1.03 15 ms</p> <p>Save Re-Measure</p>
<p>3. To navigate back to the Measurement results window, tap the icon on the top right again.</p>	 <p>Peter Schneider Cycling: Vastus m Lying, Right, Central, Relaxed</p> <p>1/3</p> <p>F: 16.3 Hz S: 267 N/m D: 0.85 R: 17.5 ms C: 1.03 15 ms</p> <p>Save Re-Measure</p>

<sup>28</sup> The evaluation criteria for the natural oscillation of Tissues are provided in chapter 2.3.

### 3.5.3 Input of new Subjects


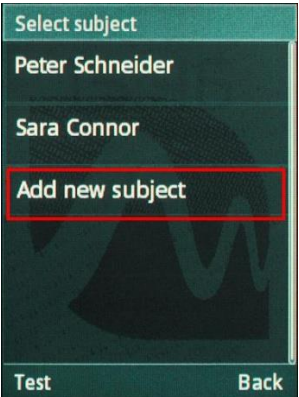
It is possible to enter new Subjects directly in the Device. To use this functionality, activate the *Enabled* option in the *Add new subject in the device* section of the *Preferences* window (chapter 4.2.8) before uploading the Measurement Set (chapter 3.4.3).

**COMMENT** It is recommended to always use the Software for the input of New Subjects (chapter 3.4.1).



Should the newly created Subjects be used for the conduct of measurements, the Subject data (chapter 3.4.1) has to be filled in later during the download process of the measurement results (chapter 3.6.1, point 3).

In order to match the right Subjects with the right personal data, it is important to write down the unique identification numbers shown for each Subject on the Device screen.

Description of the action	Screen image
<p>1. Press the <i>ON/NEXT</i> button or Tap on the <i>Measurements</i> label on the main screen.</p>	
<p>2. To enter a new Subject in the Device, tap on the <i>Add new subject</i> and confirm by tapping on <i>Yes</i>.</p>	


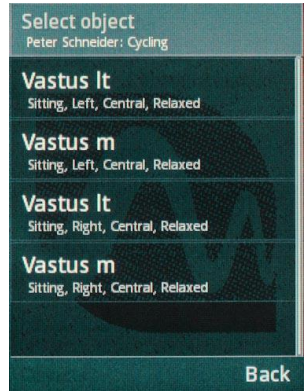
- To select an added Subject, tap on *Subject 0001*.

Each added Subject will be displayed with a new ID number – e.g. *Subject 0002*, etc.





### 3.5.4 Selection of Objects in a different order

During the measurement, it is possible to measure Objects in a different order than in uploaded order.

Description of the action	Screen image
<ol style="list-style-type: none"> <li>Open the window for Measurements (chapter 3.5.1, points 1-5), and tap the <i>Objects</i> in the lower left-corner of the screen.</li> </ol>	
<ol style="list-style-type: none"> <li>Select an Object to proceed with measurement.</li> </ol>	




### 3.5.5 Change of the impulse time


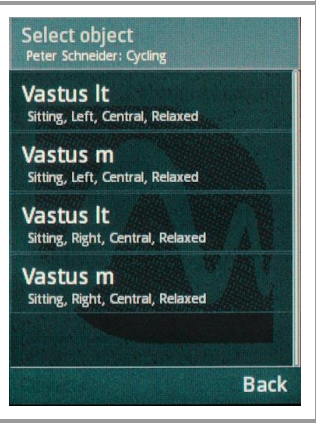
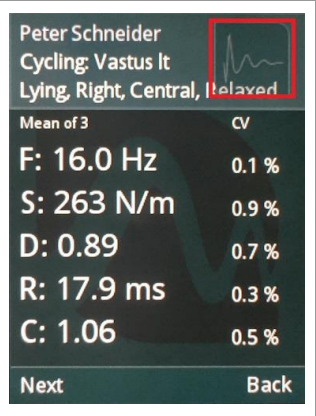
If necessary, it is possible to change the duration of the measurement impulse directly in the Device. The impulse time can be changed for each Object separately.

Description of the action	Screen image
<p>1. Open the Measurements window (chapter 3.5.1, points 1-5), and tap on the <i>Tap time</i> icon for opening the "Set Tap Time" window.</p>	
<p>2. To change Tap time, tap on the <i>UP/DOWN</i> icons, and tap <i>OK</i> when ready.</p>	

### 3.5.6 View of previous measurement results

It is possible to view the completed Measurement results directly on the Device screen.

Description of the action	Screen image
<p>1. Press the <i>ON/NEXT</i> button or tap on the <i>Measurements</i> on the main screen of the Device.</p>	
<p>2. Tap on the Subject name to view.</p>	
<p>3. Tap on the Pattern name to view.</p>	

<p>4. Tap on the measurement <i>Date/Time</i> to view.</p>													
<p>5. Tap on the <i>Object</i> to view.</p>													
<p>6. The measurement result<sup>29</sup> and the CV of the Measurement will be displayed.</p>	 <table border="1" data-bbox="1203 1066 1516 1478"> <thead> <tr> <th>Mean of 3</th> <th>CV</th> </tr> </thead> <tbody> <tr> <td>F: 16.0 Hz</td> <td>0.1 %</td> </tr> <tr> <td>S: 263 N/m</td> <td>0.9 %</td> </tr> <tr> <td>D: 0.89</td> <td>0.7 %</td> </tr> <tr> <td>R: 17.9 ms</td> <td>0.3 %</td> </tr> <tr> <td>C: 1.06</td> <td>0.5 %</td> </tr> </tbody> </table>	Mean of 3	CV	F: 16.0 Hz	0.1 %	S: 263 N/m	0.9 %	D: 0.89	0.7 %	R: 17.9 ms	0.3 %	C: 1.06	0.5 %
Mean of 3	CV												
F: 16.0 Hz	0.1 %												
S: 263 N/m	0.9 %												
D: 0.89	0.7 %												
R: 17.9 ms	0.3 %												
C: 1.06	0.5 %												

<sup>29</sup> The Measurement measured in the Triplescan or Multiscan mode, are displayed on the Device screen as the median or average value (see also chapter "Terminology").



## 3.6 Visualisation of measurement results in the Software

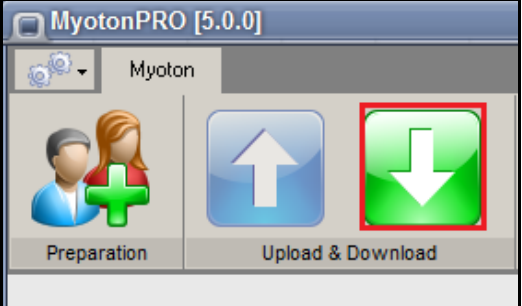

This Chapter describes the options for the visualisation of Measurement results in the Software.

The Software contains a feature of automatic export of the measurement results to Microsoft Excel for further data analysis or manual export to CSV format.

### 3.6.1 Download of measurement results

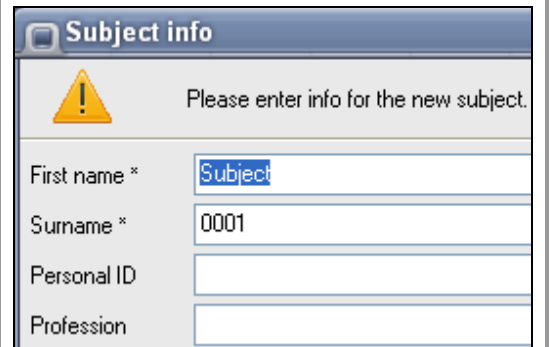
For download of the results, open the Software, and connect the Device to the computer via USB (chapter 3.2.2).

Perform the following actions:

Description of the action	Screen image
<p>1. Click the <i>Download</i> button on the main toolbar of the Software.</p> <p>NB! On connection the Device SN appears on the Software status bar.</p>	
<p>2. The Download window opens and displays the connected Device SN.</p> <p>Before proceeding with download, make sure that the name of the Operator, who did perform the measurements, is correct. If necessary, insert the correct name.</p> <p>NB! Keep results activated if there is a specific need to download the same results also to other databases of the Software.</p> <p>Click <i>Download!</i></p>	

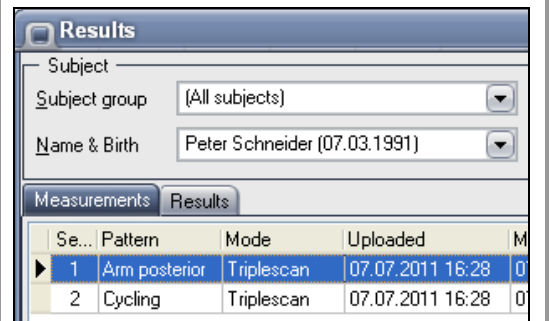
3. If new Subjects were added to the Device during the measurements, the window *Subject info* will open.

Enter the Subject (chapter 3.4.1) and click *OK* to continue with the download.



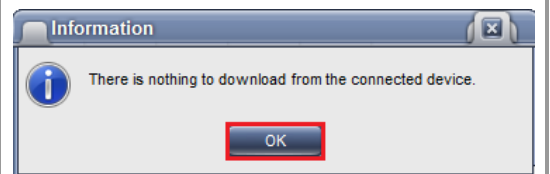

**Do not disconnect the Device until the Download is completed.**

4. Upon completed Download, the Software Results window will open with the downloaded results (chapter 4.2.3).



5. In case there is nothing to Download, the following window will open. Click *OK*.

NB! The Device automatically stores the entire measurement history as a backup until deleted (chapter 3.8.1).





**Before uploading the Measurement Set, download the previous Measurement results from the Device first (chapter 3.6.1).**

**The upload of a new Measurement set deactivates previously measured results waiting for download!**

**Both active and inactive results as a backup of the entire measurement history can be downloaded in the Software under the Device settings (chapter 3.8.1).**

### 3.6.2 Composing the *Symmetry* report

The *Symmetry report* offers the absolute values and Difference % of the left and right sides of the body.



**Myoton AS**  
**MyotonPRO palpation device**  
**Symmetry Report**

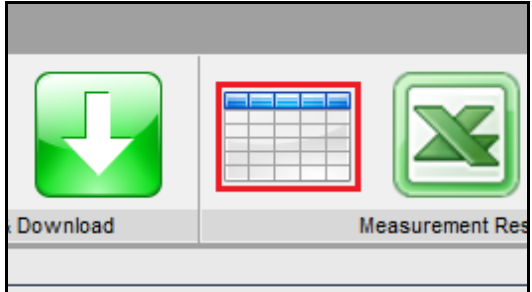
<b>Subject info:</b>	<b>Pattern info:</b>
<b>Name:</b> Peter Schneider	<b>Pattern name:</b> ExampleTriplescan
<b>DOB:</b> 7.03.1981	<b>Mode:</b> Triplescan
<b>Profession:</b> not specified	<b>Tap Qty:</b> 3 taps
<b>Group:</b> Example Group; Test Group	<b>Uploaded:</b> 17.06.2011 16:26:26
<b>Gender:</b> Male	<b>Measured:</b> 17.06.2011 16:27:24
<b>Dominant side:</b> Unknown	<b>Downloaded:</b> 1.07.2011 12:59:47
<b>Weight:</b> 80 kg	<b>Operator:</b> Example Operator
<b>Height:</b> 180 cm	<b>Device SN:</b> 000024
<b>BMI (kg/m2):</b> 24,7	<b>Address:</b> Example Address

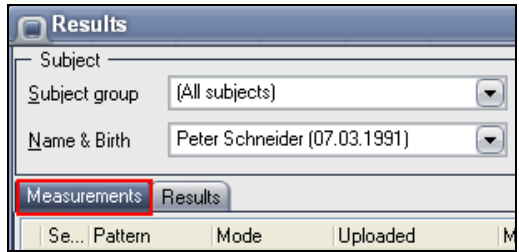
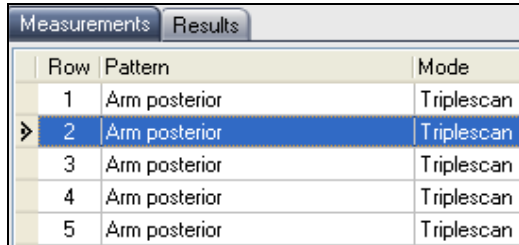
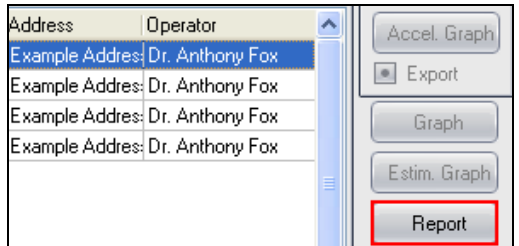
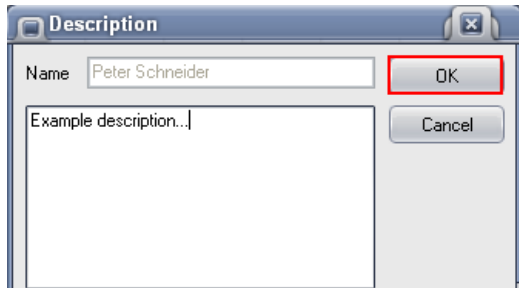
Measurement profile						Measurement series				Results				
Tissue	Object	Position	Side	Location	State	Angle	Tap time	Interval	Delay	Freq Hz	Stiff N/m	Decr	Relax ms	Creep
Muscle	Ext Digitor	Lying	Right	Central	Relaxed	↘	15 ms	0,8 s	0,7 s	31,1	710	1,26	6,9	0,45
Muscle	Ext Digitor	Lying	Left	Central	Relaxed	↘	15 ms	0,8 s	0,7 s	32,1	736	1,22	6,5	0,43
<b>Symmetry index (%):</b>										3,16	3,60	-3,23	-5,97	-4,55

**Description:**

**COMMENT** The *Difference in Symmetry report* above 5% is shown in orange and above 10% is shown in red. The *Difference* is computed:  $(y-x)/((y+x)/2)*100$ .

#### To compose a *Symmetry* report:

Description of the action	Screen image
<ol style="list-style-type: none"> <li>Open the Software and click the <i>Results</i> icon in the main window of the Software.</li> </ol>	

<p>2. Open the <i>Measurements</i> tab in the <i>Results</i> window.</p>																			
<p>3. Select any Measurement row to compose the report.</p>	 <table border="1" data-bbox="980 527 1495 770"> <thead> <tr> <th>Row</th> <th>Pattern</th> <th>Mode</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>Arm posterior</td> <td>Triplescan</td> </tr> <tr style="background-color: #e0e0e0;"> <td>2</td> <td>Arm posterior</td> <td>Triplescan</td> </tr> <tr> <td>3</td> <td>Arm posterior</td> <td>Triplescan</td> </tr> <tr> <td>4</td> <td>Arm posterior</td> <td>Triplescan</td> </tr> <tr> <td>5</td> <td>Arm posterior</td> <td>Triplescan</td> </tr> </tbody> </table>	Row	Pattern	Mode	1	Arm posterior	Triplescan	2	Arm posterior	Triplescan	3	Arm posterior	Triplescan	4	Arm posterior	Triplescan	5	Arm posterior	Triplescan
Row	Pattern	Mode																	
1	Arm posterior	Triplescan																	
2	Arm posterior	Triplescan																	
3	Arm posterior	Triplescan																	
4	Arm posterior	Triplescan																	
5	Arm posterior	Triplescan																	
<p>4. Click the <i>Report</i> button in the <i>Results</i> window.</p> <p>If the <u>Show description on the report</u> was not activated in the <i>Preferences</i> window (chapter 4.2.8), the report will open.</p>																			
<p>5. If the <u>Show description on the report</u> was activated, the <i>Description</i> window opens prior to the report.</p> <p>Enter the description info and click OK to open the report.</p>																			

### 3.6.3 Composing the *Multiscan Analysis* report

The *Multiscan Analysis* is the report where two Multiscan Measurements of a Subject are statistically compared.

Set of measure		Measurement profile	Measurement series	Results					
				Statistics	Freq Hz	Stiff N/m	Decr	Relax ms	Creep
1	Tissue:	Muscle		Average:	15,9	261	0,99	18,5	1,10
	Object:	Ext Digitor	Angle: ↘	Standard deviation:	0,1	4	0,02	0,2	0,01
	Position:	Sitting	Tap time: 15 ms	Variation coefficient %:	0,9	1,7	2,0	1,0	1,2
	Side:	Left	Interval: 0,8 s	Minimum:	15,7	255	0,95	18,2	1,08
	Location:	Central	Delay: 1 s	Maximum:	16,1	266	1,01	18,7	1,12
	State:	Relaxed		Median:	16,0	263	0,99	18,7	1,10
2	Tissue:	Muscle		Average:	16,9	281	0,82	16,7	0,98
	Object:	Ext Digitor	Angle: ↘	Standard deviation:	0,1	2	0,02	0,1	0,01
	Position:	Sitting	Tap time: 15 ms	Variation coefficient %:	0,3	0,8	2,2	0,8	1,1
	Side:	Left	Interval: 0,8 s	Minimum:	16,9	278	0,80	16,5	0,97
	Location:	Central	Delay: 1 s	Maximum:	17,0	285	0,85	16,9	1,00
	State:	Relaxed		Median:	16,9	281	0,82	16,7	0,98
Difference of averages (%):					6,1	7,4	-18,8	-10,2	-11,5
Significance by T-test ( $p < 0,05$ ):					TRUE	TRUE	TRUE	TRUE	TRUE

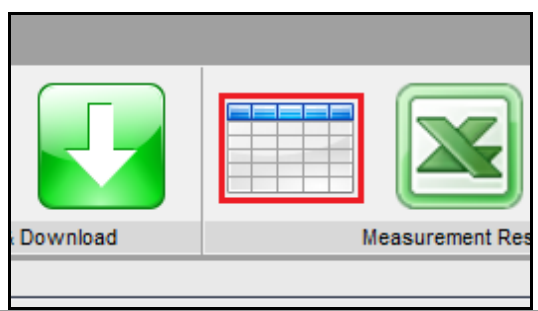
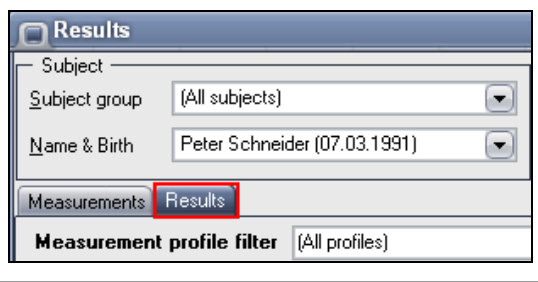
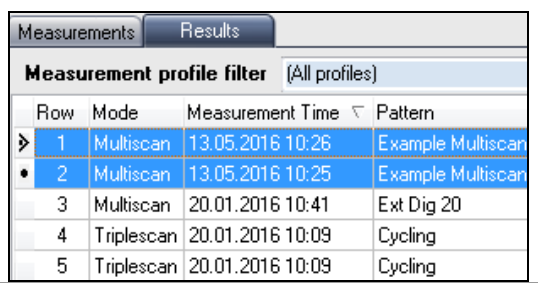
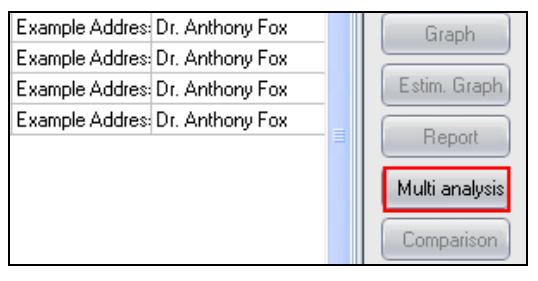
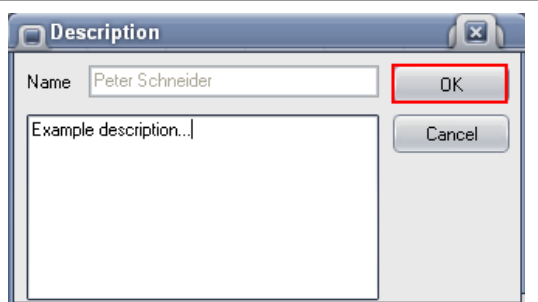
The *Multiscan Analysis* report, offers the statistics for all five parameters of the Measurement:

- *Arithmetical average;*
- *Standard deviation;*
- *Variation coefficient (%) (CV);*
- *Minimum value;*
- *Maximum value;*
- *Median;*
- *Difference of averages (%);*
- *Significance by Student's T-test (sign. 5%).*




The *Multiscan Analysis* report can be composed only for the measurements performed in the Multiscan mode (chapter 3.4.2).

To create a *Multiscan Analysis* report:

Description of the action	Screen image																								
<p>1. Open the Software and click the <i>Results</i> button in the main window of the Software.</p>																									
<p>2. Open the <i>Results</i> tab in the <i>Results</i> window.</p>																									
<p>3. Select only two rows of <i>Multiscan</i> measurement results to compose the report.</p> <p>Double-click the first row and then single-click the second row or by the keyboard hold down the <i>Ctrl</i> key while selecting rows in interest.</p>	 <table border="1" data-bbox="979 1197 1513 1386"> <thead> <tr> <th>Row</th> <th>Mode</th> <th>Measurement Time</th> <th>Pattern</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>Multiscan</td> <td>13.05.2016 10:26</td> <td>Example Multiscan</td> </tr> <tr> <td>2</td> <td>Multiscan</td> <td>13.05.2016 10:25</td> <td>Example Multiscan</td> </tr> <tr> <td>3</td> <td>Multiscan</td> <td>20.01.2016 10:41</td> <td>Ext Dig 20</td> </tr> <tr> <td>4</td> <td>Triplscan</td> <td>20.01.2016 10:09</td> <td>Cycling</td> </tr> <tr> <td>5</td> <td>Triplscan</td> <td>20.01.2016 10:09</td> <td>Cycling</td> </tr> </tbody> </table>	Row	Mode	Measurement Time	Pattern	1	Multiscan	13.05.2016 10:26	Example Multiscan	2	Multiscan	13.05.2016 10:25	Example Multiscan	3	Multiscan	20.01.2016 10:41	Ext Dig 20	4	Triplscan	20.01.2016 10:09	Cycling	5	Triplscan	20.01.2016 10:09	Cycling
Row	Mode	Measurement Time	Pattern																						
1	Multiscan	13.05.2016 10:26	Example Multiscan																						
2	Multiscan	13.05.2016 10:25	Example Multiscan																						
3	Multiscan	20.01.2016 10:41	Ext Dig 20																						
4	Triplscan	20.01.2016 10:09	Cycling																						
5	Triplscan	20.01.2016 10:09	Cycling																						
<p>4. Click the <i>Multi analysis</i> button in the <i>Results</i> window.</p> <p>If the <i>Show description on the report</i> was not activated in the <i>Preferences</i> window (chapter 4.2.8), the report will open.</p>																									
<p>5. If the <i>Show description on the report</i> was activated, the <i>Description</i> window opens prior to the report. Enter the description info and click OK to open the report.</p>																									

### 3.6.4 Composing the *Comparison* report

The *Comparison* report offers the absolute values and Difference (%) of any two selected measurements.



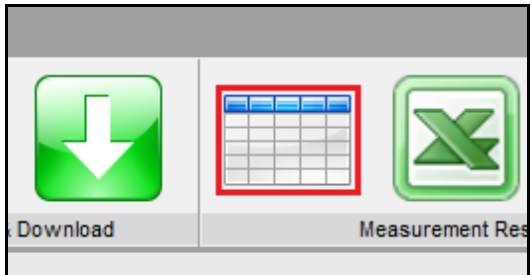
#### Myoton AS MyotonPRO palpation device Comparison Report

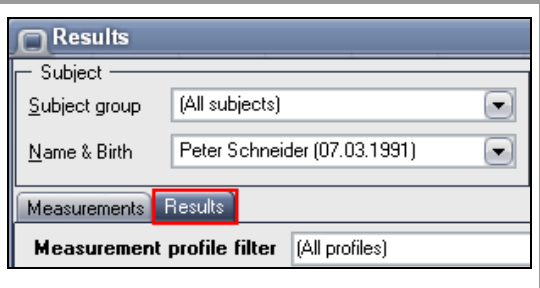
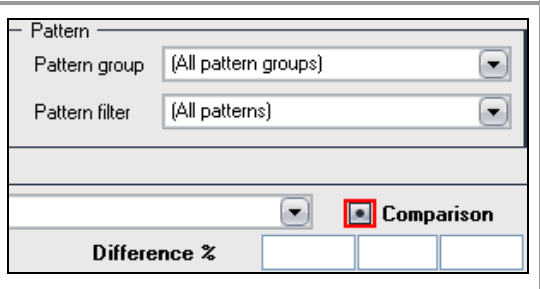
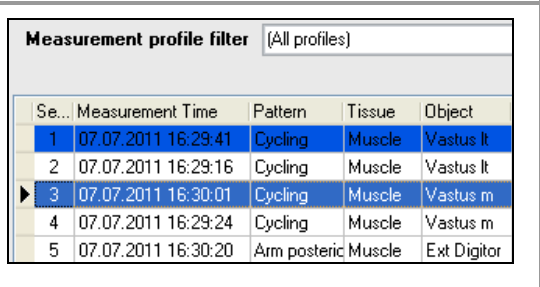
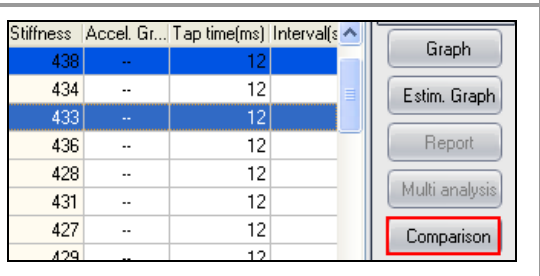
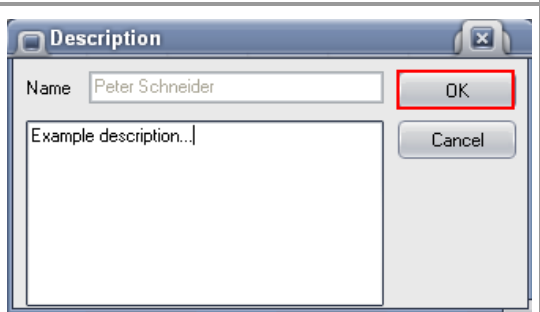
<b>Subject info:</b>		<b>Pattern info (set 1):</b>		<b>Pattern info (set 2):</b>	
Name:	Peter Schneider	Pattern name:	Cycling	Pattern name:	Cycling
DOB:	7.03.1981	Mode:	Triplescan	Mode:	Triplescan
Profession:	notspecified	Tap Qty:	3 taps	Tap Qty:	3 taps
Group:	Example Group; Test Grou	Uploaded:	20.01.201610:07:34	Uploaded:	20.01.201610:07:34
Gender:	Male	Measured:	20.01.201610:09:16	Measured:	20.01.201610:09:16
Dominant side:	Right	Downloaded:	20.01.201610:09:51	Downloaded:	20.01.201610:09:51
Weight:	80 kg	Operator:	Test Operator	Operator:	Test Operator
Height:	180 cm	Device SN:	000173	Device SN:	000173
BMI (kg/m2):	24,7	Address:	Tallinn,Estonia	Address:	Tallinn,Estonia

Time of measure	Measurement profile	Measurement series	Results				
			Freq Hz	Stiff N/m	Decr	Relax ms	Creep
20.01.2016 10:09:31	Tissue: Muscle Object: Vastus m Position: Lying Side: Right Location: Central State: Relaxed	Angle: ↓ Tap time: 15 ms Interval: 0,4 s Delay: 0,7 s	22,4	485	0,82	10,5	0,67
20.01.2016 10:09:22	Tissue: Muscle Object: Vastus m Position: Lying Side: Left Location: Central State: Relaxed	Angle: ↓ Tap time: 15 ms Interval: 0,4 s Delay: 0,7 s	22,6	495	0,83	10,3	0,66
<i>Difference (%)</i> :			0,9	2	1,2	-1,9	-1,5

**COMMENT** The *Difference in Symmetry* report above 5% is shown in orange and above 10% is shown in red. The *Difference* is computed:  $(y-x)/((y+x)/2)*100$ .

To compose a *Comparison* report:

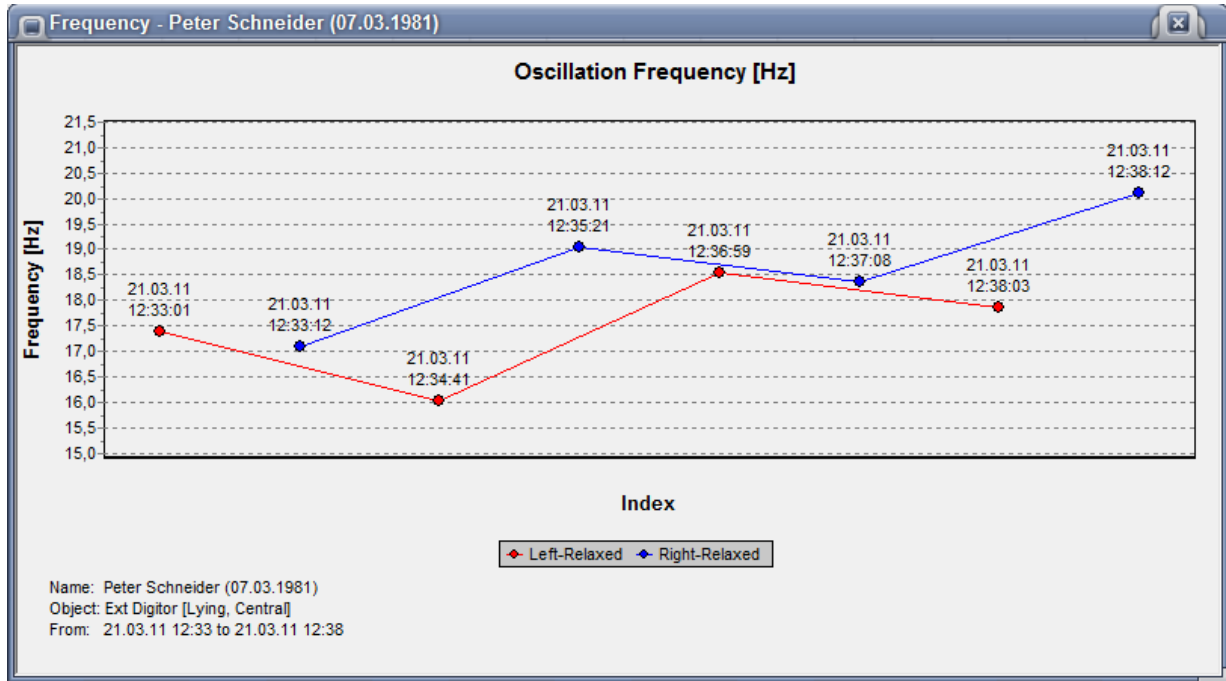
Description of the action	Screen image
<ol style="list-style-type: none"> <li>Open the Software and click the <i>Results</i> button in the main window of the Software.</li> </ol>	

<p>2. Open the <i>Results</i> tab in the <i>Results</i> window.</p>																																					
<p>3. Click the <i>Comparison</i> option in the <i>Results</i> tab.</p>																																					
<p>4. Select only two rows of any Measurement results to compose the report.</p> <p>Double-click the first row and then single-click the second row or by the keyboard hold down the <i>Ctrl</i> key while selecting rows of interest.</p>	 <table border="1"> <thead> <tr> <th>Se...</th> <th>Measurement Time</th> <th>Pattern</th> <th>Tissue</th> <th>Object</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>07.07.2011 16:29:41</td> <td>Cycling</td> <td>Muscle</td> <td>Vastus It</td> </tr> <tr> <td>2</td> <td>07.07.2011 16:29:16</td> <td>Cycling</td> <td>Muscle</td> <td>Vastus It</td> </tr> <tr> <td>3</td> <td>07.07.2011 16:30:01</td> <td>Cycling</td> <td>Muscle</td> <td>Vastus m</td> </tr> <tr> <td>4</td> <td>07.07.2011 16:29:24</td> <td>Cycling</td> <td>Muscle</td> <td>Vastus m</td> </tr> <tr> <td>5</td> <td>07.07.2011 16:30:20</td> <td>Arm posteric</td> <td>Muscle</td> <td>Ext Digitor</td> </tr> </tbody> </table>	Se...	Measurement Time	Pattern	Tissue	Object	1	07.07.2011 16:29:41	Cycling	Muscle	Vastus It	2	07.07.2011 16:29:16	Cycling	Muscle	Vastus It	3	07.07.2011 16:30:01	Cycling	Muscle	Vastus m	4	07.07.2011 16:29:24	Cycling	Muscle	Vastus m	5	07.07.2011 16:30:20	Arm posteric	Muscle	Ext Digitor						
Se...	Measurement Time	Pattern	Tissue	Object																																	
1	07.07.2011 16:29:41	Cycling	Muscle	Vastus It																																	
2	07.07.2011 16:29:16	Cycling	Muscle	Vastus It																																	
3	07.07.2011 16:30:01	Cycling	Muscle	Vastus m																																	
4	07.07.2011 16:29:24	Cycling	Muscle	Vastus m																																	
5	07.07.2011 16:30:20	Arm posteric	Muscle	Ext Digitor																																	
<p>5. For PDF report click the <i>Comparison</i> button in the <i>Results</i> window.</p> <p>If the <i>Show description on the report</i> was not activated in the <i>Preferences</i> window (chapter 4.2.8), the report will open.</p>	 <table border="1"> <thead> <tr> <th>Stiffness</th> <th>Accel. Gr...</th> <th>Tap time(ms)</th> <th>Intervals</th> </tr> </thead> <tbody> <tr><td>438</td><td>--</td><td>12</td><td></td></tr> <tr><td>434</td><td>--</td><td>12</td><td></td></tr> <tr><td>433</td><td>--</td><td>12</td><td></td></tr> <tr><td>436</td><td>--</td><td>12</td><td></td></tr> <tr><td>428</td><td>--</td><td>12</td><td></td></tr> <tr><td>431</td><td>--</td><td>12</td><td></td></tr> <tr><td>427</td><td>--</td><td>12</td><td></td></tr> <tr><td>429</td><td>--</td><td>12</td><td></td></tr> </tbody> </table>	Stiffness	Accel. Gr...	Tap time(ms)	Intervals	438	--	12		434	--	12		433	--	12		436	--	12		428	--	12		431	--	12		427	--	12		429	--	12	
Stiffness	Accel. Gr...	Tap time(ms)	Intervals																																		
438	--	12																																			
434	--	12																																			
433	--	12																																			
436	--	12																																			
428	--	12																																			
431	--	12																																			
427	--	12																																			
429	--	12																																			
<p>6. If the <i>Show description on the report</i> was activated, the <i>Description</i> window opens prior to the report.</p> <p>Enter the description info and click <i>OK</i> to open the report.</p>																																					



### 3.6.5 Composing the graphs

It is possible to compose basic graphs for each parameter of the Measurement Profile of the selected Subject. The graph displays one Measurement Profile of the left and the right side of the body, both in relaxed and contracted states.

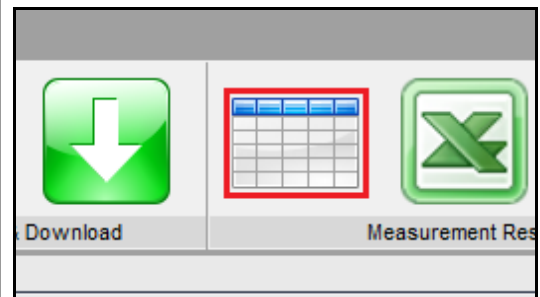


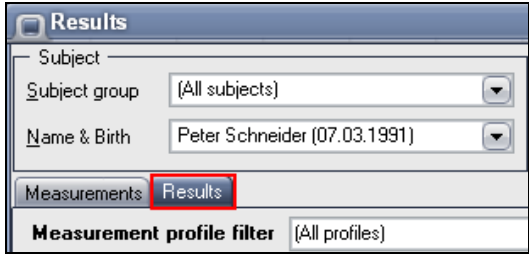
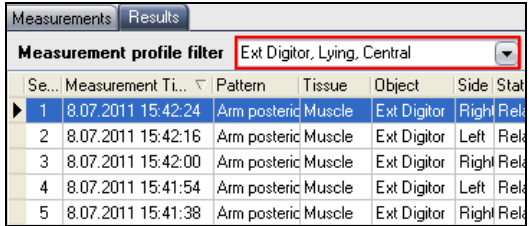
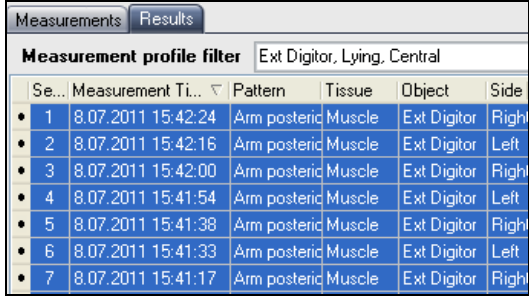
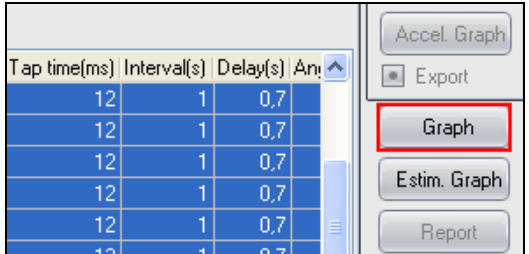
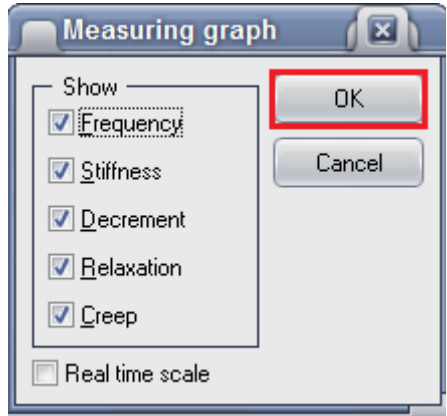
On the graph making the right-click with the mouse button opens the options for:

- Copy
- Print
- Save

**COMMENT** To zoom in on an area on the graph, mark the area from left to right. To zoom out, mark any area from right to left. The limit values of the Y-axis can be changed in the Software *Preferences* window (chapter 4.2.8).

For the composing of graphs:

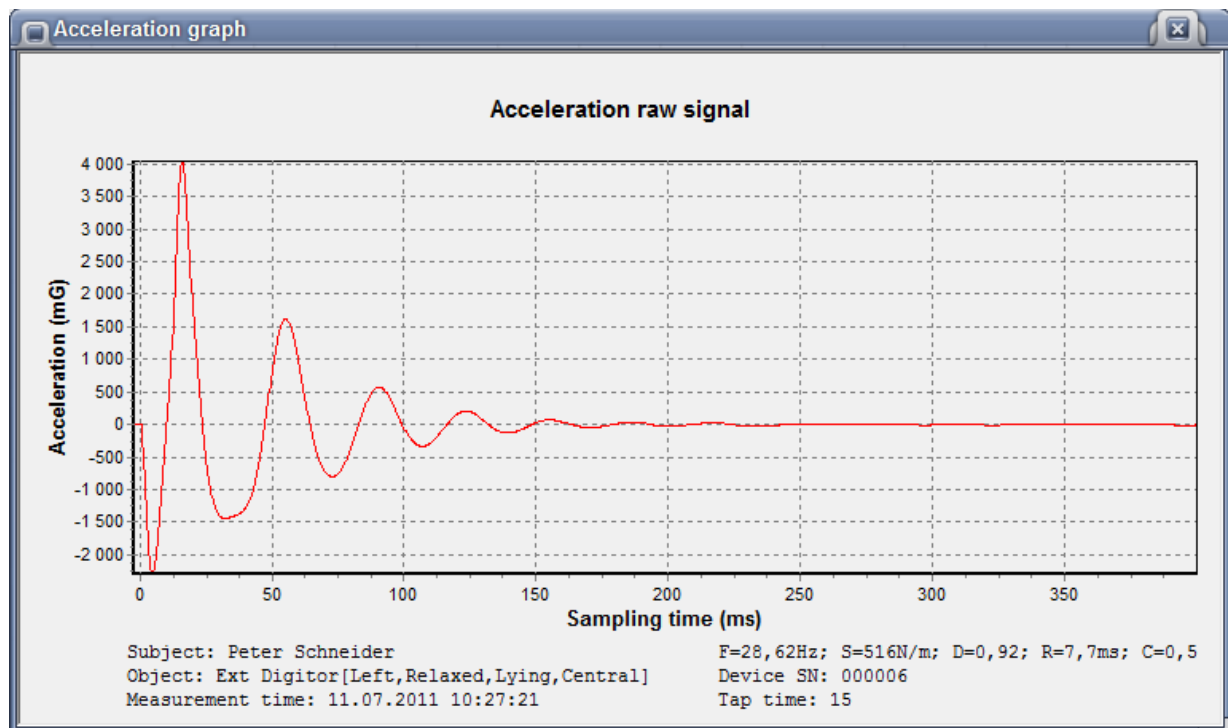
Description of the action	Screen image
<ol style="list-style-type: none"> <li>1. Open the Software and click the <i>Results</i> button in the main window of the Software.</li> </ol>	

<p>2. Open the <i>Results</i> tab in the <i>Results</i> window.</p>	
<p>3. Select one Measurement profile from the <i>Measurement profile filter</i>.</p>	
<p>4. Select rows on the Results tab, to compose the graphs.</p> <p>The rows can be selected with the mouse and/or with the keyboard using <i>Ctrl</i>, <i>Shift</i> and arrow keys.</p>	
<p>5. Click the <i>Graph</i> button in the <i>Results</i> window.</p>	
<p>6. In the <i>Measuring graph</i> window, check the parameters to compose graphs and click OK.</p> <p>The <i>Real time scale</i> checkbox enables to display the Measurement results in the real time scale.</p>	

### 3.6.6 Display of the acceleration signal

It is possible to store the raw acceleration signal of the oscillation for every single Measurement. The raw signal consists of 1280 rows of data and can be exported to CSV file format if necessary.

The raw acceleration signal is displayed in the Software *Acceleration graph* window:



On the graph making the right-click with the mouse button opens the options for:

- Copy
- Print
- Save

**COMMENT** To zoom in on an area on the graph, mark the area from left to right. To zoom out, mark any area from right to left.

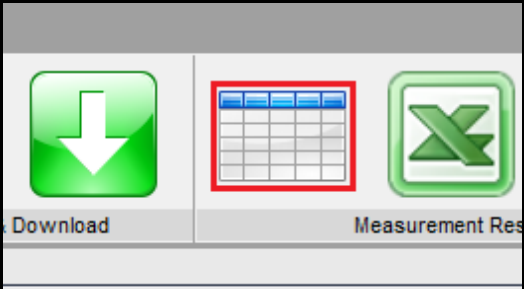
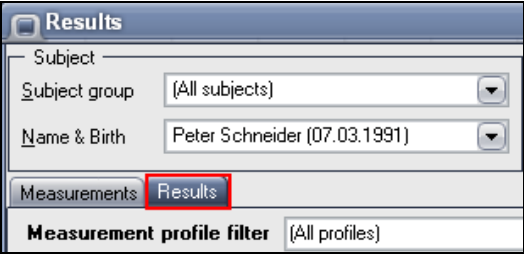
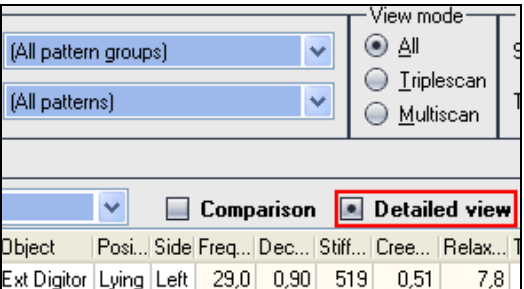
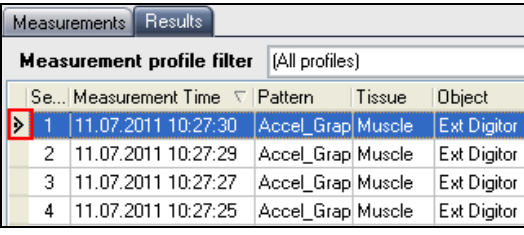
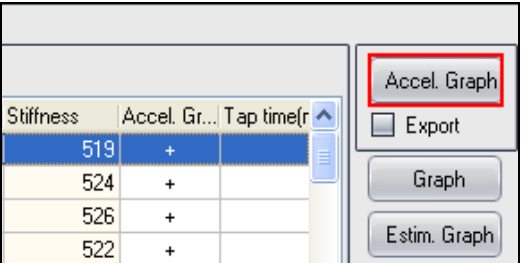


To enable the functionality of storing the raw acceleration signal:

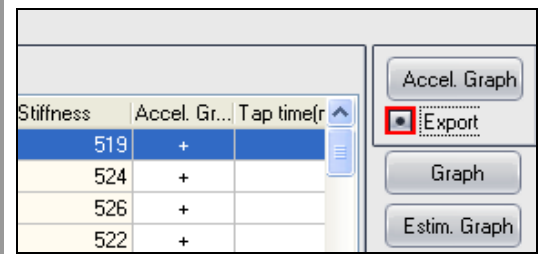
- Activate the **Accel.Signal** checkbox during composing of a new Pattern (chapter 3.4.2, point 12).
- Upload the new Measurement set with the composing Pattern to the Device.

To save memory, do not activate **Accel.Signal** unless certainly needed!

To display the raw acceleration signal:

Description of the action	Screen image																									
<p>1. Open the Software and click the <i>Results</i> button in the main window of the Software.</p>																										
<p>2. Open the <i>Results</i> tab in the <i>Results</i> window.</p>																										
<p>3. Activate the <i>Detailed view</i> checkbox in the <i>Results</i> tab.</p>																										
<p>4. Select one row of the Measurement result in the <i>Results</i> tab, the acceleration signal to view.</p>	 <table border="1" data-bbox="987 1186 1507 1419"> <thead> <tr> <th>Se...</th> <th>Measurement Time</th> <th>Pattern</th> <th>Tissue</th> <th>Object</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>11.07.2011 10:27:30</td> <td>Accel_Grap</td> <td>Muscle</td> <td>Ext Digitor</td> </tr> <tr> <td>2</td> <td>11.07.2011 10:27:29</td> <td>Accel_Grap</td> <td>Muscle</td> <td>Ext Digitor</td> </tr> <tr> <td>3</td> <td>11.07.2011 10:27:27</td> <td>Accel_Grap</td> <td>Muscle</td> <td>Ext Digitor</td> </tr> <tr> <td>4</td> <td>11.07.2011 10:27:25</td> <td>Accel_Grap</td> <td>Muscle</td> <td>Ext Digitor</td> </tr> </tbody> </table>	Se...	Measurement Time	Pattern	Tissue	Object	1	11.07.2011 10:27:30	Accel_Grap	Muscle	Ext Digitor	2	11.07.2011 10:27:29	Accel_Grap	Muscle	Ext Digitor	3	11.07.2011 10:27:27	Accel_Grap	Muscle	Ext Digitor	4	11.07.2011 10:27:25	Accel_Grap	Muscle	Ext Digitor
Se...	Measurement Time	Pattern	Tissue	Object																						
1	11.07.2011 10:27:30	Accel_Grap	Muscle	Ext Digitor																						
2	11.07.2011 10:27:29	Accel_Grap	Muscle	Ext Digitor																						
3	11.07.2011 10:27:27	Accel_Grap	Muscle	Ext Digitor																						
4	11.07.2011 10:27:25	Accel_Grap	Muscle	Ext Digitor																						
<p>5. Click the <i>Accel.Graph</i> button in the <i>Results</i> window, to view the signal</p>																										

6. To export the signal, activate the *Export* option and click the *Accel.Graph* button.



**COMMENT** In the *Results* tab, column *Accel.Graph*, rows marked with "+", show the measurements for which the acceleration signal has been stored.

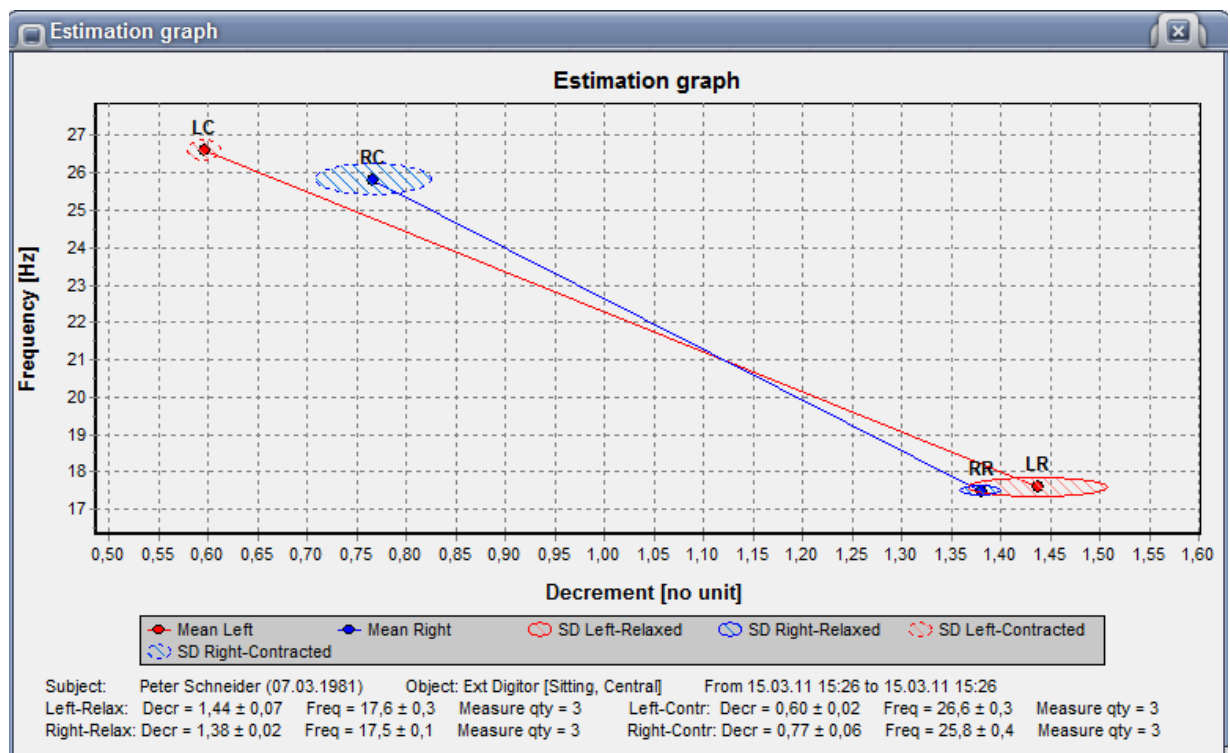
### 3.6.7 Muscle function *Estimation* graph

The *Estimation* graph enables to evaluate the muscle function as an indication of the muscle health condition.

The Estimation graph describes the muscle function by the relocation of the intersection point of two parameters, the Frequency (F) and Decrement (D), in a relaxed vs contracted state.

**The criteria of healthy muscle condition:**

The results plotted on the following graph demonstrate the healthy muscle condition by the inverse relocation of the Frequency and Decrement. Therefore, the F value increases in the contracted state, and the D value decreases, compared to the results measured in a relaxed state.



**NB!** The larger the difference and relocation of the intersection points of F and D parameters (relaxed vs. contracted state), the higher the muscle function.

The *Estimation* graph displays four intersection points:

- LC - Left Contracted
- RC - Right Contracted
- RR - Right Relaxed
- LR - Left Relaxed

On the graph making the right-click with the mouse button opens the options for:

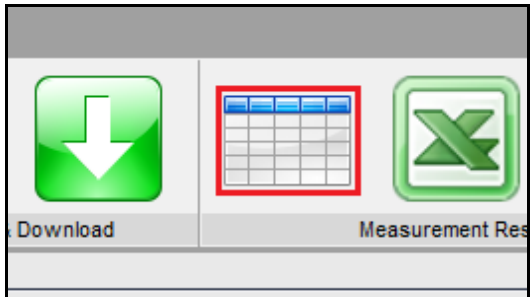
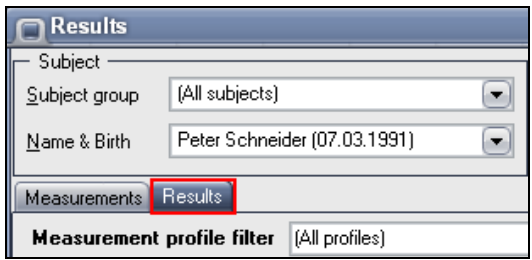
- Copy
- Print
- Save

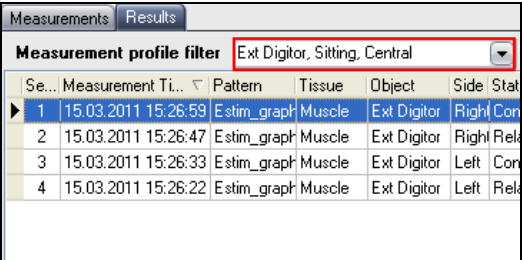
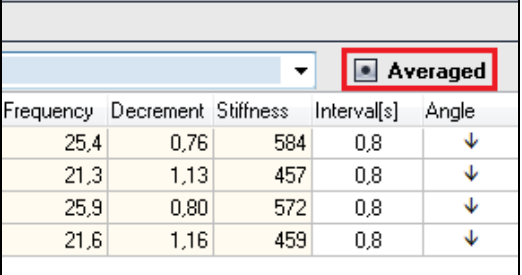
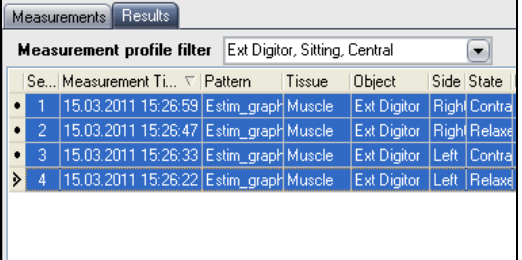
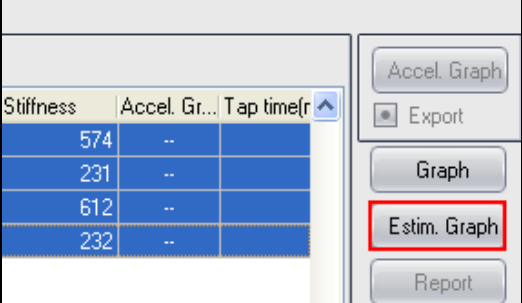
**COMMENT** To zoom in on an area on the graph, mark the area from left to right. To zoom out, mark any area from right to left. The limit values of the Y-axis can be changed in the Software *Preferences* window (chapter 4.2.8).



Composing the *Estimation* graph requires the muscle in interest measured on both body sides, first in a fully relaxed and then in contracted state, thus requiring four measurements.

To create the *Estimation* graph:

Description of the action	Screen image
<p>1. Open the Software and click the <i>Results</i> button in the main window of the Software.</p>	
<p>2. Open the <i>Results</i> tab in the <i>Results</i> window.</p>	

<p>3. Select one Measurement profile from the <i>Measurement profile filter</i>.</p>																										
<p>4. Select the option box <i>Averaged</i>.</p>	 <table border="1" data-bbox="987 625 1505 783"> <thead> <tr> <th>Frequency</th> <th>Decrement</th> <th>Stiffness</th> <th>Interval[s]</th> <th>Angle</th> </tr> </thead> <tbody> <tr> <td>25,4</td> <td>0,76</td> <td>584</td> <td>0,8</td> <td>↓</td> </tr> <tr> <td>21,3</td> <td>1,13</td> <td>457</td> <td>0,8</td> <td>↓</td> </tr> <tr> <td>25,9</td> <td>0,80</td> <td>572</td> <td>0,8</td> <td>↓</td> </tr> <tr> <td>21,6</td> <td>1,16</td> <td>459</td> <td>0,8</td> <td>↓</td> </tr> </tbody> </table>	Frequency	Decrement	Stiffness	Interval[s]	Angle	25,4	0,76	584	0,8	↓	21,3	1,13	457	0,8	↓	25,9	0,80	572	0,8	↓	21,6	1,16	459	0,8	↓
Frequency	Decrement	Stiffness	Interval[s]	Angle																						
25,4	0,76	584	0,8	↓																						
21,3	1,13	457	0,8	↓																						
25,9	0,80	572	0,8	↓																						
21,6	1,16	459	0,8	↓																						
<p>5. Select at least 4 results from both relaxed and contracted state of the same Object, on both sides of the body.</p> <p>The rows can be selected with the mouse and/or with the keyboard using <i>Ctrl</i>, <i>Shift</i> and arrow keys.</p>																										
<p>6. Click the <i>Estim.Graph</i> button to open the graph.</p>																										

## 3.7 Export of the measurement results

The Software contains an important feature of automatic export of measurement results to Microsoft Excel for further data analysis, statistics and visualization in the form of charts. Automatic export to Excel is compatible with Excel 2010 and later versions.

**There are two options to export the Measurement results:**

- Automatic export to the XLSM file format (chapter 3.7.1),
- Manual export to the CSV file format (chapter 3.7.2).

### 3.7.1 Automatic export to Microsoft Excel

The Software contains a convenient feature of Automatic data export from the connected database into Microsoft Excel for Windows for further analysis and visualization.

The Excel file of Automatic export contains a macro (the pre-programmed task) for data import and refresh of Pivot tables.

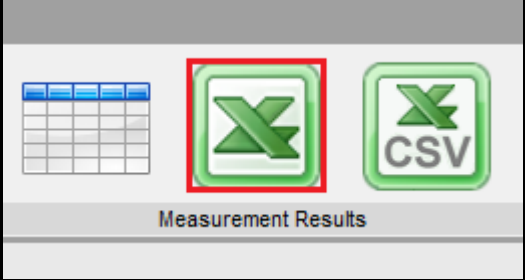
In Excel file the imported Measurement results are located in the sheet “Database”, where columns Z to AD contain some prepared formulas containing functions for data processing tasks.



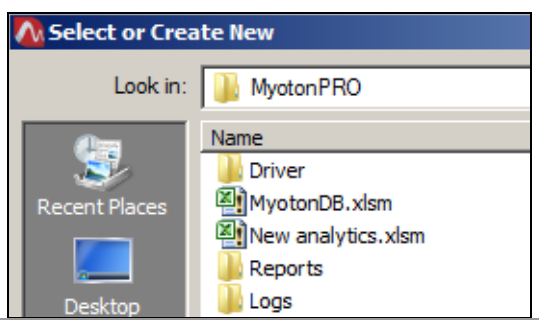
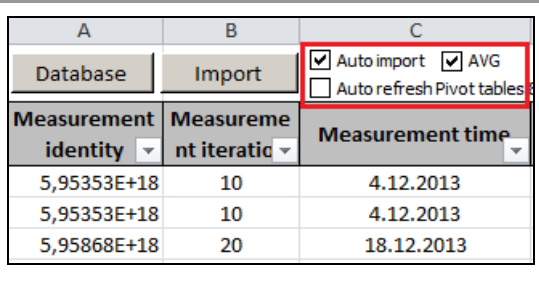
**Automatic export enables to keep the Excel reports up-to-date every time when data is being downloaded from the connected device.**

**NB! Automatic export is compatible only with the Microsoft Excel for Windows.**

**For the automatic export of results:**

Description of the action	Screen image
<p>1. Open the Software, click the <i>Automatic export</i> on the main toolbar.</p> <p>Wait until the export is completed and the Excel file opens automatically.</p>	



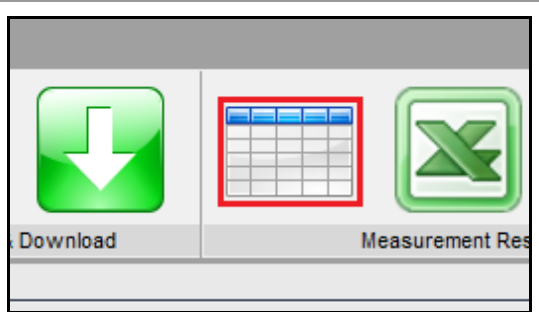
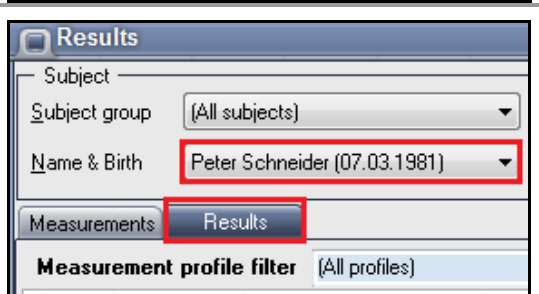
<p>2. For connecting the Excel file with the Myoton database, click the <i>Automatic export</i> button, select an Excel file (*.xslm) and click OK.</p> <p>To create a new Excel file, write a name of the new file name at the bottom of the window and click OK.</p>																			
<p>3. The “Database” sheet contains following preferences:</p> <ul style="list-style-type: none"> <li>• Auto import – on opening the file the data is imported automatically.</li> <li>• On check AVG – only average values of measurements will be imported.</li> <li>• Auto re-fresh of Pivot tables.</li> </ul>	 <table border="1" data-bbox="980 548 1515 840"> <thead> <tr> <th>A</th> <th>B</th> <th>C</th> </tr> </thead> <tbody> <tr> <td>Database</td> <td>Import</td> <td><input checked="" type="checkbox"/> Auto import <input checked="" type="checkbox"/> AVG <input type="checkbox"/> Auto refresh Pivot tables</td> </tr> <tr> <th>Measurement identity</th> <th>Measurement iteration</th> <th>Measurement time</th> </tr> <tr> <td>5,95353E+18</td> <td>10</td> <td>4.12.2013</td> </tr> <tr> <td>5,95353E+18</td> <td>10</td> <td>4.12.2013</td> </tr> <tr> <td>5,95868E+18</td> <td>20</td> <td>18.12.2013</td> </tr> </tbody> </table>	A	B	C	Database	Import	<input checked="" type="checkbox"/> Auto import <input checked="" type="checkbox"/> AVG <input type="checkbox"/> Auto refresh Pivot tables	Measurement identity	Measurement iteration	Measurement time	5,95353E+18	10	4.12.2013	5,95353E+18	10	4.12.2013	5,95868E+18	20	18.12.2013
A	B	C																	
Database	Import	<input checked="" type="checkbox"/> Auto import <input checked="" type="checkbox"/> AVG <input type="checkbox"/> Auto refresh Pivot tables																	
Measurement identity	Measurement iteration	Measurement time																	
5,95353E+18	10	4.12.2013																	
5,95353E+18	10	4.12.2013																	
5,95868E+18	20	18.12.2013																	

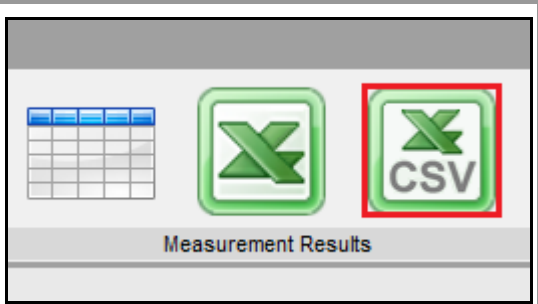
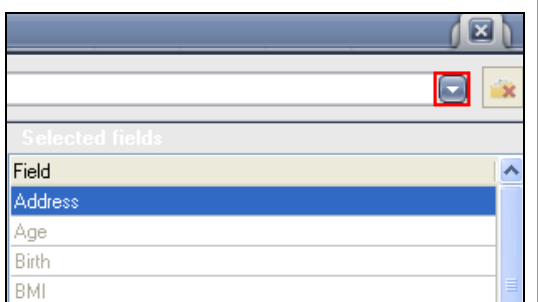
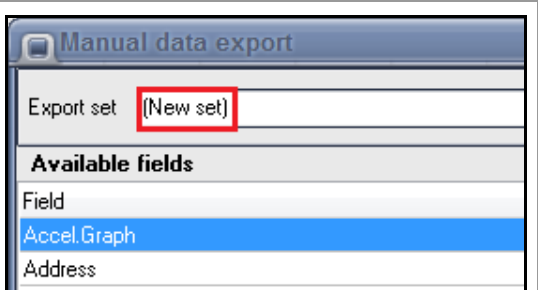
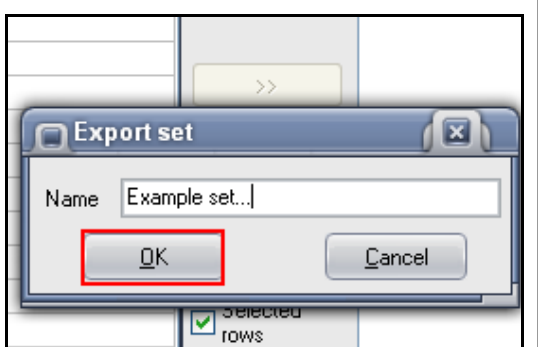
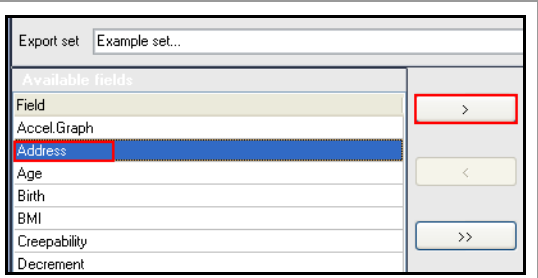
### 3.7.2 Manual export


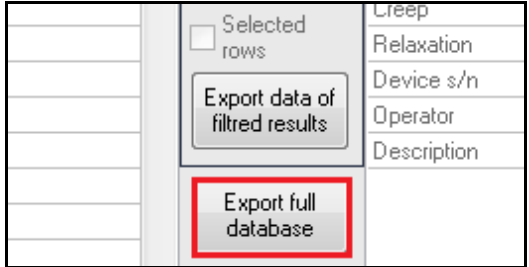
The manual export offers an option to export the data to the CSV file format (chapter 4.2.4).

**COMMENT** To export the entire database, click Manual export icon without opening the Results tab. To export selected results, open the Results window first, make a selection of the results and then click the Manual export icon.

**For the manual export of selected Results to the CSV:**

Description of the action	Screen image
<p>1. Open the Software and click the <i>Results</i> icon on the main toolbar.</p>	
<p>2. Open the Results tab and choose the Subject whose results to export.</p>	

<p>3. Click the <i>Manual export</i> icon on the main toolbar.</p>	
<p>4. Select an existing <i>Export set</i> or create new.</p> <p>The data fields to be exported will be shown in the <i>Selected fields</i> section.</p>	
<p>5. To create new export set, select New set.</p>	
<p>6. Name the set and click OK.</p>	
<p>7. Make a selection of data fields to export by double-click or use arrow buttons &gt;.</p>	

<p><b>8.</b> To export results displayed in the Results tab, click <i>Export data of filtered results</i>.</p> <p>The <i>Manual export</i> window closes automatically on completed export.</p>	 <p>A screenshot of the software's export menu. On the left is a table with several empty rows. To the right of the table are two buttons: 'Export data of filtered results' (highlighted with a red border) and 'Export full database'. Further to the right is a list of column headers: Creep, Relaxation, Device s/n, Operator, and Description.</p>
<p><b>9.</b> To export the entire content of the connected database click <i>Export full database</i>.</p>	 <p>A screenshot of the software's export menu, similar to the one above. In this instance, the 'Export full database' button is highlighted with a red border. The 'Export data of filtered results' button is also visible but not highlighted.</p>

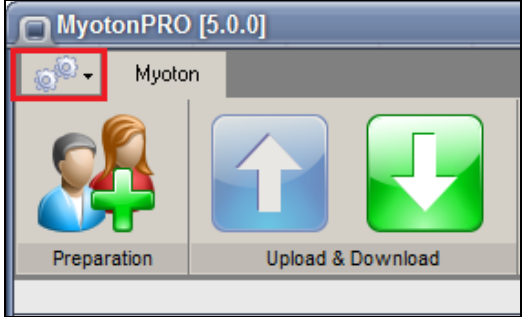
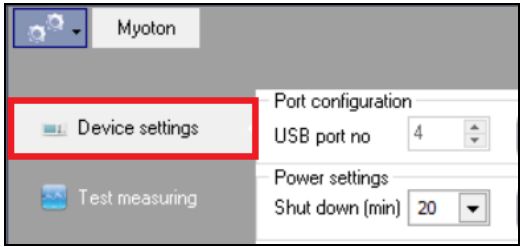
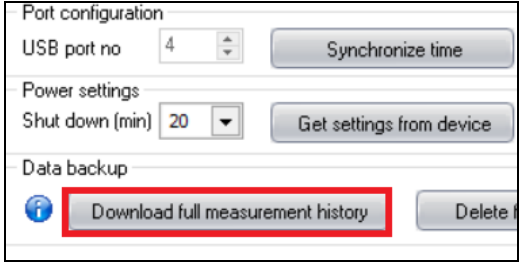
## 3.8 Data management

### 3.8.1 Data backup of the Device

The Device contains the **entire measurement history as a backup**. The backup can be downloaded to a new or to any existing database of the Software.

**COMMENT** In the Device measurement history, all single measurements of the Measurements are stored separately.

To download the entire measurement history:

Description	Screen image
<p>1. Open the Software and click the <i>Settings</i> icon on the main toolbar.</p>	
<p>2. In the <i>Settings</i> menu, click the <i>Device settings</i> button.</p>	
<p>3. Click the <i>Download entire measurement history</i> button.</p> <p>Follow the instructions of the download dialogue.</p>	

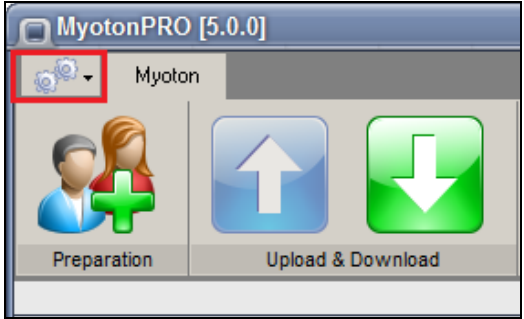
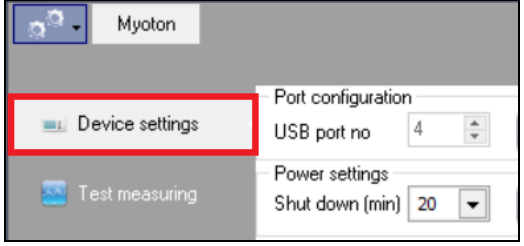
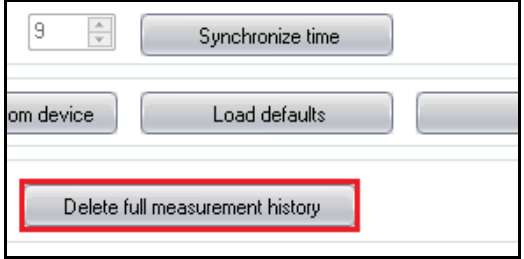
### 3.8.2 Deletion of backup from the Device

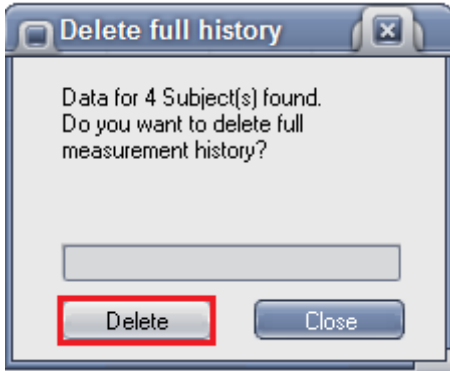
The entire measurement history of the Device can be deleted in the *Device settings* window located under the *Settings* menu (chapter 4.2.5).



**Before delete, make sure all Measurement results are downloaded (chapter 3.8.1). Delete of the measurement history permanent and cannot be undone!**

To delete the entire measurement history from the connected Device:

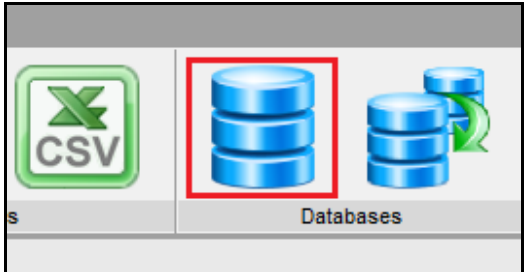
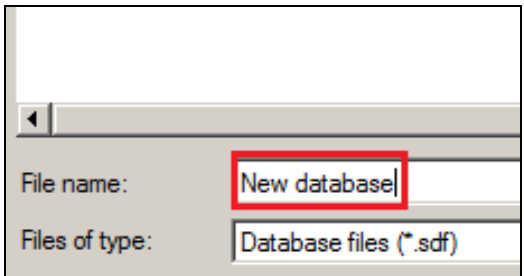
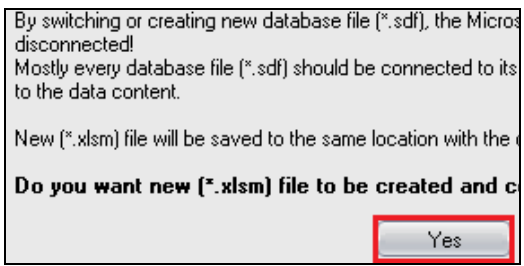
Description	Screen image
<p>1. Open the Software and connect the Device with the computer. Click the <i>Settings</i> icon on the main toolbar.</p>	
<p>2. In the <i>Settings</i> menu, click the <i>Device settings</i> button.</p>	
<p>3. Click the <i>Delete entire measurement history</i> button.</p>	

<p>4. Click the <i>Delete</i> button.</p> <p>After the deletion is completed click <i>Close</i>.</p>	
--	---

### 3.8.3 Software database management

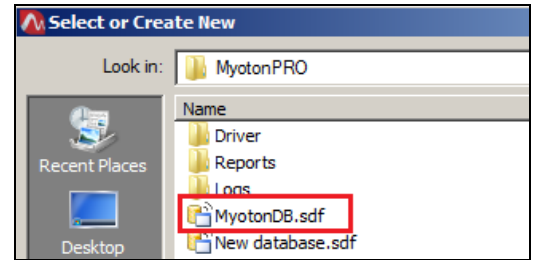
Multiple databases can be used in the Software. It is recommended to create a new database for every research project.

To switch or create a new database:

Description	Screen image
<p>1. Open the Software and click the <i>Database</i> icon on the main toolbar.</p> <p>The window <i>Select or Create New</i> will open.</p>	
<p>2. To create a new database, just write a new name for the new database into the File name field and click <i>OK</i>.</p>	
<p>3. For a new database, the dialogue window will open.</p> <p>To connect a new Excel file to the new database Click <i>YES</i>.</p>	<p>By switching or creating new database file (*.sdf), the Micros disconnected! Mostly every database file (*.sdf) should be connected to its to the data content.</p> <p>New (*.xslm) file will be saved to the same location with the c</p> <p><b>Do you want new (*.xslm) file to be created and c</b></p> 

4. To switch the database, click the database icon on the main toolbar and double-click any database file (extension \*.sdf) in the window *Select or Create New*.

NB! Database files can be saved to any location of the computer.

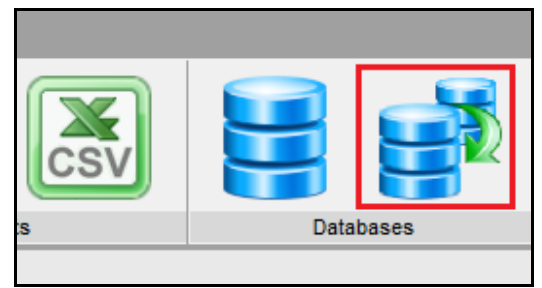
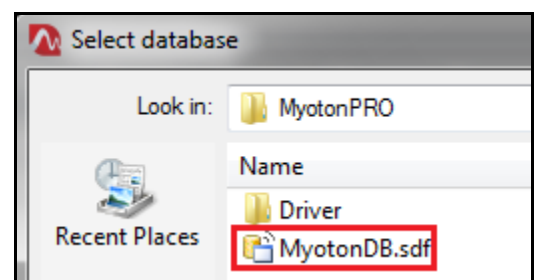


### 3.8.4 Import of database

*Import of database* enables data import from any Myoton database (\*.sdf).

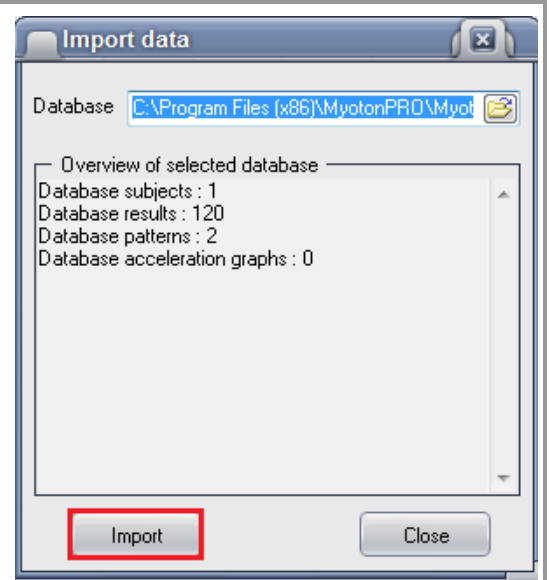
The feature can be used for a multicentre study, where separate devices and computers have been used to collect measurements. The function enables the consolidation of all results from different database files into one database before statistical analysis.

To import database, perform the following actions:

Description	Screen image
<p>1. Open the Software and click the <i>Import database</i> icon on the main toolbar.</p> <p>The <i>Select database</i> window will open.</p>	
<p>2. Double-click the database file (extension *.sdf) in the <i>Select database</i> window.</p> <p>The <i>Import data</i> window will open.</p>	

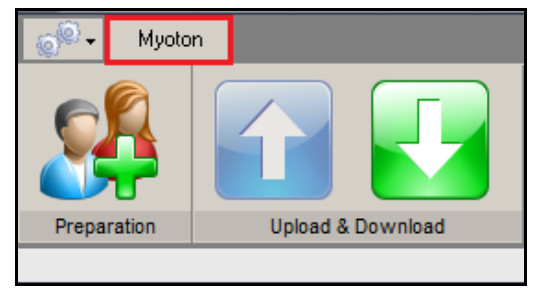
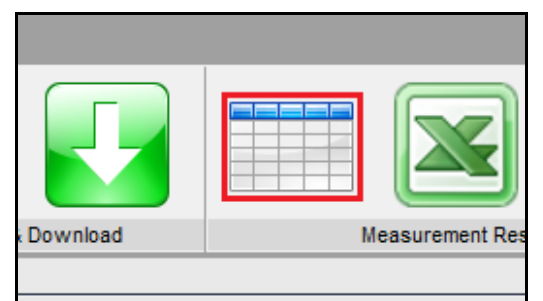
- Click *Import* button in the *Import data* window to start the import.

Wait until the import is successfully completed and click *Close*.

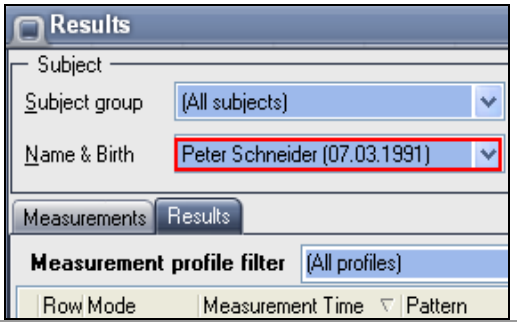
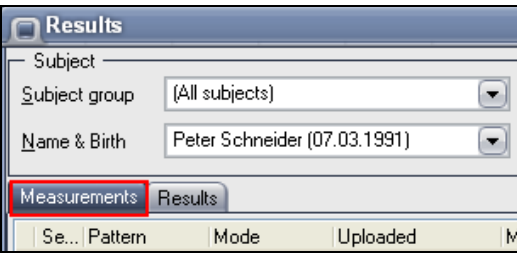
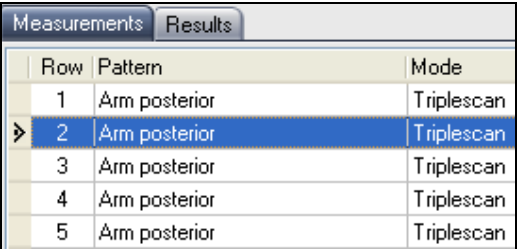
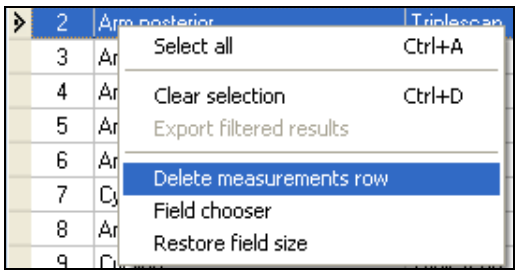


### 3.8.5 Deletion of measurement results from the Software

Measurement results can be deleted from the Software in the *Results* window (chapter 4.2.3):

Description	Screen image
<ol style="list-style-type: none"> <li>Open the Software and make sure the <i>Myoton</i> (Main) menu is activated.</li> </ol>	
<ol style="list-style-type: none"> <li>Click the <i>Results</i> button on the main toolbar.</li> </ol>	



<p>3. Choose a Subject from the <i>Name &amp; Birth</i> field whose Measurement results you want to delete.</p>																									
<p>4. Open the <i>Measurements</i> tab in the <i>Results</i> window.</p>																									
<p>5. Select row(s) to delete.</p>	 <table border="1"> <thead> <tr> <th>Row</th> <th>Pattern</th> <th>Mode</th> </tr> </thead> <tbody> <tr> <td>1</td> <td>Arm posterior</td> <td>TripleScan</td> </tr> <tr style="background-color: #e0e0e0;"> <td>2</td> <td>Arm posterior</td> <td>TripleScan</td> </tr> <tr> <td>3</td> <td>Arm posterior</td> <td>TripleScan</td> </tr> <tr> <td>4</td> <td>Arm posterior</td> <td>TripleScan</td> </tr> <tr> <td>5</td> <td>Arm posterior</td> <td>TripleScan</td> </tr> </tbody> </table>	Row	Pattern	Mode	1	Arm posterior	TripleScan	2	Arm posterior	TripleScan	3	Arm posterior	TripleScan	4	Arm posterior	TripleScan	5	Arm posterior	TripleScan						
Row	Pattern	Mode																							
1	Arm posterior	TripleScan																							
2	Arm posterior	TripleScan																							
3	Arm posterior	TripleScan																							
4	Arm posterior	TripleScan																							
5	Arm posterior	TripleScan																							
<p>6. Open the <i>Pop-up</i> window with the right-click mouse button and click on <i>Delete measurements row</i>.</p>	 <table border="1"> <tbody> <tr> <td>2</td> <td>Arm posterior</td> <td>TripleScan</td> </tr> <tr> <td>3</td> <td>Ar</td> <td></td> </tr> <tr> <td>4</td> <td>Ar</td> <td></td> </tr> <tr> <td>5</td> <td>Ar</td> <td></td> </tr> <tr> <td>6</td> <td>Ar</td> <td></td> </tr> <tr> <td>7</td> <td>Cy</td> <td></td> </tr> <tr> <td>8</td> <td>Ar</td> <td></td> </tr> <tr> <td>9</td> <td>C</td> <td></td> </tr> </tbody> </table>	2	Arm posterior	TripleScan	3	Ar		4	Ar		5	Ar		6	Ar		7	Cy		8	Ar		9	C	
2	Arm posterior	TripleScan																							
3	Ar																								
4	Ar																								
5	Ar																								
6	Ar																								
7	Cy																								
8	Ar																								
9	C																								





If a row of measurements is deleted from the *Measurements* tab, then that row's measurement results will also be deleted automatically from the *Results* tab.


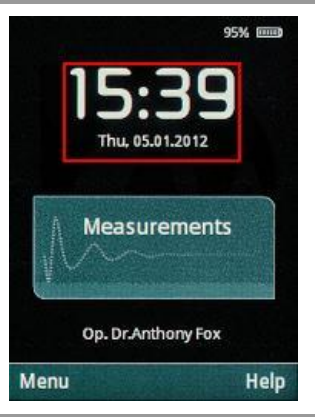


## 4. Menu instructions

### 4.1 Device menu

#### 4.1.1 The main screen

The main screen buttons and icons:

Button/icons	Screen image
<p>1. The <b>Measurements</b> icon opens the:</p> <ul style="list-style-type: none"> <li>• Test measurement mode (chapter 3.3.1)</li> <li>• Measurement mode (chapter 3.5)</li> </ul>	 <p>The screenshot shows a mobile interface with a black background. At the top right, there is a battery icon and '95%'. The time '15:39' is displayed in large white font, with 'Thu, 05.01.2012' below it. A central button labeled 'Measurements' with a white waveform icon is highlighted with a red rectangular box. At the bottom, there is a dark teal bar with 'Menu' and 'Help' buttons.</p>
<p>2. The <b>Menu</b> contains:</p> <ul style="list-style-type: none"> <li>• Shut down the Device (chapter 4.1.2)</li> <li>• System info (chapter 4.1.3)</li> <li>• Settings (chapter 4.1.4)</li> <li>• Probe replacement (chapter 4.1.5)</li> </ul>	 <p>The screenshot is identical to the one above, showing the same mobile interface. However, the 'Menu' button in the bottom dark teal bar is highlighted with a red rectangular box.</p>



<p>3. The <b>Help</b> contains the following items:</p> <ul style="list-style-type: none"> <li>• ATTENTION!</li> <li>• User manual</li> <li>• Quick instruction for use</li> <li>• Measurable parameters</li> <li>• Important information</li> <li>• <i>ON/NEXT</i> button</li> </ul>	 <p>The screenshot shows the device's main interface with a black background. At the top right, the battery level is 95%. The time is 15:39 and the date is Thu, 05.01.2012. A central teal box contains a waveform and the word 'Measurements'. Below it, the operator name 'Op. Dr.Anthony Fox' is displayed. At the bottom, there are two buttons: 'Menu' on the left and 'Help' on the right, which is highlighted with a red rectangular box.</p>
<p>4. The <b>Date &amp; time</b> format: <i>d dd.mm.yyyy hh:mm</i>.</p> <p>The date and time are automatically synchronised when connecting the Device to the Software (chapter 3.2.2) or synchronised manually (chapter 4.2.5).</p>	 <p>The screenshot is identical to the previous one, but the time '15:39' and date 'Thu, 05.01.2012' are enclosed in a red rectangular box to highlight the format.</p>
<p>5. The <b>battery icon</b> at the top-right corner of the Device screen provides a visual indication of the battery level. A yellow lightning mark over the icon shows the battery is being charged.</p>	 <p>The screenshot is identical to the previous ones, but the battery icon at the top right now shows a yellow lightning bolt, indicating that the battery is being charged. The battery level is 97%.</p>
<p>6. The <b>Operator name</b> is uploaded to the Device from the Software together with the Measurement Set (chapter 3.4.3).</p>	 <p>The screenshot is identical to the previous ones, but the operator name 'Op. Dr.Anthony Fox' is highlighted with a red rectangular box.</p>

## 4.1.2 Device Shut Down

By default, the Device is shutting down in 10 minutes. The default values can be changed in the Software under the Device settings (chapter 4.2.5). The manual shut down locates under the Device menu.



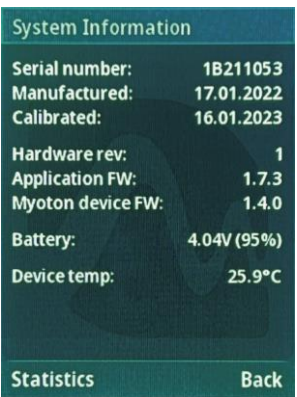
Time	1 min	1 min	Shut down time - 8 min
Display	ON	Dim	Off
System	Operational		
Lights	Off	30sec interval	

For the manual shut down of the Device:

Description of the action	Screen image
1. Open the Device main screen and tap the <i>Menu</i> button.	
2. Tap the <i>Shut down</i> button on the Device screen.	

## 4.1.3 Device system info

To display the Device system info:



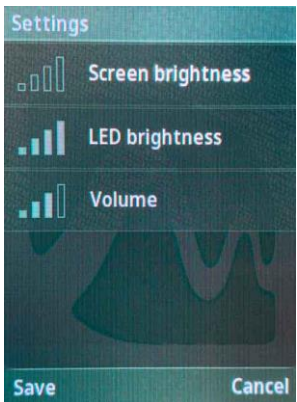
Description of the action	Screen image
<p>1. Open the Device main screen and tap the <i>Menu</i> button.</p>	
<p>2. Tap the <i>System info</i> in the <i>Menu</i> window.</p>	
<p><b>3. The System information of the Device:</b></p> <ul style="list-style-type: none"> <li>• <i>Serial number</i> – <i>Device SN</i></li> <li>• <i>Manufactured</i> – manufacturing date;</li> <li>• <i>Calibrated</i> – date of calibration<sup>30</sup>;</li> <li>• <i>Hardware rev</i> – hardware version;</li> <li>• <i>Application FW</i> – firmware version number;</li> <li>• <i>Myoton device FW</i> – firmware version number;</li> <li>• <i>Battery</i> – <i>battery voltage V</i></li> <li>• <i>Device temp</i> – Device temperature C°</li> </ul>	

<sup>30</sup> The Device does not require re-calibration.

## 4.1.4 Device settings

The settings are set in the Device and the Software (chapter 4.2.5).

To change the settings in the Device:




Description of the action	Screen image
<p>1. Open the main screen of the Device and tap the <i>Menu</i> button.</p>	
<p>2. Tap the <i>Settings</i> button in the <i>Menu</i> window.</p>	
<p>3. The following settings can be changed in the Device:</p> <ul style="list-style-type: none"> <li>• <i>Screen brightness</i>;</li> <li>• <i>LED brightness</i>;</li> <li>• <i>Volume</i></li> </ul> <p>Confirm the settings by taping <i>Save</i> or click <i>Cancel</i>.</p>	

## 4.1.5 Device probe replacement

The Device probe can be replaced:

- with the skin probes or long probes that are suitable for tissues difficult to access,
- for sterilization purposes (e.g. if tissues with mucous membrane are measured).

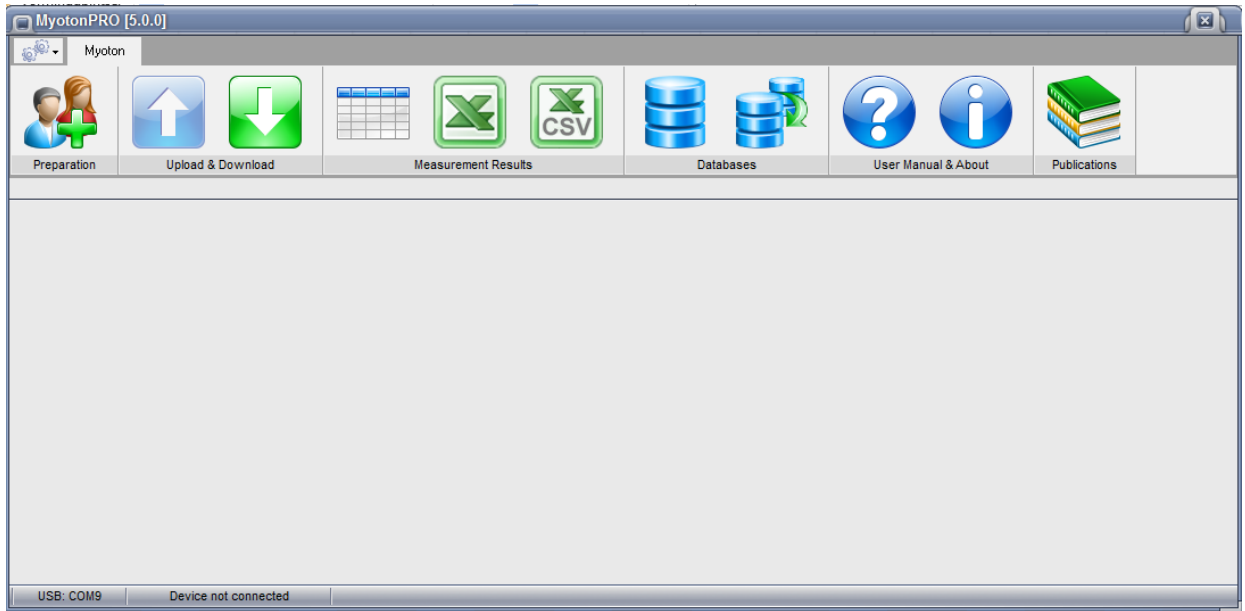
To change the Probe:

Description of the action	Screen image
<ol style="list-style-type: none"> <li>1. Open the main screen of the Device and tap on the <i>Menu</i> button.</li> </ol>	
<ol style="list-style-type: none"> <li>2. Tap on the <i>Probe</i> button in the Menu window.</li> </ol>	
<ol style="list-style-type: none"> <li>3. Pull out the Probe,</li> <li>4. Insert a new Probe,</li> <li>5. Tap the <i>Done</i> button.</li> </ol> <p>The Probe is held in the Device with a permanent magnet.</p>	

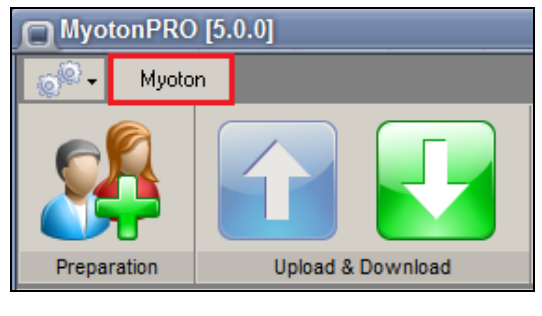
## 4.2 Software menu

### 4.2.1 The main window

On opening the Software, the main window is displayed:



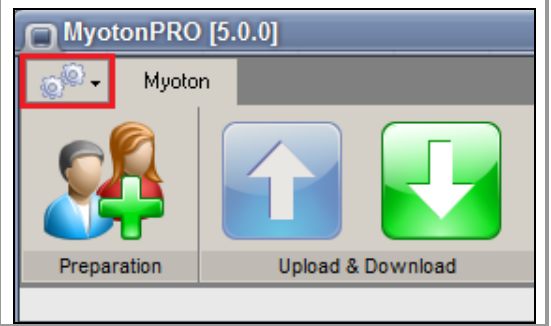
There are two tabs in the main window of the Software (Settings and Myoton):

Description of menus	Screen image
<p>1. <b>Myoton</b> menu contains:</p> <ul style="list-style-type: none"> <li>• Preparation of the Measurement set</li> <li>• Upload &amp; Download function</li> <li>• Display and export of Measurement results</li> <li>• The User Manual and About window</li> <li>• Management of Databases</li> <li>• Access to the list of Myoton publications on myoton.com</li> </ul>	

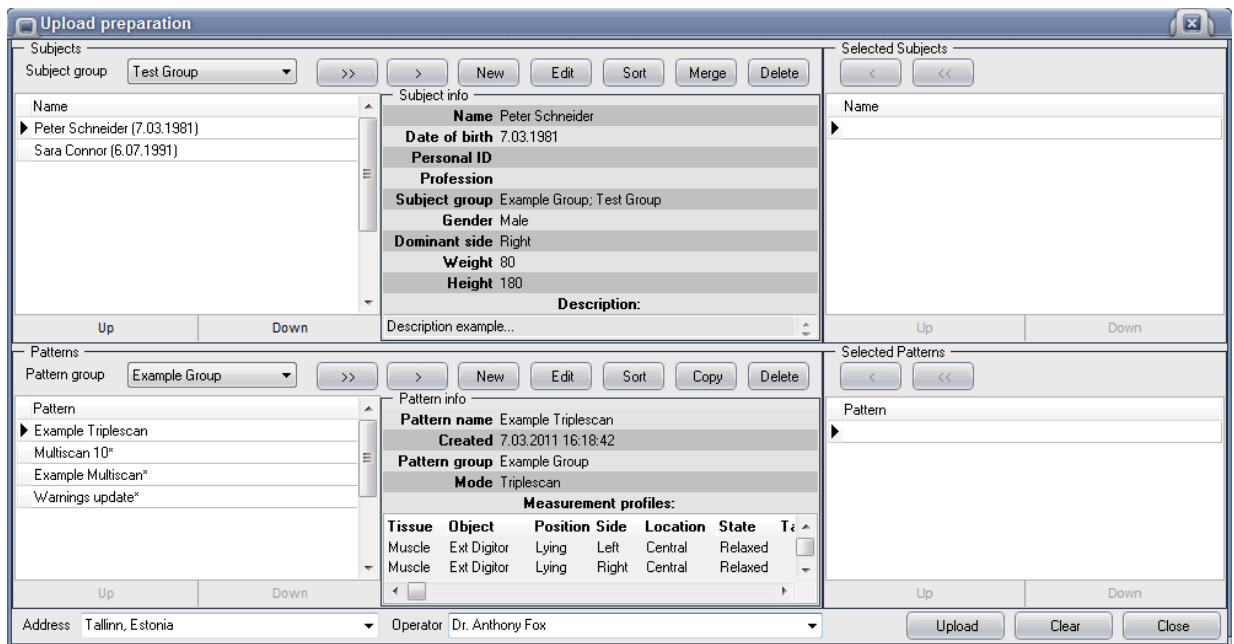


2. **Settings** menu enables to open:

- *Device settings*
- *Test measuring*
- *Locations*
- *Preferences*
- *Deleted items*



## 4.2.2 Upload Preparation window



To open the **Upload Preparation** window, click the following icon in the main toolbar:



The **Upload Preparation** window contains:

- Entering the Subject information (chapter 3.4.1),
- Composing and handling of Patterns (chapter 3.4.2),
- Upload the Measurement set (chapter 3.4.3).

**The mouse right-click pop-up menu for Subjects or Patterns:**

- **Select subject/pattern** – sends either the activated Subject or Pattern into the section Selected Subjects or Selected Patterns;
- **New subject/pattern** - opens either the Pattern Composer or Subject info window to enter the information of the new Pattern or Subject;
- **Edit subject/pattern** - opens either the Pattern Composer or Subject info window to change the information of the activated Pattern or Subject;
- **Sort** – enables to sort Subjects and Pattern in the alphabetic or date order;
- **Change groups** – opens either the Subject Groups or Pattern Groups window to change the groups of the activated Subject or Pattern;
- **Delete subject/pattern** - deletes either the activated Subject or Pattern.

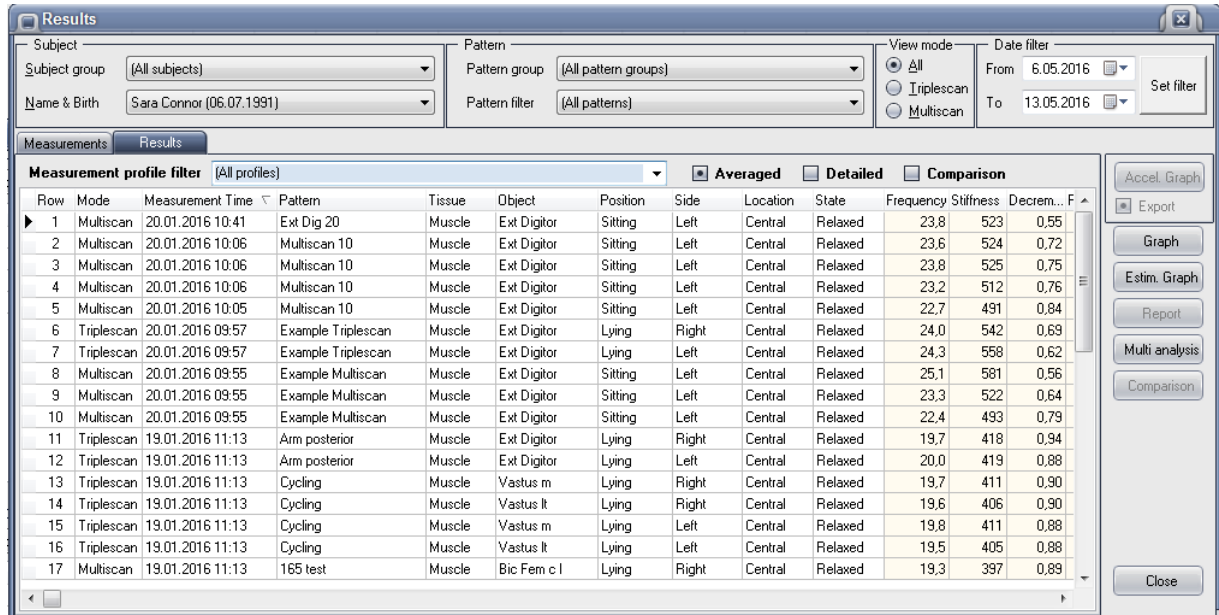
**Functions of the buttons in the Upload Preparation window:**

- **Arrow right >** – either sends the selected Subjects or Patterns from the Subjects or Patterns section into the Upload section;
- **Double arrow right >>** – either sends the entire group of Subjects or Patterns from the Subjects or Patterns section into the Upload section;
- **Arrow left <** – either sends the selected Subject or Pattern from the Upload section back to the Subjects or Patterns section;
- **Double arrow left <<** – either sends all Subjects or Patterns from the Upload section back to the Subjects or Patterns section;
- **New** - opens either the Pattern Composer or Subject info window to enter the information of the new Pattern or Subject;
- **Edit** - opens either the Pattern Composer or Subject info window to change the information of the activated Subject or Pattern;
- **Sort** – enables to sort Subjects and Pattern in the alphabetic or date order;
- **Merge** – enables to merge<sup>31</sup> two or more Subjects, including their measurement results under one Subject name.
- **Copy** – enables to copy the content of existing Pattern into a new Pattern with a different name e.g. for PRE and POST data collection;
- **Delete** - deletes either the activated Subject or Pattern;
- **Up** – moves either the name of the selected Subject or Pattern upwards within the respective section;
- **Down** – moves either the name of the selected Subject or Pattern downwards within the respective section;
- **Upload button** – performs the upload of Subject and Patterns onto the Device;
- **Clear** – sends the Subject and Patterns from the sections Selected Subjects and Selected Patterns back to the section Subjects and Patterns;
- **Close** – closes the Upload Preparation window.

---

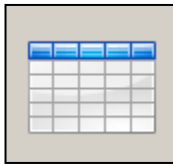
<sup>31</sup> Enables to merge two or more Subjects and their measurement results in case a user has created the same Subject more than once and has collected the measurement results under different names of the same Subject.

## 4.2.3 Results window



Row	Mode	Measurement Time	Pattern	Tissue	Object	Position	Side	Location	State	Frequency	Stiffness	Decrem... F
1	Multiscan	20.01.2016 10:41	Ext Dig 20	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	23,8	523	0,55
2	Multiscan	20.01.2016 10:06	Multiscan 10	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	23,6	524	0,72
3	Multiscan	20.01.2016 10:06	Multiscan 10	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	23,8	525	0,75
4	Multiscan	20.01.2016 10:06	Multiscan 10	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	23,2	512	0,76
5	Multiscan	20.01.2016 10:05	Multiscan 10	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	22,7	491	0,84
6	Triplescan	20.01.2016 09:57	Example Triplescan	Muscle	Ext Digitor	Lying	Right	Central	Relaxed	24,0	542	0,69
7	Triplescan	20.01.2016 09:57	Example Triplescan	Muscle	Ext Digitor	Lying	Left	Central	Relaxed	24,3	558	0,62
8	Multiscan	20.01.2016 09:55	Example Multiscan	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	25,1	581	0,56
9	Multiscan	20.01.2016 09:55	Example Multiscan	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	23,3	522	0,64
10	Multiscan	20.01.2016 09:55	Example Multiscan	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	22,4	493	0,79
11	Triplescan	19.01.2016 11:13	Arm posterior	Muscle	Ext Digitor	Lying	Right	Central	Relaxed	19,7	418	0,94
12	Triplescan	19.01.2016 11:13	Arm posterior	Muscle	Ext Digitor	Lying	Left	Central	Relaxed	20,0	419	0,88
13	Triplescan	19.01.2016 11:13	Cycling	Muscle	Vastus m	Lying	Right	Central	Relaxed	19,7	411	0,90
14	Triplescan	19.01.2016 11:13	Cycling	Muscle	Vastus lt	Lying	Right	Central	Relaxed	19,6	406	0,90
15	Triplescan	19.01.2016 11:13	Cycling	Muscle	Vastus m	Lying	Left	Central	Relaxed	19,8	411	0,88
16	Triplescan	19.01.2016 11:13	Cycling	Muscle	Vastus lt	Lying	Left	Central	Relaxed	19,5	405	0,88
17	Multiscan	19.01.2016 11:13	165 test	Muscle	Bic Fem c l	Lying	Right	Central	Relaxed	19,3	397	0,89

To open the *Results* window, click the following icon in the main toolbar:



In the *Results* window, the measurement results are displayed in two different tabs:

1. *Measurement* tab,
2. *Results* tab.

**Filters in the *Results* window:**

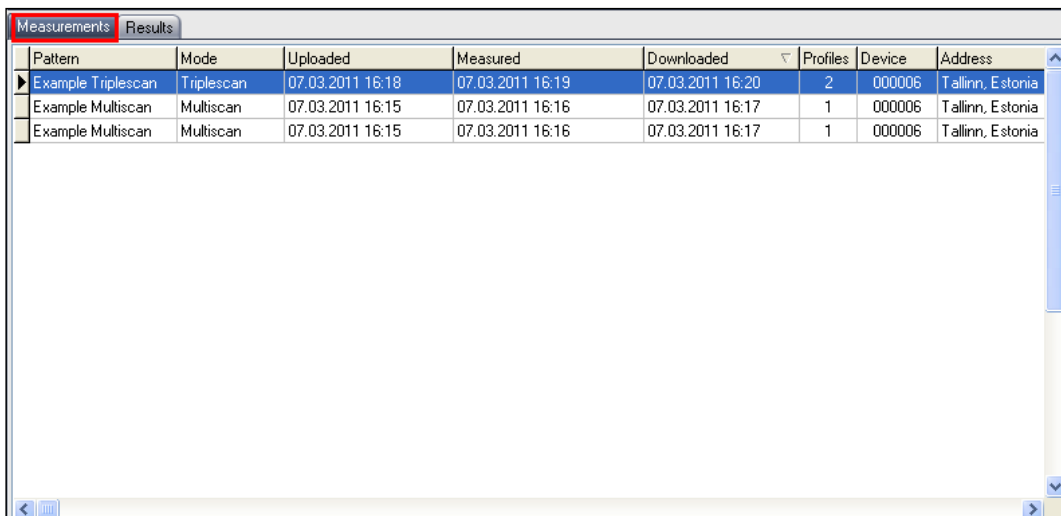
- **Subject group** (filter for Subject groups) – only the Subjects belonging to the selected group are displayed in the list Name & Birth;
- **Name & Birth** – the name and birth date of the Subject whose measurement results are displayed on the Measurements and Results tabs;
- **Pattern group** (filter for the Pattern groups) – only the Patterns belonging to the selected group are displayed in the Pattern filter list and only the measurement results belonging to the selected group are displayed on the Measurements and Results tabs;
- **Pattern filter** – only the measurement results of the selected Pattern are displayed on the Measurements and Results tabs;
- **Date Filter** – measurement results are displayed according to the determined interval:
  - **Since** – start date of the filter,
  - **To** – end date of the filter,

- **Set filter/Clear** – applies or clears the filter;
- **View Mode** – filter for the measurement mode:
  - **All** – all measurement results are displayed,
  - **Triplescan** – only *Triplescan* measurement results are displayed,
  - **Multiscan** – only *Multiscan* measurement results are displayed.

## Measurements tab

On the *Measurements* tab, the measurement results of the selected Subject are displayed by the Patterns used. Each row on the tab shows the time when a particular Pattern was used.

To open the *Measurements* tab, click the *Measurements* button in the *Results* window:



Pattern	Mode	Uploaded	Measured	Downloaded	Profiles	Device	Address
▶ Example Triplescan	Triplescan	07.03.2011 16:18	07.03.2011 16:19	07.03.2011 16:20	2	000006	Tallinn, Estonia
Example Multiscan	Multiscan	07.03.2011 16:15	07.03.2011 16:16	07.03.2011 16:17	1	000006	Tallinn, Estonia
Example Multiscan	Multiscan	07.03.2011 16:15	07.03.2011 16:16	07.03.2011 16:17	1	000006	Tallinn, Estonia

The *Measurements* tab is used for:

- creating a *Symmetry* report (chapter 3.6.2);

The mouse right-click pop-up menu on the for *Measurements* tab:

- **Select all** – selects all rows that are displayed on the *Measurements* tag;
- **Clear Selection** – deselects selected rows;
- **Export filtered results** – exports selected rows into a file format that can be opened with MS Excel;
- **Delete measurements row** – deletes the selected rows of measurements;
- **Field chooser** – opens the window columns where the columns to be displayed can be selected.

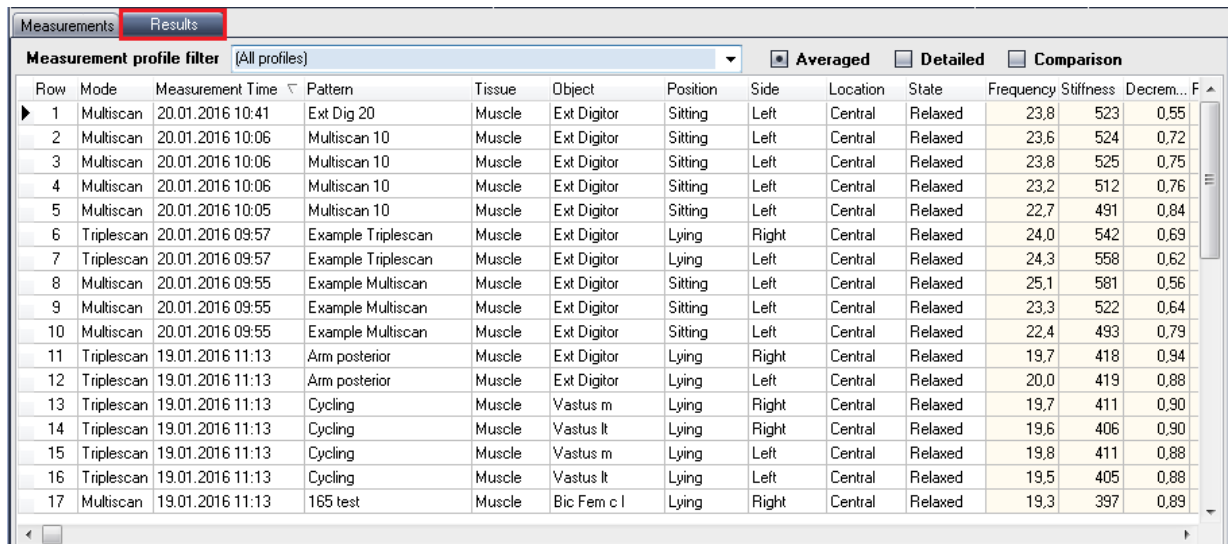


If a row of measurements is deleted from the *Measurements* tab, then that row's measurement results will also be deleted automatically from the *Results* tab.

## Results tab

On the *Results* tab, all measurement results for the selected Subject are displayed by a Measurement profile.

To open the *Results* tab, click the *Results* button in the *Results* window:



Row	Mode	Measurement Time	Pattern	Tissue	Object	Position	Side	Location	State	Frequency	Stiffness	Decrem...	F
1	Multiscan	20.01.2016 10:41	Ext Dig 20	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	23,8	523	0,55	
2	Multiscan	20.01.2016 10:06	Multiscan 10	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	23,6	524	0,72	
3	Multiscan	20.01.2016 10:06	Multiscan 10	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	23,8	525	0,75	
4	Multiscan	20.01.2016 10:06	Multiscan 10	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	23,2	512	0,76	
5	Multiscan	20.01.2016 10:05	Multiscan 10	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	22,7	491	0,84	
6	Triplscan	20.01.2016 09:57	Example Triplscan	Muscle	Ext Digitor	Lying	Right	Central	Relaxed	24,0	542	0,69	
7	Triplscan	20.01.2016 09:57	Example Triplscan	Muscle	Ext Digitor	Lying	Left	Central	Relaxed	24,3	558	0,62	
8	Multiscan	20.01.2016 09:55	Example Multiscan	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	25,1	581	0,56	
9	Multiscan	20.01.2016 09:55	Example Multiscan	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	23,3	522	0,64	
10	Multiscan	20.01.2016 09:55	Example Multiscan	Muscle	Ext Digitor	Sitting	Left	Central	Relaxed	22,4	493	0,79	
11	Triplscan	19.01.2016 11:13	Arm posterior	Muscle	Ext Digitor	Lying	Right	Central	Relaxed	19,7	418	0,94	
12	Triplscan	19.01.2016 11:13	Arm posterior	Muscle	Ext Digitor	Lying	Left	Central	Relaxed	20,0	419	0,88	
13	Triplscan	19.01.2016 11:13	Cycling	Muscle	Vastus m	Lying	Right	Central	Relaxed	19,7	411	0,90	
14	Triplscan	19.01.2016 11:13	Cycling	Muscle	Vastus lt	Lying	Right	Central	Relaxed	19,6	406	0,90	
15	Triplscan	19.01.2016 11:13	Cycling	Muscle	Vastus m	Lying	Left	Central	Relaxed	19,8	411	0,88	
16	Triplscan	19.01.2016 11:13	Cycling	Muscle	Vastus lt	Lying	Left	Central	Relaxed	19,5	405	0,88	
17	Multiscan	19.01.2016 11:13	165 test	Muscle	Bic Fem c l	Lying	Right	Central	Relaxed	19,3	397	0,89	

The *Results* tab is used for:

- creating a *Multiscan Analysis* report (chapter 3.6.3);
- creating graphs (chapter 3.6.5);
- viewing acceleration signals (chapter 3.6.6);
- creating a *Comparison* report (chapter 3.6.4);
- creating Estimation graphs (chapter 3.6.7);
- export of the measurement results (chapter 3.7.2).

Functions of the *Pop-up* window (click mouse right button) on the *Results* tab:

- **Select all** – selects all rows that are displayed on the *Results* tab;
- **Clear Selection** – deselects all the selected rows;
- **Export filtered results** – exports selected rows that can be opened with MS Excel;
- **Copy selection** – copies the selected rows;
- **Field chooser** – opens the window columns where the columns to be displayed can be selected.

Additional functions on the *Results* tab:

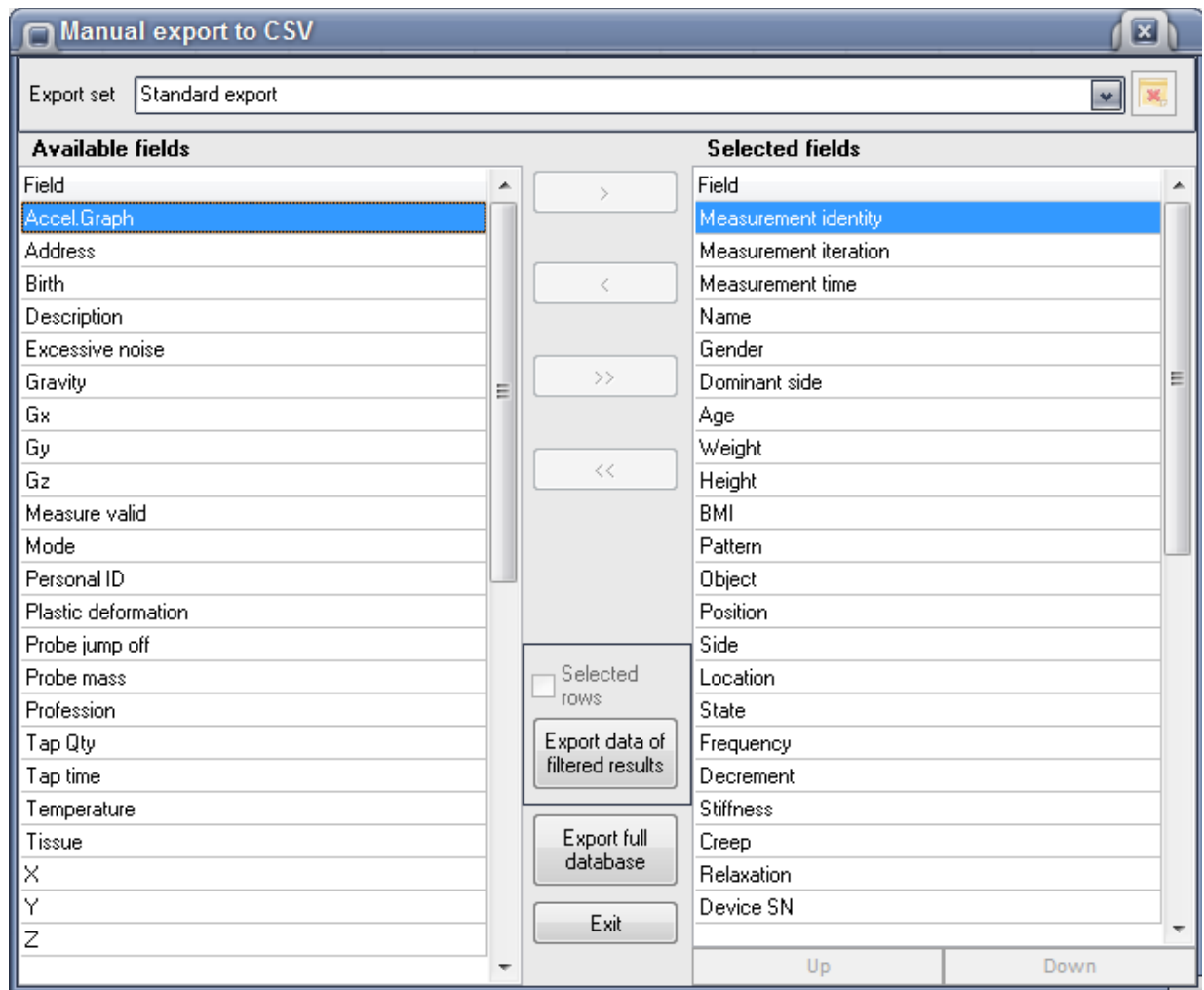
- **Measurement profile filter** – shows the measurement results of the selected Measurement profile;
- **Comparison** – shows the Difference % of two measurement results and allows composing a comparison report. The difference up to 5% is shown in green, 5-10% in yellow and above 10% in red;
- **Detailed view** – all single measurements of the series are displayed in the *Results* tab.



Only two rows of measurement results can be activated if the *Comparison field* is activated. To activate more rows, deactivate the *Comparison field*.

## 4.2.4 Manual export to CSV

Manual export window (chapter 3.7.2):



To open the *Manual export to CSV* window, click the *Manual export* icon on the main toolbar (chapter 3.7.2, point 3).

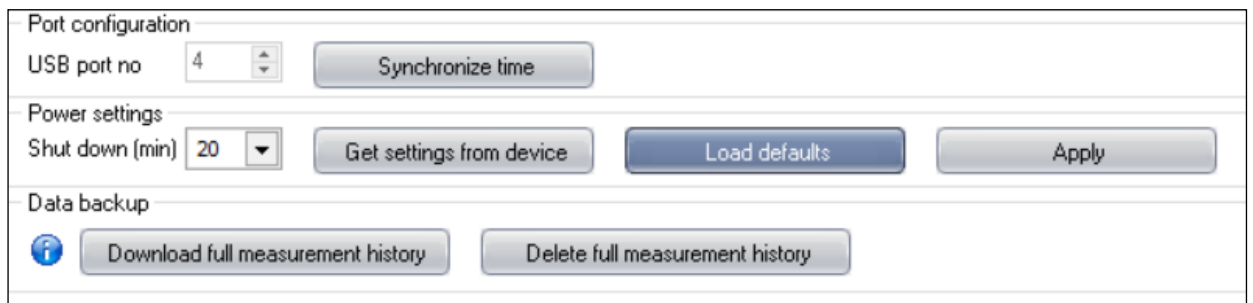
The *Manual export* window contains the following buttons and functions:

- **Arrow right >** – sends selected data field from the section *Available fields* to the section *Selected fields* to be exported;
- **Arrow left <** – sends the selected field from the section *Selected fields* back to the section *Available fields*;

- **Double arrow right >>** – sends all the data fields from the section *Available fields* to the section *Selected fields* to be exported;
- **Double arrow left <<** – sends all the data fields from the section *Selected fields* back to the section *Available fields*;
- **Export data of filtered results** – exports filtered Measurement results from the *Results* tab;
- **Selected rows** – if checked, then only the selected results in the *Results* tab will be exported by clicking the button *Export data of filtered results*;
- **Export full database** – exports the full database of the Measurements results of Selected data fields;
- **Up** – moves the selected data field upwards;
- **Down** – moves the selected data field downwards.

## 4.2.5 Device settings window

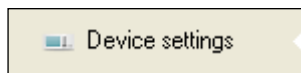
The Device settings can be changed in the *Device settings* window of the *Settings* menu:



The screenshot shows the 'Device settings' window with three main sections:

- Port configuration:** Includes a 'USB port no' dropdown menu set to '4' and a 'Synchronize time' button.
- Power settings:** Includes a 'Shut down (min)' dropdown menu set to '20', and buttons for 'Get settings from device', 'Load defaults', and 'Apply'.
- Data backup:** Includes an information icon and two buttons: 'Download full measurement history' and 'Delete full measurement history'.

To open the *Device settings*, click the following button in the *Settings* menu:



To apply the Device power settings, switch the Device ON and connect to the computer via USB. After changing the settings, click *Apply*.

**Functions of the *Port configuration* section:**

- **USB port no** – determines the port for the Device and Software communication;
- **Synchronize time** – synchronises the computer's Date & Time setting with the Device.

**Functions of the *Power settings* section:**

- **Shut down [min]** – determines the automatic shut-down of the Device after the last action;
- **Get settings from device** – shows the Power setting for the connected Devices;
- **Load defaults** – loads the default value into the Power settings cells.

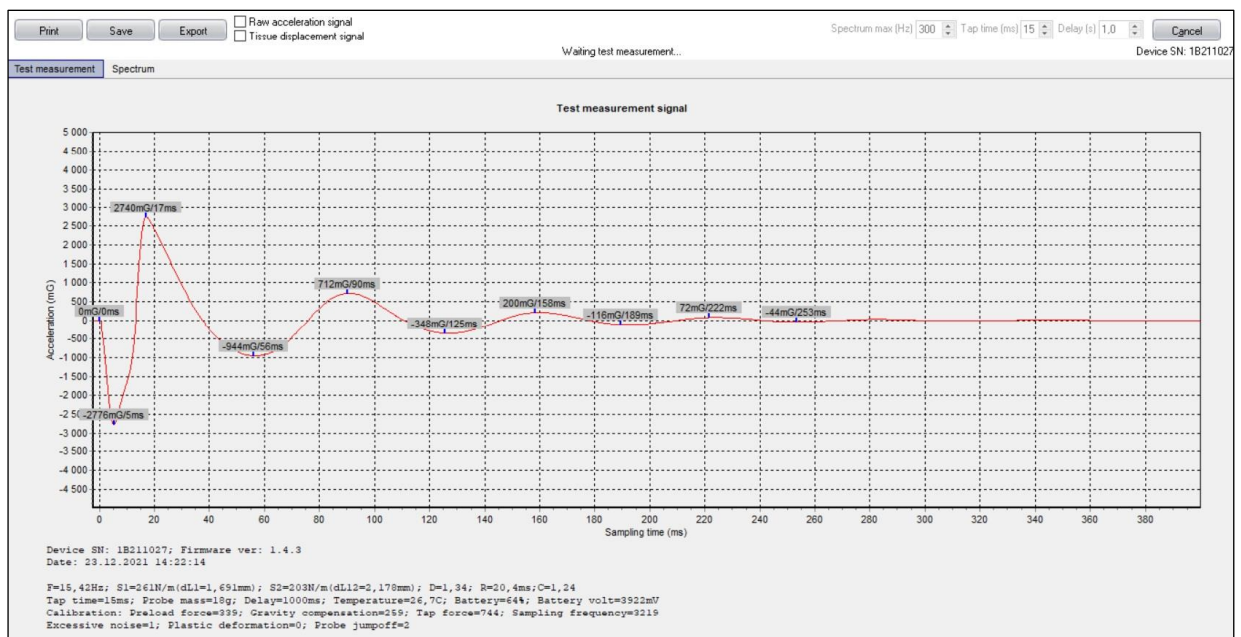
The functions of the *Data backup* section:

- **Download entire measurement history** – downloads the entire measurement history from the connected Device;
- **Delete entire measurement history** – deletes the entire measurement history from the connected Device.

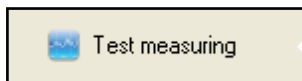
## 4.2.6 Test measuring window

Test mode enables to display measurement results, raw acceleration signal<sup>32</sup> and the signal spectrum of the test measurement (chapter 3.3.2).

The acceleration signal can be displayed in the *Test measuring* window:



To open the *Test measuring* window, click the following button in the *Settings* menu:



The acceleration signal and the signal spectrum are displayed in the following tabs:

1. *Test Measurement*,
2. *Spectrum*.

The functions of the *Test measuring* window:

- **Print** – prints the Test measuring protocol;

<sup>32</sup> The evaluation criteria for the acceleration signal are provided in chapter 2.3.



- **Save** – saves the Test measuring protocol in the JPG format;
- **Export** – exports the acceleration signal into the CSV format;
- **Raw acceleration signal** – displays the raw acceleration signal along with the filtered signal;
- **Tissue displacement signal** – displays the displacement signal along with the acceleration signal;
- **Spectrum max [Hz]** – changes the upper limit value for the Y-axis in the Spectrum tab;
- **Tap time** – changes the measurement impulse time;
- **Delay** – changes the time delay preceding the measurement impulse.
- **Test/Cancel** – starts or cancels the Test measurement.

## 4.2.7 Locations window

Locations of the Software files are shown in the *Locations* window of the *Settings* menu:

Database location	
Database	C:\Program Files (x86)\MyotonPRO\MyotonDB.sdf
Automatic export location	
MS Excel file	C:\Program Files (x86)\MyotonPRO\MyotonDB.xlsm
Help location	
Help file	http://www.myoton.com/UserFiles/Updates/MyotonPRO_User_Manual.pdf
Log file locations	
Application	C:\Program Files (x86)\MyotonPRO\Logs\app_%y_%m_%d.log
Data access	C:\Program Files (x86)\MyotonPRO\Logs\data_%y_%m_%d.log
Data read	C:\Program Files (x86)\MyotonPRO\Logs\read_%y_%m_%d.log
Device communication	C:\Program Files (x86)\MyotonPRO\Logs\mkcl_%y_%m_%d.log
Leave blank to disable logging	
Apply	

To open the *Locations* window, click the following button in the *Settings* menu:



- **Database** – name and location of the connected database.
- **MS Excel file** – shows the name and location of the XLSM file for the automatic export.
- **Help file** – shows the location of the User Manual.  
Given address: [http://www.myoton.com/UserFiles/Updates/MyotonPRO\\_User\\_Manual.pdf](http://www.myoton.com/UserFiles/Updates/MyotonPRO_User_Manual.pdf) provides access to the latest User Manual.
- **Application** – name and location of the Software functioning log file;
- **Data access** – name and location of the log file for the events related to the manipulation of the Software data;
- **Data read** – name and location of the log file of the events related to the reading of data;

- **Device communication** – name and location of the log file for the events related to the communication with the *Device*.

**COMMENT** The default database file of the Software is **MyotonDB.sdf** (stored in the directory: C:\Program Files (x86)\MyotonPRO\Myoton\_DB.sdf).

NB! Many database files can be created and used in the Software (chapter 3.8.3).

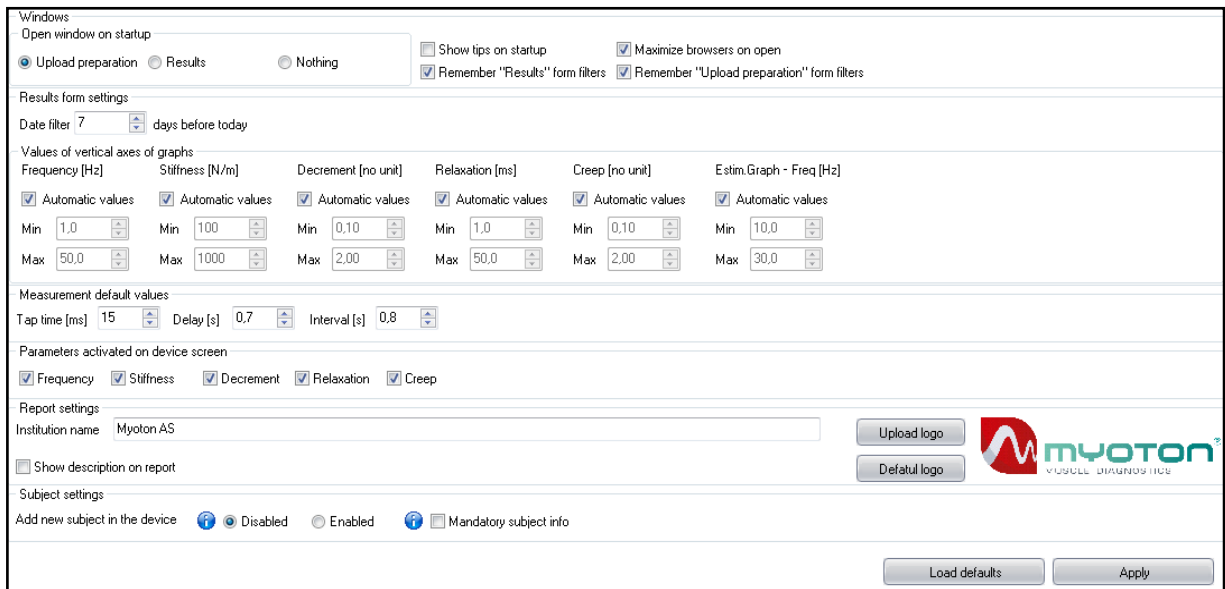


To save and apply the changes of application settings, click **Apply**.

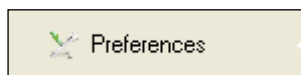
If the location of log files is incorrect, the functioning of the Software can be affected.

## 4.2.8 Preferences window

The configuration of the Software can be performed in the *Preferences* window of the menu *Settings* window:



To open *Preferences*, click the following button in the *Settings* menu:



To save and apply the changes of application settings, click **Apply**.

**COMMENT** To apply the default settings of the Software, click the *Load defaults* button.

**Windows section:**

- **Open window on start-up** – determines which window opens automatically on start-up.
- **Show tips on start-up** – shows *Tips and Suggestions* on start-up of the Software.
- **Maximize browsers on open** – maximises the *Upload preparation* and *Results* windows.
- **Remember “Results” form filters** – remembers applied filters in the *Results* window after shutting down the Software.
- **Remember “Upload preparation” form filters** – remembers applied filters in the *Upload preparation* window after shutting down the Software.

**Settings of the Results form:**

- **Date filter - days before today** – determines the start date in the *Date filter*.

**Values for charts vertical axis:**

- **Automatic values** – the default upper and lower limits of the vertical axis. The automatic option provides  $\pm 10\%$  from max and min values plotted on the chart.
- **Min Max** – To change the limits, uncheck the checkbox, type a new **Min Max** value and click *Apply*.

**Measuring default values:**

- **Tap time [ms]** – the default Tap time (**15ms**) in the *Pattern composer*.
- **Delay [s]** – the default time delay (**0.7s**) of the measurement series.
- **Interval [s]** – the default interval (**0.8s**) of the measurement series in the *Pattern composer*.

**Parameters displayed on Device screen:**

- **F, S, D, R, C** – determines which parameters of the measurement results will be displayed on the Device screen. NB! All parameters are always calculated regardless of which ones are displayed.

**Report settings:**

- **Institution name** – the name of the user organisation that will be displayed on the reports of the measurement results (e.g. on *Multiscan analysis* report).
- **Show description on the report** – before generating the reports, the *Description window* opens to enter a comment for the reports.
- **Upload logo** – enables to upload a logo that will be displayed on the reports of the measurement results.
- **Default logo** – the default logo that will be displayed on the reports of the measurement results.

**Subject settings:**

- **Add new subject in the device** – Enables/Disables the function “add new subject in the Device” without Myoton Desktop Software. This function should be activated if the computer for upload is not available!
- **Mandatory subject info** - Uncheck of the checkbox enables to add subjects in the Software with the first name only. All other subject info data fields remain optional.

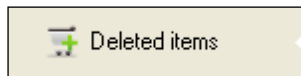
## 4.2.9 Deleted items window

Deleted data can be restored under *Deleted items* window in the *Settings* menu:

Deleted subjects						
First name	Surname	Gender	Person...	Date of B...	Deleted	Restore
▶ Test	Subject	Male		1.01.1980	21.02.2014 15:53	<input type="checkbox"/>

Deleted objects				
Tissue	Body region	Object	Deleted	Restore
▶				

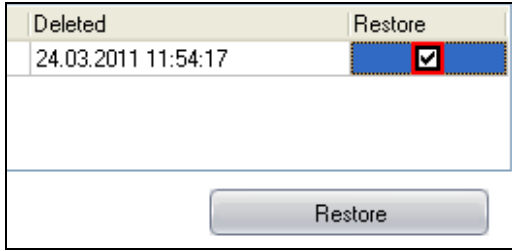
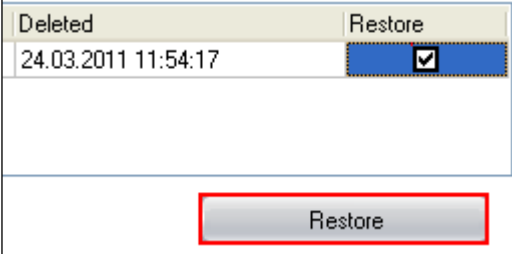
To open *Deleted items*, click the following button in the *Settings* menu:



Following deleted items can be restored in the *Deleted items* window:

- Deleted subjects,
- Deleted objects,
- Deleted patterns,
- Deleted measurements.

To restore deleted items:

Description of the action	Screen image
1. Double-check the row in the <i>Restore</i> section.	
2. Click the <i>Restore</i> button:	

# Appendixes

## Appendix 1 – Definitions of parameters

### STATE OF TENSION

#### **F – Oscillation Frequency [Hz]**

*Characterizes the intrinsic tension of biological soft tissues on the cellular level.*

*Oscillation Frequency characterizes the tone of superficial skeletal muscles in their passive or resting state without voluntary contraction (EMG signal silent).*

The Oscillation Frequency of a muscle in its contracted state characterizes state of tension.

### BIOMECHANICAL PROPERTIES

#### **S – Dynamic Stiffness [N/m]**

*Characterizes the resistance of biological soft tissues to a force of deformation.*

The term Dynamic Stiffness originates from the dynamic measurement method applied in Myoton technology. The inverse of stiffness is compliance.

#### **D – Logarithmic Decrement**

*Characterizes the dampening of tissue oscillation.*

*The faster the tissue oscillation fades, the higher the dissipation of mechanical energy induced by the measurement impulse. The decrement of tissues' natural oscillation inversely describes elasticity.*

*Elasticity is the biomechanical property of soft tissues that characterizes the ability to recover its initial shape from being deformed.*

The higher the decrement, the lower the elasticity. In theory, a decrement of zero (0) represents absolute elasticity (absence of dampening). The inverse of elasticity is plasticity.

### VISCO-ELASTIC PROPERTIES

#### **R – Mechanical Stress Relaxation Time [ms]**

*Characterizes tissue's recovery time from displacement.*

The higher a tissue's tension or stiffness, the faster a tissue recovers its shape, meaning the lower the R value.

#### **C – Ratio of Relaxation and Deformation time**

*Characterizes creep, the gradual elongation of tissue over time when placed under constant tensile stress.*

The higher a tissue's tension, structural integrity or stiffness, the higher its resistance to creep, meaning the lower the C value.

# Appendix 2 – The formulas and Legend of the Acceleration Signal

**Frequency [Hz]:**

$F = f_{\max}$  (signal spectrum computed by Fast Fourier transform - FFT)

**Mechanical Stress Relaxation time [ms]:**

$R = t_R - t_1$

**Dynamic Stiffness [N/m]:**


$S = a_{\max} \cdot m_{\text{probe}} / \Delta l$

**The ratio of the Relaxation and Deformation time:**

$C = R / (t_1 - t_T)$

**Logarithmic Decrement:**

$D = \ln(a_1 / a_3)$

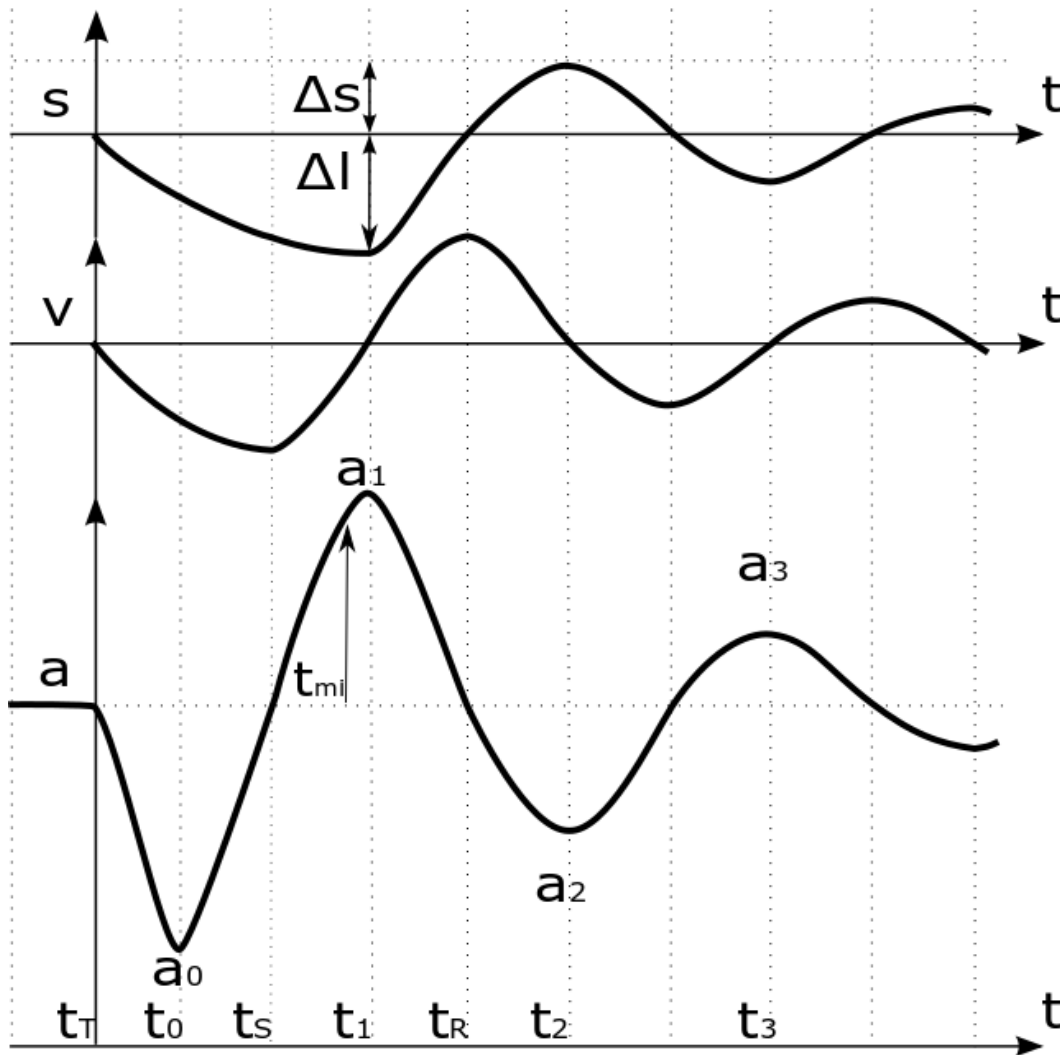

**Figure 5:** Relationship of the displacement (**S**), velocity (**V**), and acceleration (**a**).

After a quick-released measurement impulse, the tissue being measured responds immediately in the form of a damped co-oscillation.

**The co-oscillation includes:**

- a) a tissue being measured,
- b) the pre-compressed subcutaneous tissue layers above a tissue structure being measured,
- c) the Device probe and measurement system.

**Oscillation of a soft biological tissue is registered in the form of an acceleration signal (a).**

**The legend of the figure:**

Symbol	Description
a	Acceleration of the damped oscillation – unit: milli G [mG]
a <sub>0</sub>	Maximum acceleration
a <sub>1</sub>	Maximum displacement i.e. maximum tissue resistance measured in milli g [mG]
a <sub>2</sub>	Maximum opposite displacement due to the residual inertia of the tissue oscillation
a <sub>3</sub>	Maximum displacement of the second oscillation period
ΔS	Pre-compression of subcutaneous tissues above a tissue structure being measured
Δl	Maximum displacement of a tissue
m <sub>probe</sub>	Mass of the measurement mechanism
S	Displacement : mm
t	Time in milli seconds [ms]
t <sub>mi</sub>	End of the mechanical impulse
t <sub>0</sub>	The timepoint of maximum acceleration
t <sub>↑</sub>	Start of the mechanical impulse
t <sub>r</sub>	The timepoint of a tissue recovery from being deformed
t <sub>s</sub>	The timepoint of maximum velocity and acceleration shift to deceleration
t <sub>t</sub> – t <sub>0</sub>	The time between zero to maximum acceleration
t <sub>0</sub> – t <sub>s</sub>	The time between maximum acceleration to zero acceleration
t <sub>s</sub> – t <sub>1</sub>	The time from maximum velocity to complete stop
t <sub>1</sub> – t <sub>r</sub>	The time between maximum displacement and full recovery.
t <sub>r</sub> – t <sub>2</sub>	The time between recovered initial shape and opposite maximum displacement
t <sub>1</sub>	The timepoint of maximum displacement (deformation)
t <sub>2</sub>	The timepoint of opposite maximum displacement (deformation)
t <sub>3</sub>	The timepoint of maximum displacement of the second oscillation period
t <sub>2</sub> – t <sub>3</sub>	The time of two consecutive displacements
V	Oscillation velocity: m/s

## Appendix 3 – Technical Specification

DEVICE TYPE AND MEASUREMENT METHOD	
<b>Classification Category</b>	<ul style="list-style-type: none"> <li>For Research Use Only</li> <li>Monitoring device for non-vital physiological parameters</li> </ul>
<b>Device</b>	<ul style="list-style-type: none"> <li><b>Digital palpation device</b> for <i>in vivo</i> non-invasive measurement of soft biological tissues</li> </ul>
<b>Measurement method</b>	<ul style="list-style-type: none"> <li>Mechanical Dynamic Response method</li> </ul>
<b>Measurable parameters</b>	<ul style="list-style-type: none"> <li><b>State of tension:</b> <ul style="list-style-type: none"> <li>Natural Oscillation Frequency [Hz] - characterizes Tone or Sate of Tension</li> </ul> </li> <li><b>Biomechanical properties:</b> <ul style="list-style-type: none"> <li>Dynamic Stiffness [N/m]</li> <li>Logarithmic Decrement of the oscillation [relative arbitrary unit] characterizes Elasticity and dissipation of mechanical energy</li> </ul> </li> <li><b>Viscoelastic properties:</b> <ul style="list-style-type: none"> <li>Mechanical Stress Relaxation time [ms]</li> <li>Ratio of Relaxation and Deformation time [relative arbitrary unit] characterizes Creep</li> </ul> </li> </ul>
<b>Measurement range Min – Max (limited to)</b>	<ul style="list-style-type: none"> <li>Natural oscillation frequency - 6 – 80 Hz (110 Hz)</li> <li>Dynamic Stiffness - 70 – 2000 N/m (3000 N/m)</li> <li>Logarithmic Decrement - 0.20 – 3.50</li> <li>Mechanical Stress Relaxation time - 3.50 – 50 ms</li> <li>Creep - 0.30 – 2.80</li> </ul>
<b>Parameter's Coefficient of Variation</b>	<ul style="list-style-type: none"> <li>Natural Oscillation Frequency - 0.6%</li> <li>Dynamic Stiffness - 1.7%</li> <li>Logarithmic Decrement - 2.0%</li> <li>Mechanical Stress Relaxation time - 1.2%</li> <li>Creep - 1.5%</li> </ul>
<b>Raw signal processing</b>	<ul style="list-style-type: none"> <li>Proprietary algorithm</li> </ul>
MEASUREMENT SYSTEM	
<b>Measurement mechanism</b>	<ul style="list-style-type: none"> <li>System of frictionless translation motion, driven by solenoid actuator</li> </ul>
<b>Measurement sensor</b>	<ul style="list-style-type: none"> <li>3-axis Digital Acceleration Sensor,</li> <li>Output resolution 13-bit at <math>\pm 8</math> g</li> <li>Sensitivity range due to temp. <math>\pm 0.01\%</math></li> </ul>



<b>Measurement probe</b>	<ul style="list-style-type: none"> <li>• PMMA – (Polymethyl methacrylate), Standard diameter D=3 mm</li> <li>• Replaceable by user</li> </ul>
<b>Probe drive</b>	<ul style="list-style-type: none"> <li>• Shock protected servo mechanism</li> </ul>
<b>Probe positions</b>	<ul style="list-style-type: none"> <li>• Transport position: probe positioned inside the device</li> <li>• Replacement position: probe max exertion 14 mm</li> </ul>
<b>Pre-load force</b>	<ul style="list-style-type: none"> <li>• 0,18 [N] – automatic and independent of the measurement direction</li> </ul>
<b>Tap force</b>	<ul style="list-style-type: none"> <li>• 0,42 [N] – automatic and independent of the measurement direction</li> </ul>
<b>Measurement impulse time duration</b>	<ul style="list-style-type: none"> <li>• Measurement impulse duration is 15 ms (default value)</li> <li>• Automatically reduced impulse times are from 15 to 13, 10, 7 and 6 ms</li> <li>• User configurable from 5 to 15 ms</li> </ul>
<b>Tissue oscillation recording time</b>	<ul style="list-style-type: none"> <li>• 400 ms</li> </ul>
<b>Measurement position tolerance</b>	<ul style="list-style-type: none"> <li>• ±1,5mm in measurement direction</li> </ul>
<b>Measurement direction</b>	<ul style="list-style-type: none"> <li>• Any-directional measurements enabled from 0 to 1,3G gravity</li> </ul>
<b>Measurement sequence</b>	<ul style="list-style-type: none"> <li>• Automatic, pre-defined by user</li> </ul>
<b>Calibration</b>	<ul style="list-style-type: none"> <li>• Pre-calibrated by the manufacturer</li> </ul>
<b>Measurement assistance</b>	<ul style="list-style-type: none"> <li>• Probe light, audio and user interface assistance</li> </ul>

## POWER AND BATTERY

<b>Battery</b>	<ul style="list-style-type: none"> <li>• Built-in rechargeable Lithium Polymer Battery</li> <li>• 2260mAh, 3.7V, 8.4Wh</li> <li>• Replaceable only by the manufacturer</li> </ul>
<b>Charger</b>	<ul style="list-style-type: none"> <li>• AC-DC Adapter, Input: AC 100-240V, 50/60 Hz, 0.3 – 0.15A</li> <li>• Output: DC 5V...1A, USB type A socket</li> <li>• Interchangeable AC plug type: EU, US, AU, GB</li> </ul>
<b>Recharge time</b>	<ul style="list-style-type: none"> <li>• Max 5 hours</li> </ul>
<b>Recharge cycles</b>	<ul style="list-style-type: none"> <li>• Min 500 cycles</li> </ul>
<b>Operational time</b>	<ul style="list-style-type: none"> <li>• Min 8 hours (in case of normal workload)</li> <li>• Min 15 000 continuous measurements in horizontal position</li> </ul>

- |                         |   |
|-------------------------|---|
| <b>Power management</b> | <ul style="list-style-type: none"> <li>• User configurable</li> <li>• Automatic shut-down in 20 minutes (adjustable)</li> </ul> |
|-------------------------|---|

## DISPLAY AND DATA TRANSFER

- |                            |  |
|----------------------------|--|
| <b>Display</b>             | <ul style="list-style-type: none"> <li>• LCD display 2.8" diagonal, Capacitive Touch Panel</li> <li>• LED Backlight, High Brightness, Full view angle</li> </ul> |
| <b>Data storage</b>        | <ul style="list-style-type: none"> <li>• Internal memory available for minimum 500 000 measurements.</li> </ul>  |
| <b>Data transfer cable</b> | <ul style="list-style-type: none"> <li>• USB Type A plug to Type C plug</li> </ul>   |

## DEVICE FIRMWARE

- |                         |  |
|-------------------------|--|
| <b>Operating system</b> | <ul style="list-style-type: none"> <li>• Proprietary Linux-based operating system</li> </ul>           |
| <b>Content upload</b>   | <ul style="list-style-type: none"> <li>• Subject and measurement profiles</li> </ul>                   |
| <b>Content download</b> | <ul style="list-style-type: none"> <li>• Measurement results</li> </ul>                                |
| <b>Date &amp; time</b>  | <ul style="list-style-type: none"> <li>• Configurable via computer Date &amp; Time settings</li> </ul> |

## ENVIRONMENTAL REQUIREMENTS

- |                              |   |
|------------------------------|---|
| <b>Operating conditions</b>  | <ul style="list-style-type: none"> <li>• in-door use</li> </ul>                 |
| <b>Operating temperature</b> | <ul style="list-style-type: none"> <li>• -5 to +50° C (23 to 122° F)</li> </ul> |
| <b>Relative air humidity</b> | <ul style="list-style-type: none"> <li>• 0 – 95 % RH not condensed</li> </ul>   |

## DEVICE HOUSING

- |                 |   |
|-----------------|---|
| <b>Height</b>   | <ul style="list-style-type: none"> <li>• 162 mm</li> </ul>  |
| <b>Width</b>    | <ul style="list-style-type: none"> <li>• 67 mm</li> </ul>   |
| <b>Depth</b>    | <ul style="list-style-type: none"> <li>• 29 mm</li> </ul>   |
| <b>Device</b>   | <ul style="list-style-type: none"> <li>• 240 g</li> </ul>   |
| <b>Colour</b>   | <ul style="list-style-type: none"> <li>• Combination of white, green and grey</li> </ul>                    |
| <b>Material</b> | <ul style="list-style-type: none"> <li>• Polycarbonate UV Stabilized and Elastomer thermoplastic</li> </ul> |

END OF SPECIFICATION

# Appendix 4 – List of Error Messages

## Error Message Severity and Priority

Error messages displayed on the Device screen are written to the error log without being displayed on the screen. For error messages that are displayed, the box title indicates the severity of the message.

**WARNING!** – indicates that the Device requires attention soon. It does not affect the results of the Subject being measured or the operation of the Device.

**ERROR!** - requires the operator's attention but does not stop the entire Device from running except in case of Boot errors.

**DETECTION MESSAGE!** – informs the operator, corrective action may be needed.

**BOOT ERROR!** – requires the operator to contact support if the problem persists.

**SEVERE ERROR!** - may stop the Device and requires the operator's immediate attention.

**The Device does not contain error messages which may cause danger to an operator or the Subject.**

**Device temperatures given in Warning or Error messages are in Celsius.**

**Too high temperature above 60 degrees Celsius may damage the battery inside the Device.**

Error message from 26 to 36 may occur when the Device hardware errors cause the Device firmware communication issues. If the message persists, please contact support!



**In case the Device does not start or the firmware freezes, press the RESET button (chapter 1.4.2 p. 17) on the reverse side of the Device by using any rod (e.g. straighten paper-clip) with diameter less than 1,5 mm.**

Seq no	Error window title	Error Message
1	Warning!	Battery is empty! Please re-charge the battery! Shutting down!
2	Warning!	Device temperature (X) is high! Please shut down for cooling!
3	Severe Error!	Device temperature (X) exceeds allowed maximum! Shutting down for cooling!
4	Warning!	Device temperature (X) is low! Please regain working temperature!
5	Severe Error!	Device temperature (X) is too low for normal functioning! Regain working temperature! Shutting down!
6	Error!	Unable to read the device settings. If the problem persists, please contact support.
7	Error!	Please remove any obstacle blocking the probe from being exerted. If the problem persists, please contact support.
8	Error!	Please remove any obstacle blocking the probe from being retracted. If the problem persists, please contact support.
9	Warning!	The probe was pushed in. Do not use excessive force when replacing the probe!
10	Error!	Please upload a Measurement set to perform measurements!

11	Error!	Subject database full. To add a new Subject on the Device, please upload a new Measurement set with less than 50 Subjects!
12	Error!	Database error occurred while adding a new Subject. If the problem persists, please contact support.
13	Error!	Unable to retrieve information from the database. If the problem persists, please contact support.
14	Error!	Unable to read measurements from the database. If the problem persists, please contact support.
15	Error!	Unable to process measurement data. If the problem persists, please contact support.
16	Error!	Unable to read the required information from the database. If the problem persists, please contact support.
17	Error!	No measurement profiles defined in the patterns. Please upload valid pattern.
18	Error!	Unable to store measurements to database. If the problem persists, please contact support.
19	Error!	This device is not calibrated. Performing measurements with an un-calibrated device is not possible. Please contact support.
20	Error!	Unable to initialize a measurement subsystem. If the problem persists, please contact support.
21	Error!	Myoton device failed to perform measurement.
22	Detection message!	Optimizing impulse * <i>chapter 2.3.2</i>
23	Detection message!	Probe jump-off detected! * <i>chapter 2.3.3</i>
24	Detection message!	Excessive noise detected! * <i>chapter 2.3.4</i>
25	Error!	Measurement data invalid! * Obtained acceleration signal is not suitable for parameters computation.
26	Error!	Unable to initialize a measurement subsystem.
27	Boot error!	Unable to start up the power management. Shutting down!
28	Boot error!	Unable to start up the system monitor. Shutting down!
29	Boot error!	Unable to initialize databases. Shutting down!
30	Boot error!	Unable to initialize user interface. Shutting down!
31	Boot error!	Unable to initialize the device hardware. Shutting down!
32	Boot error!	Unable to initialize the LED driver. Shutting down!
33	Boot error!	Unable to initialize the real time clock. Shutting down!
34	Boot error!	Unable to initialize the sound driver. Shutting down!
35	Boot error!	Unable to initialize USB communication. Shutting down!
36	Boot error!	Unable to open the battery log. Shutting down!
37	System status!	This device is disabled! Please contact support!

## Appendix 5 – Environmental Conditions



**After the transportation of the Device in a temperature lower than -5°C (23°F), warm up the Device to the room temperature before switch on.**

Maximum Permitted Operational Temperature and Humidity	<p>Temperature range: -5 to 50°C (23 to 122°F) Humidity range: 0 to 95%RH (not condensed)</p> <p>No atmospheric pressure limits.</p>
Recommended Operational Temperature and Humidity	<p>Temperature range: +15 to 30°C (59 to 86°F) Humidity range: 10 to 75%RH</p> <p>No atmospheric pressure limits.</p>
Transportation	<p>Temperature range: -10 to 50°C (14 to 122°F) Humidity range: 10 to 95%RH (not condensed)</p> <p>No atmospheric pressure limits.</p>
Transportation package	<p>See handling symbols outside of the package.</p> <p>Temperature range: -10 to 50°C (14 to 122°F) Humidity range: 10 to 95%RH (not condensed)</p> <p>No atmospheric pressure limits.</p>
Storage	<p>Preferably store device in a room temperature. Temperature range: +15 to 30°C (59 to 86°F) Humidity range: 10 to 75%RH</p> <p>No atmospheric pressure limits.</p>

## Important information regarding Electro Magnetic Compatibility (EMC)

With the increased number of electronic devices such as computers and mobile (cellular) telephones, and other electronic devices in use may be susceptible to electromagnetic interference from other devices. Electromagnetic interference may result in incorrect operation of the device and create alterations of measurement results.

In order to regulate the requirements for EMC (Electro Magnetic Compatibility) with the aim to prevent unsafe product situations, the EN60601-1-2:2007 standard has been implemented. This standard defines the levels of immunity to electromagnetic interferences as well as maximum levels of electromagnetic emissions. This device manufactured by Myoton AS conforms to this EN60601-1-2:2007 standard for both immunity and emissions.

**Nevertheless, special precautions need to be observed:**

NB! Do not use devices, which generate strong electromagnetic fields near the device. This may result in incorrect operation of the unit. The recommendation is to keep a minimum distance of 2 meters. Verify the correct operation of the device in case the distance is shorter.

The MyotonPRO device EMC test report is available at request.

## Mechanical and electrical safety

Accordance with mechanical and electrical safety test program MyotonPRO is fully compliant with IEC 60601 1:2005 + Corr. 1:2006 + 2:2007 + AM1:2012; Cl. 7, 8, 9, 15.

The MyotonPRO palpation device does not compromise the safety of the Subject, device operator nor other electronic equipment.

The MyotonPRO device mechanical and electrical safety test report is available at request.

## Correct Disposal Electrical & Electronic Equipment



This marking shown on the Device or its documentation indicates that it should not be disposed with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this product from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources. This product should not be mixed with other commercial wastes for disposal. This product does not contain any hazardous substances.